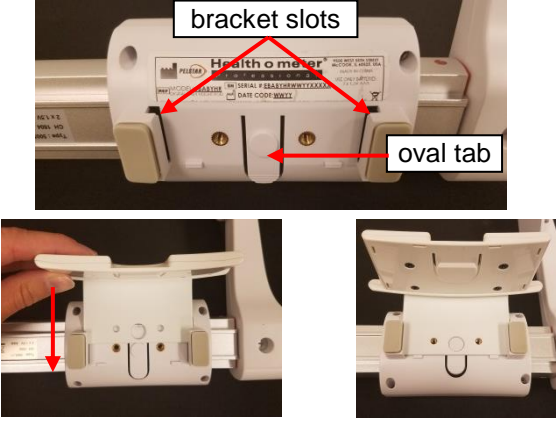
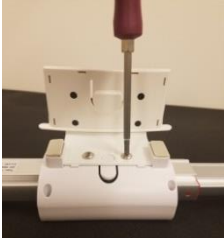

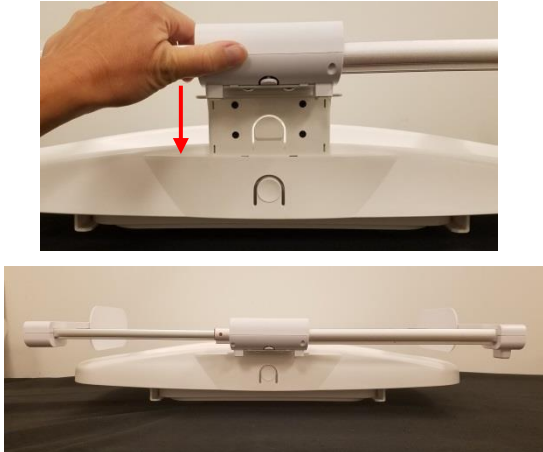


Instructions to Attach eBABYHR Digital Height Rod to 550KL Scale

Thank you for purchasing this Health o meter® Professional product. Please read these instructions carefully and keep it for easy reference or training.

Notes:

- Ensure that the scale is off during the height rod mounting process. If the scale was powered on during the process, be sure to turn the scale off and power on again before weighing the first patient.
- If the height rod needs to be removed, ensure the scale is off before removal and off before replacing the height rod onto the scale.

<p>1. Obtain the eBABYHR digital height rod. <i>(The height rod is sold separately from the 550KLEHRBRACKET).</i> Press down the oval tab on the back of the height rod display then slide the 550KLEHRBRACKET into the bracket slots. Slide the bracket until the three holes in the back of the bracket align with the screw holes and circle on the oval tab on the back of the display.</p>	 <p>The image shows a close-up of the back of the height rod display. A white bracket is being slid onto the display. Red arrows point to the 'bracket slots' on the display and the 'oval tab' on the back of the display. Below this are two smaller images: the left one shows a hand pressing down on the oval tab, and the right one shows the bracket fully attached to the display.</p>
<p>2. Attach the bracket to the eBABYHR display using the (2) included screws. Use a Phillips head screwdriver to tighten the screws.</p>	 <p>The image shows a Phillips head screwdriver being used to tighten a screw on the back of the height rod display. The screw is being inserted into one of the two screw holes on the back of the display.</p>
<p>3. Obtain the 550KL scale. Remove the plastic piece located on the top rear of the weighing tray. Ensure the scale is off.</p>	 <p>The image shows the top rear of the weighing tray. A red circle highlights a plastic piece that needs to be removed. The plastic piece is a small, rectangular piece of plastic that is attached to the top rear of the weighing tray.</p>
<p>4. Slide the height rod with bracket into the slots on the back of the 550KL scale. Slide down until you hear a click. The height rod is now mounted onto the scale. Note: Power the scale on and off once before first use.</p>	 <p>The image shows the height rod with bracket being inserted into the slots on the back of the 550KL scale. A hand is shown sliding the height rod down until it clicks into place. Below this is another image showing the height rod fully mounted onto the scale.</p>



PELSTAR, LLC
9500 West 55th St. McCook, IL 60525-7110 USA
1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR PRODUCT FOR WARRANTY COVERAGE AT:
www.homscases.com

Health o meter[®] is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter[®] Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter[®] Professional product features or specifications without notice.

© Pelstar, LLC 2019