

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. Calibration of your scale must be performed using kilograms (KG). 20 kilograms are required for calibration, applied in 5 kg increments.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the LB/KG button (522KL) or the blank blue button (522KG) and ZERO/TARE buttons at the same time, then press the ON/OFF button to turn on the scale. "CAL" will briefly appear on the display, followed by four numbers indicating that you can begin calibration. Release all three buttons.	"CAL" followed by "XXXX"
Press the ZERO/TARE button. "C-5" will flash on the display, followed by "0" or "1".	"C-5" then "0" or "1".
Place 5 kg on the scale. Four numbers will appear. When calibration numbers stabilize, press the ZERO/TARE button. "C-10" will appear on the display, followed by "0" or "1".	"XXXX" followed by "C-10" then "0" or "1".
Place an additional 5 kg on the scale, for a total of 10 kg. Four numbers will appear. When calibration numbers stabilize, press the ZERO/TARE button. "C-15" will appear on the display, followed by "0" or "1".	"XXXX" followed by "C-15" then "0" or "1".
Place an additional 5 kg on the scale, for a total of 15 kg. Four numbers will appear. When calibration numbers stabilize, press the ZERO/TARE button. "C-20" will appear on the display, followed by "0" or "1".	"XXXX" followed by "C-20" then "0" or "1".
Place an additional 5 kg on the scale, for a total of 20 kg. Four numbers will appear. When calibration numbers stabilize, press the ZERO/TARE button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"