

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. Calibration of your scale is performed using kilograms (KG). 20 kilograms total are required for calibrating, applied in 5 kg increments.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the ZERO/CLEAR as well as the kg/LB button. While holding down both buttons, momentarily press the ON/OFF button to power on the scale. When “CAL” appears on the display, release all buttons. Numbers will appear indicating that you can calibrate.	“CAL” followed by “XXXXX”
Press the ZERO/CLEAR button. “C-5” will flash on the display and then return to “0.”	“C-5” then “0”
Place 5 kg on the scale and when the calibration number stabilizes, press the ZERO/CLEAR button. “C-10” will flash on the display and then the display will return to “0”.	“C-10” then “0”
Place an additional 5 kg on the scale for a total of 10 kg. When the calibration number stabilizes, press the ZERO/CLEAR button. “C-15” will flash on the display and then the display will return to “0”.	“C-15” then “0”
Place an additional 5 kg on the scale for a total of 15 kg. When the calibration number stabilizes, press the ZERO/CLEAR button. “C-20” will appear on the display, followed by “0”	“C-20” then “0”
Place an additional 5 kg on the scale for a total of 20 kg. When the calibration numbers stabilize, press the ZERO/CLEAR button. “END” will appear on the display and the scale will automatically shut off.	“END”