DIGITAL PEDIATRIC SCALE
OPERATION MANUAL FOR REFERENCE & TRAINING

Health o meter®
Professional

8320KL

Professional and Portable
Capacity 33lb / 15kg

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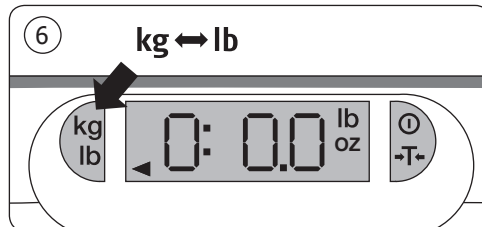
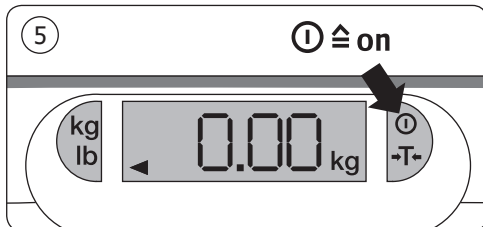
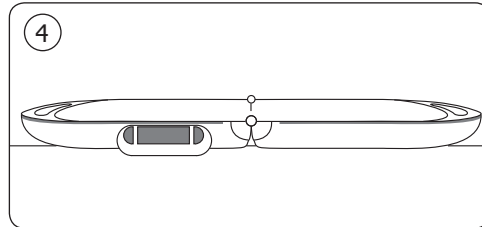
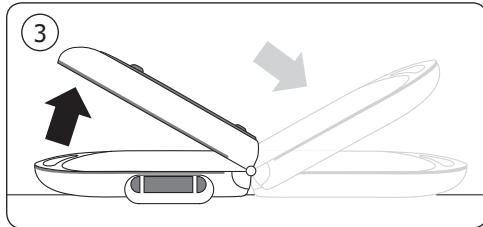
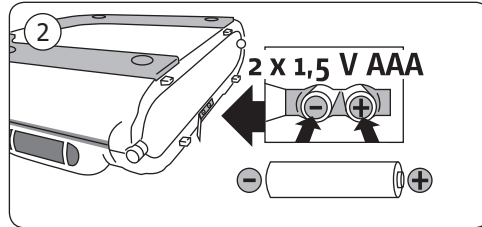
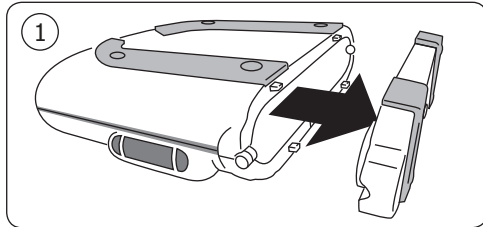
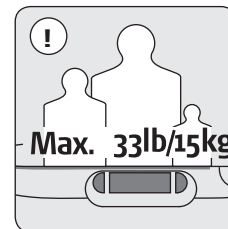
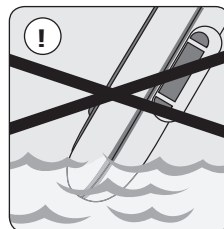
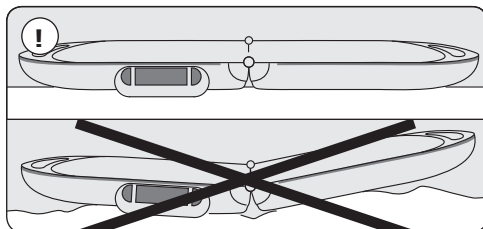
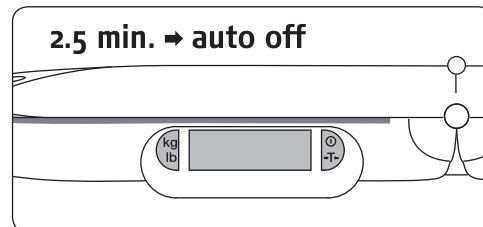
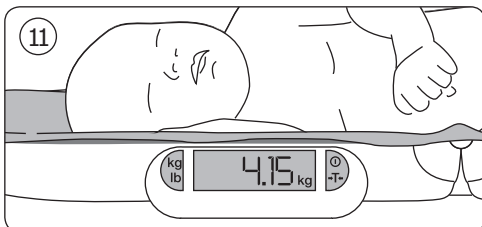
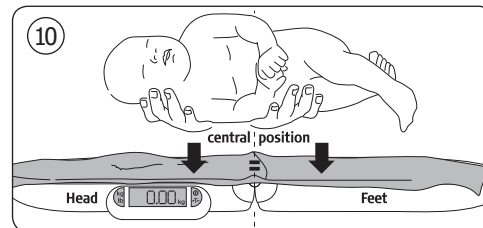
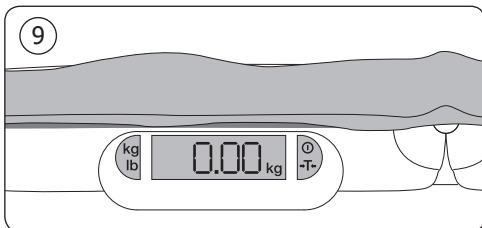
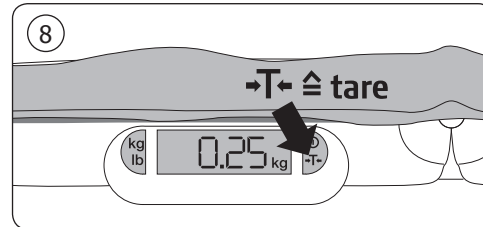
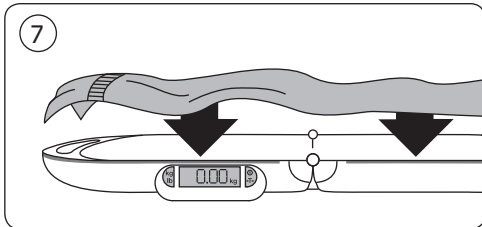
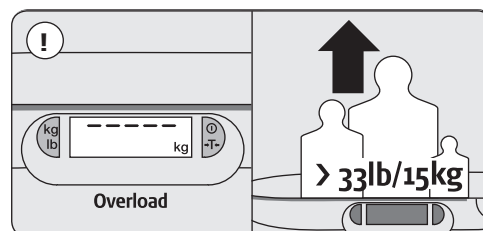
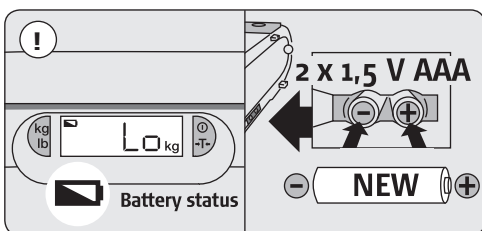
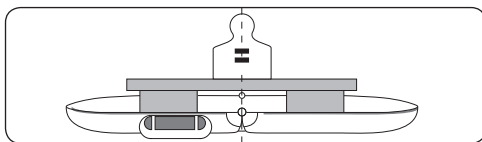
English

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated following this procedure.

1. Press ON key to power on the scale.
2. Before the display reads zero, press the KG/LB button three times. On the third push hold down the LB/KG button until the display reads "CAL".
3. When the display reads "5.00", place 5 kg weight on the scale and wait until the display shows 4 digits.
4. When the display reads "10.00", place an additional 5 kg weight on the scale, for a total of 10 kg, and wait until the display shows 4 digits.
5. The scale will power off automatically and calibration is complete.

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Professional

www.homscales.com

For more information, visit www.homscales.com

ENGLISH

Important information

Thank you for choosing this Health o meter® Professional scale. It has all the features you'd expect from the latest technology and is designed for ease of operation. If you have any questions or if you experience problems with your Portable Pediatric Scale that are not dealt with in the operating instructions, please call a Health o meter® Professional customer service representative or visit our website, www.homscases.com.

Purpose

This Health o meter® Professional Pediatric Scale is designed for weighing infants. It is portable and intended to be used without requiring approval.

Safety instructions

This scale is designed to be used indoors. Please read the information contained in the operating instructions before you switch this device on. They provide important advice on the correct usage and maintenance of this portable scale. The manufacturer accepts no liability if the following instructions are not followed. This scale meets the requirements for electromagnetic compatibility. Avoid exceeding the maximum values specified in the standards. If problems arise, please contact your Health o meter® Professional customer service representative.

Cleaning

A damp cloth and a commercially available cleaning agent are all that are required to clean your scale regularly. Do not use abrasive cleaners. High levels of humidity, vapor and aggressive liquids should be avoided.

Maintenance

Calibration for measuring accuracy was completed at the factory. This scale does not require routine maintenance. However, it is advisable to check its accuracy periodically. The frequency of these checks depends on your frequency of use and the state of the scale. If you find any discrepancies (at 15 kg \geq 50 g) you should contact a Health o meter® Professional representative. Repairs or warranty replacements must be authorized by a Health o meter® Professional customer service representative in advance.

Note

This device is a measuring instrument. Vibrations, drafts or rapid changes in temperature and direct sunlight can affect the weighing results. This device is interference-free. However, under extreme electrostatic or electromagnetic interference (e.g. when operating a radio or mobile phone in the immediate vicinity of the device), the displayed value can be falsified. After the interference has ended, the product can be used again as intended, though it may be necessary to turn it off and switch it on again. The scale complies with protection class IP 54. High levels of humidity, vapor, aggressive liquids and heavy dirt should be avoided.

Power Requirements: 2 AAA batteries included
Storage temperatures: 32°F to 122°F (0°C to 50°C)

Warranty/liability

The warranty period is 2 years and starts on the day of purchase. Please keep your invoice as proof of purchase. All returns must be in original packaging or equivalent and must be returned to Pelstar, LLC FREIGHT PREPAID. Pelstar, LLC shall have the sole right to determine if merchandise is defective and whether credit or replacement will be issued. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts, including repaired and replaced parts, are covered only for the original warranty period. No returns will be accepted without prior written authorization. Call the Returned Goods Department at 800-815-6615 for your Return Authorization (RA).

Note: the Return Authorization (RA) Number must appear near the shipping label, on the outside of the shipping box.

If there is a defect in the materials or workmanship or the product fails to function properly, return the product, FREIGHT PREPAID, to Pelstar, LLC. If the company determines that a defect in material or workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Dealers that sell Pelstar products do not have the right to alter, modify or in any way change the terms and conditions of this product warranty.

Neither credit nor replacement will be issued for this product beyond the warranty period. If a scale is no longer covered by this warranty, or has been damaged in use, an estimate of repair or replacement costs will be provided to you prior to servicing or replacement. No warranty is offered on damage arising from unsuitable or incorrect storage, improper use by the client or by a third party, natural wear, modifications or interventions, incorrect or negligent handling, particularly excessive stress or weight in excess of the maximum the scale is designed for, electro-chemical, chemical or electrical influences or high humidity insofar as these cannot be traced back to fault on the manufacturer. If operational, climatic or other user influences lead to product damage, the warranty for the correct overall function of the device shall no longer apply.

Disposing of the scale

According to our current level of knowledge, this Health o meter® Professional Portable Pediatric Scale does not contain any special materials that are hazardous to the environment. This product should not be treated as normal refuse, but should be sent to a collection point for recycling electrical and electronic devices. For further information, contact your local authority, a local waste disposal business or the dealer from whom you purchased the product.

Disposing of batteries: Batteries do not belong with regular waste. Used batteries can be sent to public collection points in your locality.

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PELSTAR, LLC