# Wireless Communication Set Up Instructions for “BT” Model Scales

PELSTAR®, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

[www.homscales.com](http://www.homscales.com)

weigh**easier**®

© Pelstar®, LLC 2018

# Health o meter®

P r o f e s s i o n a l

---

Thank you for purchasing this Health o meter® Professional product. Please read this manual carefully and keep it for easy reference or training.

## TABLE OF CONTENTS

PRODUCT OVERVIEW .....	3
HARDWARE SET UP FOR WELCH ALLYN® CONNEX DEVICE .....	5
HARDWARE/DRIVER SET UP FOR WINDOWS® PC.....	6
TROUBLESHOOTING .....	10
REGULATORY INFORMATION .....	11
WARRANTY.....	12

# PRODUCT OVERVIEW

## **Product Overview**

Your “BT” version scale includes a built-in wireless module. Using the included hardware, the scale can transmit data to a Windows® PC or a Welch Allyn® device. See below for set up instructions.

## **Transmitting Data to a Windows® PC**

To allow the scale to wirelessly transmit data to a Windows® PC, drivers for the included USB wireless dongle must first be installed on the user’s PC. See page 6 for more information about setting up the drivers on a PC. After installation, the USB wireless dongle must be inserted into a USB port on the PC. Transmitting data into an EMR system requires that the user’s Windows® PC has one of the following systems installed: Allscripts TouchWorks® or Professional™, Midmark® IQmanager® or ChARM Health EHR.

- **Allscripts Interface:** To complete the connection to an Allscripts systems, users must download the Allscripts app available at [www.homscales.com/innovations/connectivity-solutions](http://www.homscales.com/innovations/connectivity-solutions). Follow the installation instructions included with the download. To complete installation, Allscripts must activate the app within the user’s account to allow for interface between the scale and the Allscripts system on the user’s PC.
- **Midmark Interface:** Connection between the scale and Midmark IQ Manager requires that the user’s PC have the IQ Manager software installed. For further information users must contact Midmark Technical Support.
- **ChARM Health EHR Interface:** ChARM Health users must contact their account manager to set up the service.

## **Transmitting Data to a Welch Allyn Connex Device**

To allow the scale to wirelessly transmit data to a Welch Allyn® Connex® monitor, the included hardware must be attached to the Welch Allyn® device. See page 5 for details about connecting the scale with WelchAllyn® monitors. From the Welch Allyn® device, data can be transferred into an EMR. For data to transfer to the EMR, the EMR installed on the user’s PC must be included on the list of Welch Allyn EMR partners. Visit [www.welchallyn.com](http://www.welchallyn.com) to view a complete list of EMR partners. “BT” scales are preconfigured to interface with the following Welch Allyn® Connex® devices: Connex® Spot Monitor, Connex® Vital Signs Monitors, Connex® Integrated Wall Systems.

Health o meter® Professional supports customers who wish to develop interfaces to their EMR and other computer systems. Developers can obtain the communication protocols needed for the scale models used in their particular environment at [www.homscales.com/innovations/connectivity-solutions](http://www.homscales.com/innovations/connectivity-solutions).

## **Windows® PC Requirements**

- This installation is exclusively compatible with Windows® XP/Vista/7
- CPU of 1.7GHz and above
- Minimum 512MB RAM
- Available USB 2.0 Port

## **Cautions and Warnings**

- To prevent installation and performance issues to your PC, please follow these instructions carefully.
- For accurate data collection, confirm and upload the data according to the procedure described in this manual.
- Data validation must be confirmed by the user from the scale to the receiving device to ensure accurate data collection.

# PRODUCT OVERVIEW (CONT.)

## Hardware Included

**HOMWA (black box) with USB Wireless Dongle**


If connecting to a Windows® PC, disconnect the USB dongle from the black box and connect to any available USB 2.0 port on the PC. The unused black box may be returned to Health o meter® Professional Scales. In return for unused black boxes, you will receive a free ScaleSurance® extended warranty for your scale. Call 1-800-815-6615 for more information.

 <p><b>USB Cable 6 ft.</b></p>	 <p><b>Welch Allyn® Connex Vital Signs Monitor (CVSM) Quick Start Guide</b></p>	 <p><b>USB Cable 1 ft.</b></p>
---	--	---

**Note:** The serial number on the USB wireless dongle must match the BT device # on the label located on the back of the scale display head.

**SERIAL NUMBER AND BT DEVICE NUMBER MUST MATCH TO ENABLE WIRELESS COMMUNICATION.**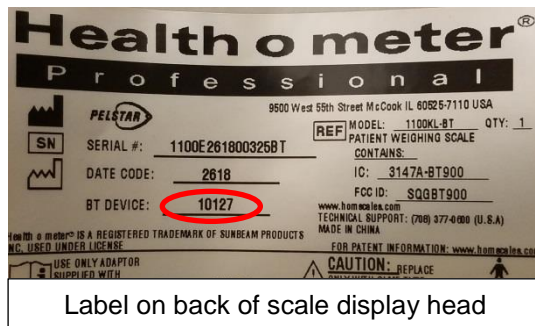


## HARDWARE SET UP FOR WELCH ALLYN CONNEX DEVICE

The following instructions illustrate the hardware set up for connecting to a Welch Allyn<sup>®</sup> Connex<sup>®</sup> Vital Signs Monitor (CVSM). USB ports on the Connex Spot and Integrated Wall Systems are located on the underside of the monitor.

1. Obtain the HOMWA (black box) with USB Wireless Dongle from the carton.

2. Connect the HOMWA (black box) with dongle to the Welch Allyn<sup>®</sup> CVSM as shown below.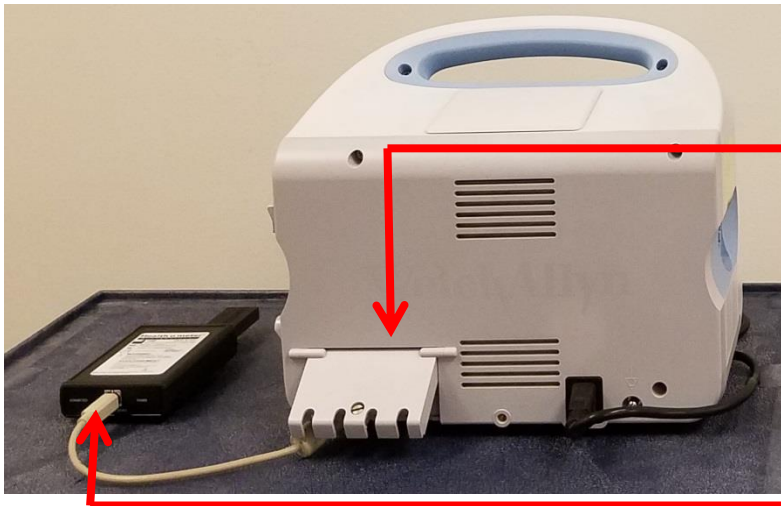

### HOMWA Unit Connection to CVSM

- 1) Connect the 1 ft. USB cable to any available USB port on the Welch Allyn<sup>®</sup> CVSM
- 2) Connect the other end of the 1 ft. USB cable to the “**TO CONNEX DEVICE**” USB port on the HOMWA (black box)

3. Power on the Welch Allyn<sup>®</sup> CVSM unit and power on the scale to initialize wireless communication. The connection is now established.
4. To enable weight scale communication on the Welch Allyn<sup>®</sup> CVSM, follow these steps.
  - a) Connect the CVSM to a PC to access the Welch Allyn<sup>®</sup> Service Tool. This service tool comes with the Welch Allyn<sup>®</sup> device or can be downloaded at [www.welchallyn.com/en/service-support/service-center/service-tool.html](http://www.welchallyn.com/en/service-support/service-center/service-tool.html)
  - b) Follow the prompts in the Service Tool to activate the weight scale license.
  - c) Activate the license by entering the authorization code A66FF29A3B2F85E1


**\*Note:** Weight scale communication is already enabled on Welch Allyn<sup>®</sup> Connex<sup>®</sup> Spot monitors.

# HARDWARE/DRIVER SET UP FOR WINDOWS® PC

1. Download the CP210x USB to UART Bridge VCP Driver using one of the following methods:
  - a. Go to <http://www.silabs.com/products/mcu/pages/usbtouartbridgevcpdrivers.aspx>
  - b. Go to <http://www.silabs.com/> and search for “CP2102 Driver Download”.
2. Select the download for Windows® XP/Server 2003/Vista/7/8/8.1/10 (6.7) – Version 6.7 or the latest version available.
3. Unzip the downloaded file.
  - a) The following example shows installation for a Windows 64 bit OS.


Name ^	Type	Size
x64	File folder	
x86	File folder	
CP210xVCPInstaller_x64	Application	1,026 KB
CP210xVCPInstaller_x86	Application	901 KB
dpinst	XML Document	12 KB
ReleaseNotes	Text Document	11 KB
SLAB_License_Agreement_VCP_Windows	Text Document	9 KB
slabvcp	Security Catalog	12 KB
slabvcp	Setup Information	5 KB

- b) Follow the installation wizard steps. Click the “**Next**” button.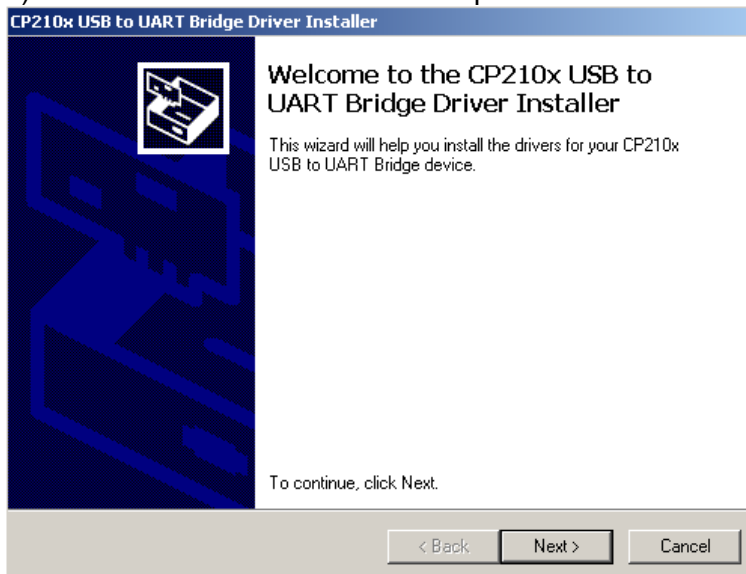

# HARDWARE/DRIVER SET UP FOR WINDOWS® PC (CONT.)

c) Accept the license agreement and click “Next”

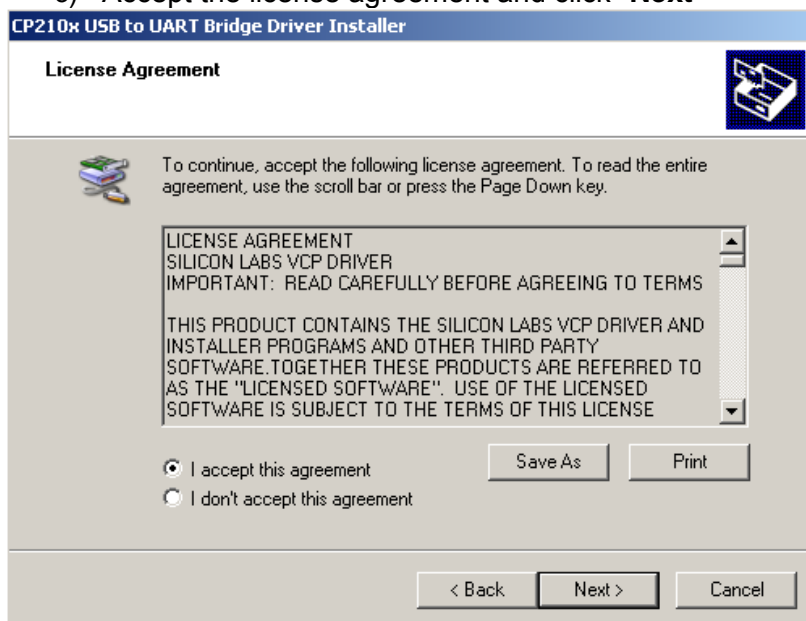
d) Wait while the driver installs.

## HARDWARE/DRIVER SET UP FOR WINDOWS® PC (CONT.)

- e) When driver installation is complete, click “**Finish**” to close the installation Wizard.

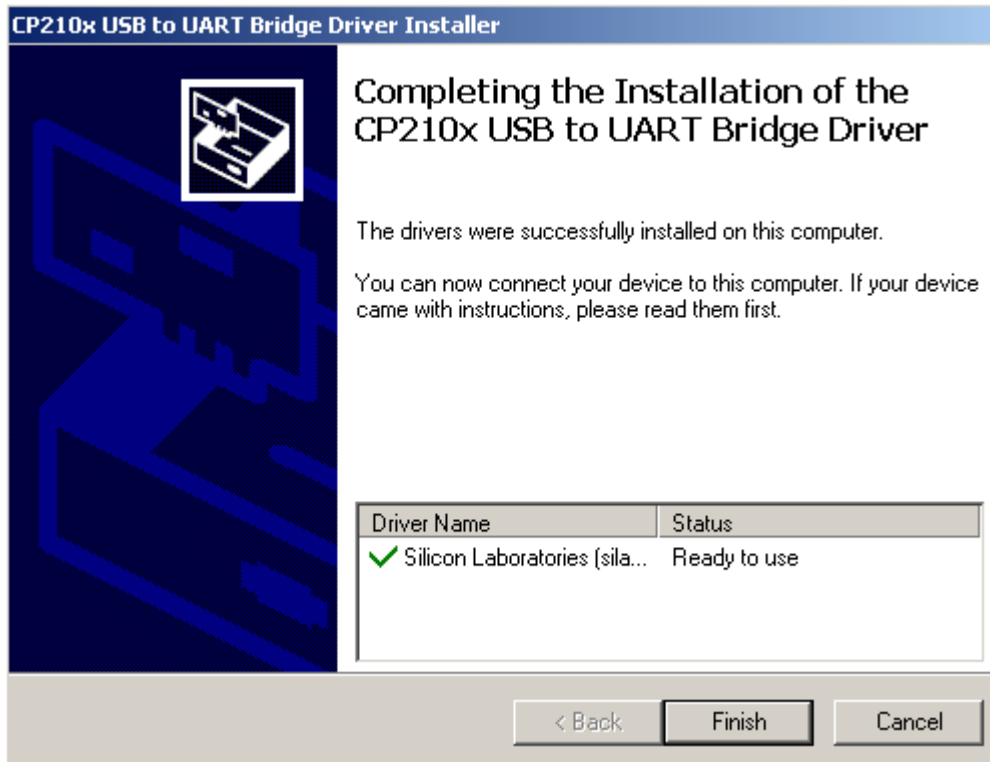4. Remove the USB wireless dongle from the black box included with this kit.\* Connect the USB wireless dongle to any available USB 2.0 port on the Windows® PC.


\*The unused black box may be returned to Health o meter® Professional Scales. In return for unused black boxes, you will receive a free ScaleSurance extended warranty for your scale. Call 1-800-815-6615 for more information.

## HARDWARE/DRIVER SET UP FOR WINDOWS® PC (CONT.)

5. Open Windows® “**Device Manager**” and check that the USB wireless dongle was recognized by the Windows® OS.

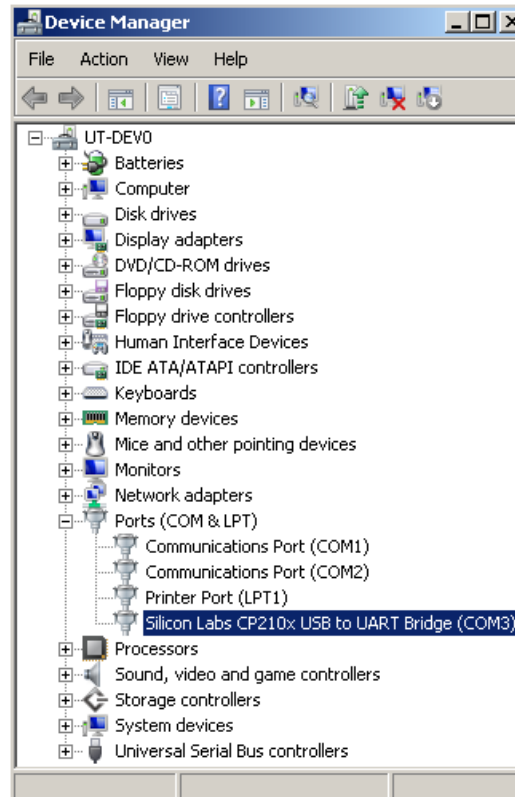6. Once the driver installation is complete, the USB wireless dongle will automatically connect to the paired BT scale and will be ready to receive data.
7. The driver installation is now ready to communicate with applications on the PC. Power on the scale to initialize wireless communication.

**Note:** Transmitting data into an EMR system on the PC requires that the user’s Windows® PC has one of the following systems installed: Allscripts TouchWorks, Allscripts Professional, Midmark IQ Manager Software, or ChARM Health EHR. See the Product Overview section on page 3 for more information.

# TROUBLESHOOTING

## USB Wireless Dongle Fault Symptoms

Problem	Possible Cause	Suggested Action
No Power/USB	Power failure or USB error failure	Re-cycle power to the HOMWA (black box) by disconnecting the USB cable connected to <b>"TO CONNEX DEVICE"</b> and re-connect
No Communication	USB wireless dongle out of communication range	Check that the distance between the scale and Connex <sup>®</sup> device or PC is less than ~328ft (100m)
	Serial number on wireless dongle does not match the BT Device Number on the scale.	Check that the BT Device Number on back of the scale's display head matches the Serial Number on the USB wireless dongle. If the number does not match contact Customer Service at 1-800-815-6615.
	Wireless network interference	Move scale or Connex <sup>®</sup> device away from nearby wireless devices
	HOMWA to Connex <sup>®</sup> Device	Re-cycle power to the HOMWA (black box) by disconnecting the USB cable connected to <b>"TO CONNEX DEVICE"</b> and re-connect
	USB wireless dongle to Windows <sup>®</sup> PC	Check that the <b>Virtual Comport CP210x Driver Installation</b> is installed properly via Windows <sup>®</sup> Device Manager. Re-install driver if necessary.

## Welch Allyn Connex Fault Symptoms

Problem	Possible Cause	Suggested Action
<u>No</u> Weight, Height, Body Mass Index (BMI) Displayed on Connex <sup>®</sup> device	Weight scale communication license <u>NOT</u> activated on the Connex <sup>®</sup> device	<p>*Weight scale communication is already enabled on Welch Allyn<sup>®</sup> Connex<sup>®</sup> Spot monitors.</p> <p>To enable weight scale communication on the Welch Allyn<sup>®</sup> CVSM, follow these steps.*</p> <p>a) Connect the CVSM to a PC to access the Welch Allyn<sup>®</sup> Service Tool. This service tool comes with the Welch Allyn<sup>®</sup> device or can be downloaded at <a href="http://www.welchallyn.com/en/service-support/service-center/service-tool.html">www.welchallyn.com/en/service-support/service-center/service-tool.html</a></p> <p>b) Follow the prompts in the Service Tool to activate the weight scale license.</p> <p>c) Activate the license by entering the authorization code A66FF29A3B2F85E1</p> <p>For assistance with the Welch Allyn<sup>®</sup> Service Tool, please contact a Welch Allyn<sup>®</sup> representative or visit <a href="http://www.welchallyn.com/">www.welchallyn.com/</a></p>

## REGULATORY INFORMATION

### **FEDERAL COMMUNICATION COMMISSION (FCC) STATEMENT – USB WIRELESS DONGLE**

The KC-1114 (BT module used in KC-4134 USB wireless dongle) has a granted modular approval by the Federal Communications Commission. The module meets the conducted and radiated emission requirements of the FCC "Code of Federal Regulations" Title 47, Part 15, Subpart C, Section 15.247 for wireless spread spectrum transmitters.

Modular Approval, FCC and IC

FCC ID: S2242 IC: 8193A-BTMODULECL2

Qualified Profiles: BB, DUN, GAP, HCI, L2CAP, LM, RFCOMM, SDP, SPP

IC Warning Statement: The device's user manual does not contain the following or equivalent statement as per RSS-GEN section 7.1.5: Operation of this device is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

### **FEDERAL COMMUNICATION COMMISSION (FCC) STATEMENT – BT900 MODULE**

This EUT is in compliance with SAR for general population/uncontrolled exposure limits in ANSI/IEEE C95.1-1999 and had been tested in accordance with the measurement methods and procedures specified in OET Bulletin 65 Supplement C.

The BT900 is fully approved for mobile and portable applications.

Modular Approval, FCC and IC

FCC ID: SQGBT900, IC: 3147A-BT900

FCC Warning: This device complies with part 15 of the FCC rules and industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

### **CE REGULATORY – USB WIRELESS DONGLE**

The KC-1114 (BT module used in KC-4134 USB wireless dongle) complies with the following EMC Directives:

ETSI EN 300.328.V1.6.1 ; ETSI EN 300.328 V1.8.1 [Approval Pending]

### **CE REGULATORY – BT900 Module**

The BT900-SA has been tested for compliance with relevant standards for the EU market. See table below.

EU Directives: 2014/53/EU – Radio Equipment Directive (RED)

Article Number	Requirement	Reference Standard(s)
3.1a	Low voltage equipment safety RF Exposure	EN 60950-1:2006+A11:2009+A1 :2010+A12 :2011+A2 :2013 EN 62311 :2008
3.1b	Protection requirements with respect to electromagnetic compatibility	EN 301 489-1 v2.2.0 (2017-03) EN 301 489-17 v3.2.0 (2017-03)
3.2	Means of the efficient use of the radio frequency spectrum (ERM)	EN 300 328 v2.1.1 (2016-11)

### **SAR COMPLIANCE**

The USB Wireless Dongle and BT900 Module are SAR compliant.

# WARRANTY

## **Limited Warranty**

### **What does the Warranty Cover?**

This Health o meter<sup>®</sup> Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar<sup>®</sup>, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the product at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

### **Who is Covered?**

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar<sup>®</sup> dealers or retail stores selling Pelstar<sup>®</sup> products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

### **What is Excluded?**

**Your warranty does not cover normal wear of parts or damage resulting from any of the following:** negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

**To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase.** Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# \_\_\_\_\_

Return Department

9500 West 55<sup>th</sup> Street

McCook, IL 60525

**PELSTAR<sup>®</sup>, LLC**

**9500 West 55<sup>th</sup> Street • McCook, IL 60525 • USA**

**1-800-638-3722 or 1-708-377-0600**

PLEASE REGISTER YOUR PRODUCT FOR WARRANTY COVERAGE AT:

[www.homscales.com](http://www.homscales.com)

Health o meter<sup>®</sup> is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter<sup>®</sup> Professional products are manufactured, designed, and owned by Pelstar<sup>®</sup>, LLC.

We reserve the right to improve, enhance, or modify Health o meter<sup>®</sup> Professional product features or specifications without notice.

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Health o meter® es una marca comercial registrada de Sunbeam Products, Inc. usada bajo licencia.
Los productos Health o meter® Professional son fabricados, diseñados y propiedad de Pelstar®, LLC.