Model 2595KL / 2595KG Digital Chair Scale

Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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<u>Date Codes</u>	<u>Link/Page #</u>
all date codes ending in 25	Click here for skip to page 2
For all other date codes not shown above, click here or skip to page 34	

Date code can be found on the product label on the back of the display head.

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Model 2595KL/2595KG

CLICK THE BOX BELOW OR
SKIP TO THE PAGE NOTED

**Cautions and
Warnings
p. 3**

**Specifications /
Definition of
Symbols
p. 4**

**Certifications /
Connectivity /
Disposal
p.5**

**Assembly
Instructions
p.6**

**Powering the
Scale / Battery
Replacement
p.8**

**Set Up
Instructions
p.9**

**Everlock®
Permanent Unit
Lock Instructions
p.11**

**Operating
Instructions
p.12**

**Cleaning /
Replacement
Parts
p.14**

**Troubleshooting
p.15**

**Calibration
p.16**

**Warranty
p.22**

**Español
p.23**

**Francais
p.34**

CAUTION AND WARNING

Intended Use

This Health o meter® Professional scale is intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh patients that are sitting on the chair. Do not modify the product or use it for anything other than its intended purpose.

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Assemble and operate the scale per the enclosed user instructions.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not transport the scale with a patient or object on the chair.
- Do not exceed the weight capacity specified for the scale.
- Do not use in the presence of flammable or explosive materials.
- Ensure that the scale or AC adapter do not come in contact with liquids, excessive temperature, or excessive humidity.
- If the scale becomes damaged, it should not be operated until properly serviced.

Patient/caregiver safety

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer. Pelstar, LLC assumes no responsibility for patient injury or scale damage should this caution be ignored by the scale operator.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- Ensure to lock or block casters while weighing a patient. Wheels must not roll freely when positioning or seating a patient.

When operating the scale with batteries:

- If the display indicates that the batteries are low, replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

When operating the scale with the (optional) AC adapter:

- Operate this scale exclusively with the AC adapter that is specified (ADPT30) (not included). Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

This equipment has been tested and found to comply with the EMC limits for the Medical Device Standards EN 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates, uses and may radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. There is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to eliminate the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.
- Consult Customer Service or a field service technician for help.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS FOR THIS SCALE

GENERAL

The Health o meter® Professional Digital Chair Scale uses highly sophisticated microprocessor technology. This precision instrument is designed to provide accurate, reliable and repeatable weight measurements by providing features that make the weighing process simple, fast and convenient. Weight is displayed in kilograms or pounds. The scale includes a seat with arm supports, and a footstool that can be folded up for easy entry/exit. Also included in the scale are two large casters, caster brakes, and a handrail for easy mobility. The unit can be operated using its AC adapter or with 6 C-cell batteries.

SCALE SPECIFICATIONS

Capacity and Resolution	600 lb x 0.2 lb (270 kg x 0.1 kg)	
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE160714HKKK1-P or Adapter Model No. UES06WNCP-090050SPA, Part No. UE190716HKSH2RM Health o meter® Professional Model (ADPT30) (included) Input: 100-240V AC, 50/60Hz, 500mA Output: 9.0V 0.5A or 6 C batteries (not included)	
Environmental	Operating temperatures: 10°C to 35°C (50°F to 95°F) Storage temperatures: 0°C to 50°C (32°F to 122°) Atmospheric Pressure: 50hPa-1060hPa Humidity: 10%-85% RH	
Physical Dimensions	Seat Depth: 15-3/4" (400mm) Width: 18-1/4" (464mm) Arms Span: 19-3/4" (502mm)	Entire Scale Length: 36-1/2 (927mm) Width: 23-1/2" (597mm) Height: 40" (1016mm) Weight: 80lb, 4oz (36.5kg)

DEFINITION OF SYMBOLS

Attention, Consult Accompanying Documents

Type BF Applied Parts

AC to DC power supply

CERTIFICATIONS / CONNECTIVITY / DISPOSAL

Certification Descriptions

 	Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
	This product is tested to and meets the requirements of: 1. UL60601-1 First Edition: Safety for Medical Electrical Equipment 2. CSA C22.2 No. 601-1 M90: Medical Electrical Equipment

Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or batteries. Do not dispose of this device in the domestic waste stream.

Scale Connectivity Information

For reliable transmission of weight data, this scale is designed to connect to a computer, monitor, or other electronic data device via various Health o meter® Professional Connectivity Solutions. This connectivity helps minimize transcription errors when recording patient data. For further information regarding how to connect this scale to other electronic devices, please contact Health o meter® Professional Scales Technical Support at 800-638-3722 or visit www.homscales.com/innovations/connectivity.

ASSEMBLY INSTRUCTIONS

To avoid damaging the 2595KL when unpacking it, **do not use a box cutter, knife, scissors or any sharp object when opening the protective inner packaging.** If damage is found, contact your shipper or a Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the carton as you unpack the parts for assembly. Set the carton aside for storage.

The 2595KL Digital Chair Scale is shipped assembled in one carton, except for final installation of scale casters and brakes. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

For further information or telephone support, please contact Customer Service at:


Telephone: 1 (800) 815-6615 Email: HomProCS@homscales.com

Parts List:

Carton

- (1) Chair Scale Assembly
- (2) Casters
- (2) Caster Breaks
- (2) Leveling Feet
- (2) Bolts
- (1) AC Adapter
- (1) 3/16" or 5mm Allen Wrench

Note: This scale has been factory calibrated, and does not require calibration prior to use.

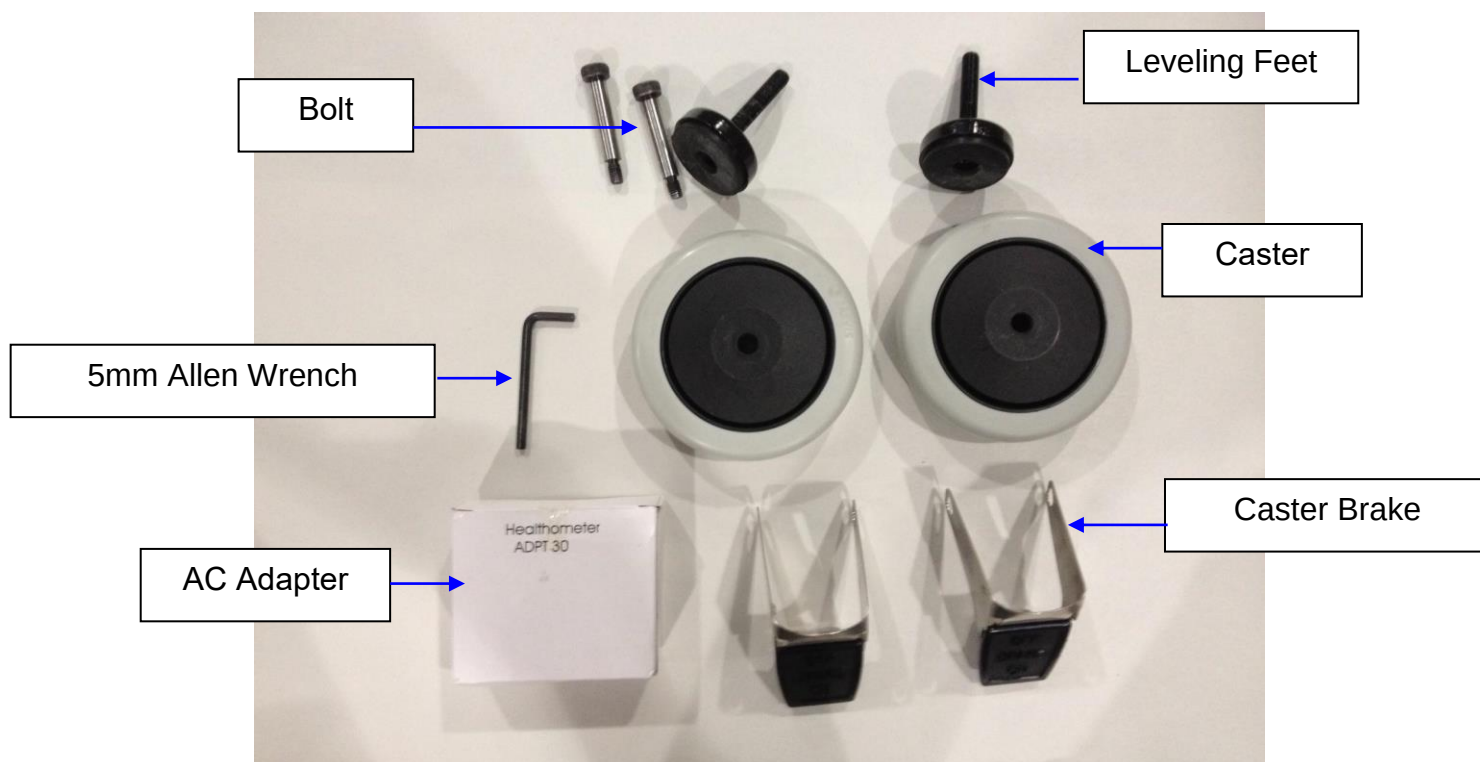ASSEMBLY INSTRUCTIONS

Note: Do not attach the AC adapter until scale is fully assembled.

Caution: It is recommended that the assembly be performed by two persons.

1. Remove the scale from the carton and place on a flat, level, and dry surface.

Pull forward on the handrail to tip the scale over, and gently place it face down on the floor.

2. Thread the (2) leveling feet into the underside of the scale base. Each foot should be completely hand tightened.

3. Obtain the casters and caster brakes from the carton. Position the caster and caster brake into the scale base aligning the holes with the hole in the side of the base.

Note: Ensure caster brake is positioned with one tab on the exterior of the caster and top of caster brake is aligned with top of caster slot.

Insert the bolt through the caster brake, scale base, and caster. Tighten using the 3/16" (5mm) Allen wrench.

Carefully tilt the scale back to an upright position on the floor.

Warning: Do not drop scale, as this can permanently damage the load cells.

The scale is now assembled. See Powering the Scale on the next page for battery/AC adapter instructions.

POWERING THE SCALE

The scale may be operated by an AC adapter (included), or by 6 C-cell batteries (not included). Plug the AC adapter into the power adapter jack (the jack is located on the rear of the display head), then plug the scale's AC adapter into the power source. Note: Do not plug the scale into a power source until the scale is fully assembled.

Warning: To avoid a safety hazard, do not use an unspecified AC adapter. Only use the Health o meter® Professional AC adapter model ADPT30 designed for this scale.

For battery operation, see Battery Installation below.

1. Loosen the two plastic knobs found underneath the display assembly until the battery holder is released from the scale and rests on the plastic knobs.	
2. Raise the battery holder and simultaneously rotate it counterclockwise 1/4 turn. Carefully support the holder and install/replace 6 new C-cell batteries.	
3. Replace the battery holder onto the plastic knobs by rotating it 1/4 turn clockwise. Hold the battery holder close to the scale and tighten the two plastic knobs to secure in place. Be careful not to damage the cable.	

SET UP INSTRUCTIONS

To enter the Options mode, push and hold the **UNIT** button while pressing and releasing the  button.

Auto Off Function

When powering the scale via the AC adapter, the display can be set to power off after a period of inactivity. The Auto Off default setting is “disabled”. Follow this procedure to enable or disable the Auto Off function. Note: When powering the scale via batteries, the Auto Off function is automatically enabled.

Enable / Disable Auto Off

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Press **UNIT** again to change the setting.
AOF=d Disables the Auto Off function
AOF=E Enables the Auto Off function
2. Press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Setting Auto Off Time

When powering the scale via batteries the period of inactivity before the scale powers off can be set to a specific amount of time. The default setting for Auto Off time is 2 minutes. Follow this procedure to set the number of minutes of inactivity before power off.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use  **ZERO/TARE** to advance or the  button to scroll back the display screen until “AFt” appears. Press **UNIT** to change the Auto Off time.
AFt = 1 Scale will power off after 1 minute of inactivity
AFt = 2 Scale will power off after 2 minutes of inactivity
AFt = 3 Scale will power off after 3 minutes of inactivity
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Auto Hold

The Auto Hold function allows the scale to continue displaying the weight measurement after the patient is removed from the weighing tray/seat. The Auto Hold setting can be adjusted to display the weight for 1, 30, or 60 seconds. The default setting is 30 seconds. Follow this procedure to adjust the Auto Hold setting.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=F” appears on the display, then release. Use  **ZERO/TARE** to advance or the  button to scroll back the display screen until “AHd” appears. Press **UNIT** to change the setting.
AHd = 0 Weight measurement will hold on the display for 1 second
AHd = 1 Weight measurement will hold on the display for 30 seconds
AHd = 2 Weight measurement will hold on the display for 60 seconds
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS

Setting Sound Options

The display can be set to stay silent or beep while operating the scale. The default setting for Sound is “enabled”. Follow this procedure to enable or disable sound.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “Snd” appears. Press **UNIT** to change the setting.
Snd = E Enables sound so the scale beeps when keys are pressed.
Snd = d Disables sound so the scale stays silent when keys are pressed.
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Locking Unit of Measure (LB or KG) (KL versions only)

The weight measuring unit (pounds / LB or kilograms / KG) can be locked to display weight only in the selected unit of measure. The unit of measure lock default setting is “unlocked”. Follow this procedure to lock the **UNIT** button. **Note:** If Everlock® is engaged, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “Unt” appears. Press **UNIT** button to change the setting.
Unt = U Allows the user to toggle weight measurement between LB and KG.
Unt = L Locks the unit button so weight is only displayed in pounds (LB)
Unt = 6 Locks the unit button so weight is only displayed in kilograms (KG)
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Enabling Unit View with Unit of Measure Locked

If the scale is locked per the instructions above, users can still view the alternate weight measuring unit by enabling the “Unit View” option. Enabling the “Unit View” option allows the user to press the **UNIT** button to briefly display the weight in the alternate weight measuring unit. After releasing the **UNIT** button the display will return to the locked weight measuring unit. The “Unit View” default setting is “Disable”. Follow this procedure to enable “Unit View” when the weight measuring unit is locked. **Note:** If Everlock® is enabled, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “UdP” appears. Press **UNIT** to change the setting.
UdP = d Disables the **UNIT** button so it will not display the other weight measuring unit.
UdP = E Allows the user to view the other weight measuring unit by holding down the **UNIT** button even if the weight measuring unit is locked. Enabling this function will only allow the user to view the other unit while pressing the **UNIT** button; it will not unlock the weight measuring unit.
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS

Setting Auto Zero

This option allows the user to select whether the scale will re-set itself to ZERO each time it is powered up or whether it will use the zero reading from the last time it was zeroed. The auto zero default setting is “enabled”. Follow this procedure to enable or disable this feature.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “PU2” appears. Press **UNIT** to change the setting.
PU2 = d Disabling this feature means the scale does not attempt to find a zero when powered on, rather it will use the zero setting it had from its most recent use. Health o meter® Professional recommends that if the Disable option is selected, users should manually zero the scale before each weight measurement by pressing **ZERO / TARE**. Users will see dashes on the screen as the scale is finding zero.
PU2 = E Enabling this feature means the scale will zero itself when it is powered on. After each use, the scale will clear the weight of the patient and set itself back to the initial zero level it found on start-up. Health o meter® Professional recommends keeping this setting as Enable.
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Everlock® (KL versions only)

The Everlock® feature allows the measuring unit to be permanently locked on the scale, disabling the **UNIT** button. The Everlock® default setting is “disabled”. Follow this procedure to engage Everlock®.

Warning: This lock is permanent and cannot be reversed.


Notes:

- Press **HOLD/RELEASE** at any time to exit the Everlock® procedure.
 - Once Everlock® is engaged the Unt and UdP options will no longer be displayed when scrolling through the Options menu.
 - There is no method to undo Everlock® without sending the scale back to the manufacturer.
1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “ELC=d” appears. This is the default setting with Everlock® disabled. Press **UNIT** to start the Everlock procedure. .
 2. “PASS1” will appear on the display. This prompts the user to enter a password. Press the following keys in this sequence:
, **ZERO/TARE**, **UNIT**
 3. If entered correctly, “ELC=L” (if scale was in LB mode) or “ELC=6” (if scale was in KG mode) will appear on the display.
 4. Use the **ZERO/TARE** button to toggle between (L) Pounds and (6) Kilograms. When the desired unit is displayed, press **UNIT** and “PASS2” will appear on the display.
 5. This will prompt the user to enter the second password. Press the following keys in this sequence:
ZERO/TARE, **UNIT**,  (**Warning: Entering this password will engage Everlock**)
 6. If entered correctly, the scale will display “ELC = L” or “ELC = 6,” depending on the unit of measurement that was selected. Everlock® is now engaged.
 7. Press **HOLD/RELEASE** to save your selection and exit the options mode.
 8. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Warning: Do not power off the scale until **HOLD/RELEASE** is pressed to save the Everlock® option, the scale has restarted and the display has returned to “0.0”.

OPERATING INSTRUCTIONS

Keypad

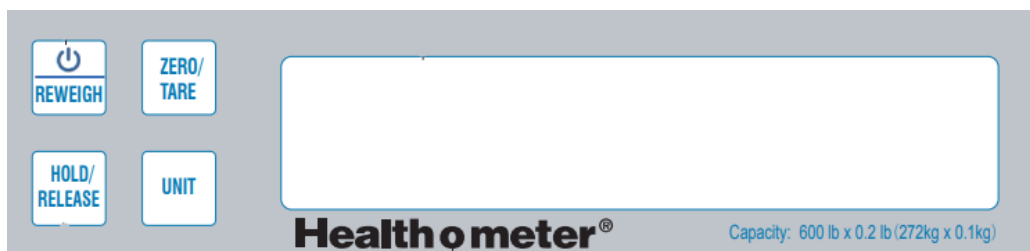DESCRIPTION	FUNCTION
 / REWEIGH	Powers scale on and off. Allows repeated weighing of the patient while remaining on the scale.
ZERO/TARE	Zeroes out or tares the scale prior to a weighing.
HOLD/RELEASE	Holds the displayed weight until the button is pressed again to release the value. Will recall the previously locked weight if pressed when nothing is on the scale.
UNIT	Selects kilograms or pounds.

Weighing a Patient

Note: When this scale is turned on, it will always default to the settings and units last used (LB or KG). The scale will not lock on a weight less than 5 lb / 2.27 kg.

1. Make sure there is nothing on the weighing seat. Press  to power on the scale. The scale is ready for use when "0.0" appears on the display.
2. Assist the patient onto the scale. "LOCK" will appear and flash on the display along with a weight measurement. When a stable weight is determined "LOCK" will stay displayed along with the patient's weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

3. Assist the patient off of the scale. The weight will be held on the display for a period of time if the Auto Hold setting is enabled. The weight can also be held by pressing **HOLD/RELEASE**. To clear a held weight, press **HOLD/RELEASE**.

After the weight measurement has been taken and is recorded, remove the patient from the scale. Or press the  / **REWEIGH** button to reweigh the patient without removing them from the scale.

4. To power off the scale, press and hold  for approximately three seconds.

Note: If the display shows "Err" you have exceeded the capacity of the scale.

Weight Recall

Note: Weight will only recall if scale has remained powered on.

1. To recall the last weight measured, press **HOLD/RELEASE**.
2. To clear the display of the recalled weight, press **HOLD/RELEASE** again.

OPERATING INSTRUCTIONS

Zero Function

Zeroes the scale prior to a weight measurement. The **ZERO/TARE** button is also used in the Tare function outlined below.

Tare Function

When using this scale, the weight of an object, such as a portable oxygen tank, can be subtracted from the total weight in order to determine the weight of the patient alone.

Weighing a Patient and Taring an Unknown Weight:

The minimum weight that can be tared is 5 lb / 2.27 kg. **Note:** The scale must lock onto a weight before it can be tared.

1. Make sure there is nothing on the scale. Press  to power on the scale. The scale is ready for use when "0.0" appears on the display.
2. Place the object to be tared on the scale. The display will show a value for the object's weight.
3. Press **ZERO/TARE**. The word "tArE" will appear on the display, and the scale will return to zero.
4. Keep the object on the platform and assist the patient onto the scale. The scale will automatically deduct the weight of the object and only display the patient's weight.
5. When you remove the patient and the item from the scale, press **ZERO/TARE** to clear the tared value.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

Weighing a Patient and Taring a Known Weight (Pre-Tare):

Pre-Tare can be performed for items between .2 lb and 200 lb (0.1 kg and 91 kg).

1. Make sure there is nothing on the scale. Press  to power on the scale. The scale is ready for use when "0.0" appears on the display.
2. Press and hold **ZERO/TARE** until the word "p-tArE" appears on the display.
3. The tare icon will flash along with "0.0" prompting you to enter the weight to be tared. Use the  and **ZERO/TARE** buttons to increase or decrease the weight. Pounds will change in 0.2 lb increments and KG will change in 0.1 kg increments.
4. Once the desired tare value is set, lock the value by pressing **HOLD/RELEASE**. The tare icon will stop flashing and remain on, and the entered amount will show as a negative.
5. Weigh the patient following the weighing procedure outlined on the previous page. The scale will automatically deduct the tare value and only display the patient's weight.
6. To clear the tare value, press the **ZERO/TARE** button.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, and troubleshooting for the 2595KL Digital Chair Scale. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the scale for any obvious damage, wear and tear.
2. Inspect the AC adapter for cord cracking or fraying, or for broken or bent prongs.

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:

- mild soap and water solution
- 70% isopropyl alcohol
- solution with 1-5% hydrogen peroxide concentration

After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure all scale parts are completely dry after cleaning.

2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

Replacement Parts

Parts may be available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

Part #	Description
ADPT30	Power Adapter
2595BATTHOLD	2595KL Battery Holder
B3817001-0	Adapter Holder
2595KLCASTER	2595 Wheel Lock
B401280	Pipe Plug Front Frame 2595KL
B401289	Pipe Plug Back Frame 2595KL
B401294	2595KL Arm Rest
B2266801-0	2595KL Wheel
B3438901-2	Plastic foot footrest 2595KL
B3810002-0	Plastic foot (frame) 2595KL
B3819701-0	2595KL Foot Mat
B401296	2595kl Seat Assembly

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	- Defective battery - Faulty electrical outlet - Defective power supply	- Replace batteries - Use a different outlet - Replace AC adapter
Questionable weight or the scale does not zero	- External object interfering with the scale - The display did not show "0.0" before weighing - Scale is not placed on a level floor - Scale is out of calibration	- Remove interfering object from the scale - Remove the patient from the scale. Zero the scale and begin weighing process again - Place the scale on a level floor and begin weighing process again - Check weight with known weight value
The display shows "-----"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its limits
The display shows "LobAt"	The battery is depleted	Replace batteries according to instructions

CALIBRATION PROCEDURE

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. 200lb or 80 kg of certified weight is required for calibration.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold **HOLD/RELEASE**, then press and release the  button to power on the scale. Remain pressing **HOLD/RELEASE** until "CAL" appears on the display. Release all buttons.
2. "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ZERO/TARE**.
3. The scale will zero and "PLACE 200 ON SCALE PrESS HOLD" in LB mode, or "PLACE 80 ON SCALE PrESS HOLD" in KG mode. Press **UNIT** to toggle between units.
4. Place 200lb or 80 kg of certified weight on the center of the seat, then press **HOLD/RELEASE**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a five digit number will be displayed for approximately 5 seconds then display the calibration weight. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard, "Out OF RANGE" will appear on the display. Calibration is not performed. Press **ZERO/TARE** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 W. 55th St.
McCook, IL 60525

Extended Warranty Available*

This scale is eligible for Health o meter® Professional ScaleSurance® Extended Warranty Program. ScaleSurance® extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit www.homscales.com/scalesurance/ or contact your medical supply distributor.

*Not available in all countries

PELSTAR, LLC

9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2020

weigh**easier**®

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Caution and Warning.....	36
Specifications for This Scale	37
Certifications / Connectivity / Disposal / Service.....	38
Assembly Instructions 2595KL Scale	39
Set Up Information / Batteries	42
Operating Instructions	44
Maintenance.....	45
Troubleshooting	46
Calibration Procedure.....	47
Warranty.....	48

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Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTION AND WARNING

Intended Use

The Health o meter® Professional 2595KL scales were intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh patients that are sitting on the chair. Do not modify the product or use it for anything other than its intended purpose.

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- Assemble and operate the scale per the enclosed user instructions.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not transport the scale with a patient or object on the tray.
- Do not exceed the weight capacity specified for the scale.
- Do not use in the presence of flammable or explosive materials.
- Ensure that the scale or AC adapter do not come in contact with liquids, excessive temperature, or excessive humidity.
- If the scale becomes damaged, it should not be operated until properly serviced.

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- If “LOBAT” appears on the display, replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

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This equipment has been tested and found to comply with the EMC limits for the Medical Device Directive 93/42/EEC (EN 55011 Class A and EN 60601-1-2). These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates, uses and may radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. There is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to eliminate the interference by one or more of the following measures:

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SPECIFICATIONS FOR THIS SCALE

GENERAL

The Health o meter® Professional 2595KL Digital Chair Scale uses highly sophisticated microprocessor technology. This precision instrument is designed to provide accurate, reliable and repeatable weight measurements by providing features that make the weighing process simple, fast and convenient. Weight is displayed in kilograms or pounds. The scale includes a seat with arm supports, and a footstool that can be folded up for easy entry/exit. Also included in the scale are two large casters, caster brakes, and a handrail for easy mobility. The unit can be operated using its AC adapter or with 6 C-cell batteries.

2595KL SCALE SPECIFICATIONS

Capacity and Resolution	600 lb x 0.2 lb (270 kg x 0.1 kg)	
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE141023HKKK1-P Health o meter® Professional Model (ADPT30) (included) Input: 100-240V – 50/60Hz, 500mA Output: 9.0V - - - 0.5A or 6 C-cell batteries (not included)	
Environmental	Operating temperatures: 10°C to 35°C (50°F to 95°F) Storage temperatures: 0°C to 50°C (32°F to 122°) Atmospheric Pressure: 50hPa-1060hPa Humidity: 10%-85% RH	
Physical Dimensions	Seat Depth: 15-3/4" (400mm) Width: 18-1/4" (464mm) Arms Span: 19-3/4" (502mm)	Entire Scale Length: 36-1/2" (927mm) Width: 23-1/2" (597mm) Height: 40" (1016mm) Weight: 80lb, 4oz (36.5kg)

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Type BF Applied Parts

AC to DC power supply

CERTIFICATIONS / CONNECTIVITY / DISPOSAL / SERVICE

Certification Descriptions

	Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
	This product is tested to and meets the requirements of: 1. UL60601-1 First Edition: Safety for Medical Electrical Equipment 2. CSA C22.2 No. 601-1 M90: Medical Electrical Equipment

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Scale Connectivity Information

For reliable transmission of weight data, this scale is designed to connect to a computer, monitor, or other electronic data device via various Health o meter® Professional Connectivity Solutions. This connectivity helps minimize transcription errors when recording patient data. For further information regarding how to connect this scale to other electronic devices, please contact Health o meter® Professional Scales Technical Support at 800-638-3722 or visit www.hom scales.com/innovations/connectivity.

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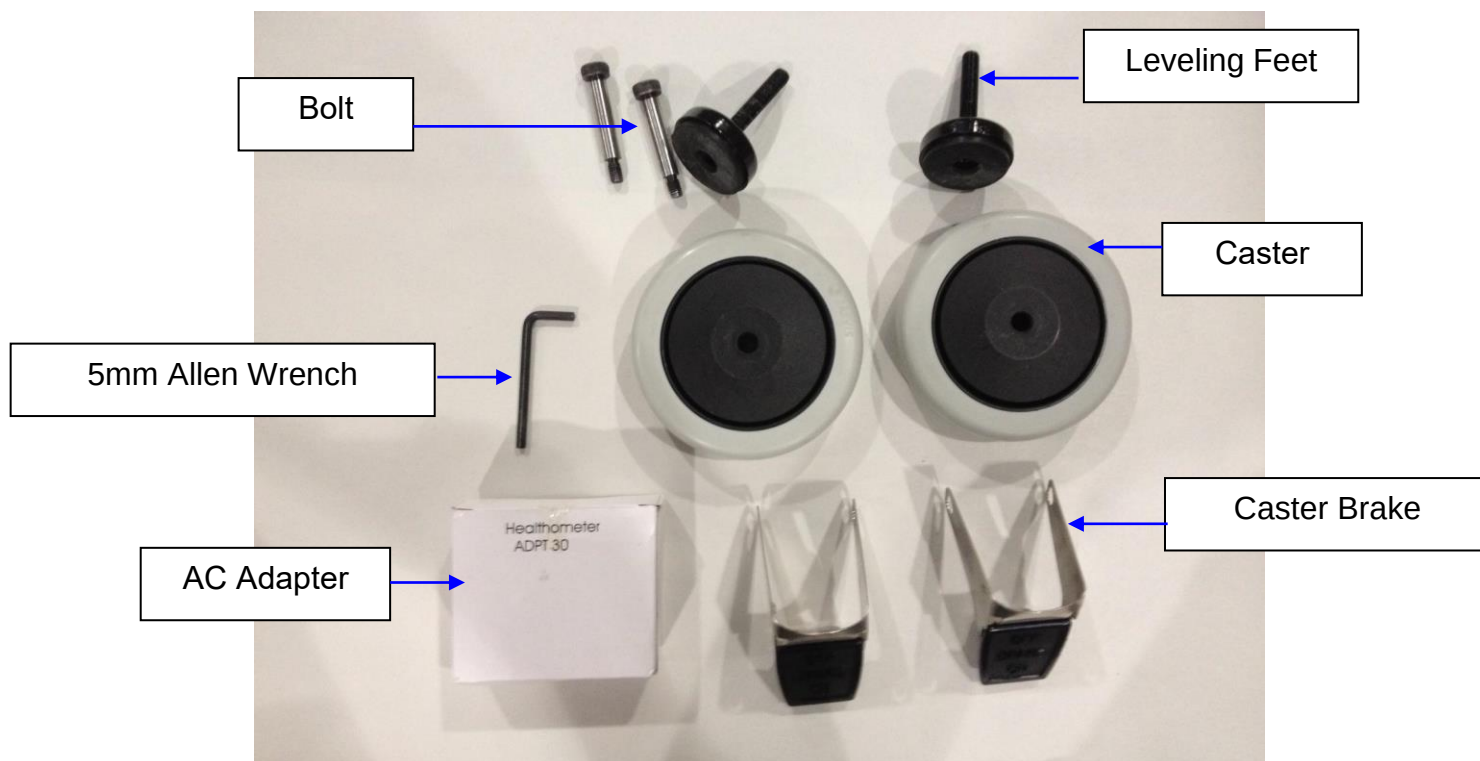
Figure 1

2595KL ASSEMBLY INSTRUCTIONS

Assembly Instructions (Refer to Figures 1- 3)

Note: Do not attach the AC adapter at this point. Wait until Step 9.

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1. Remove the chair scale assembly from the carton and place chair scale on a flat, level and dry surface.
2. Pull forward on the handrail to tip the scale over, and gently place it face down on the floor.
3. Thread the (2) leveling feet into the underside of the chair scale assembly. Each foot should be completely hand tightened.
4. Position the casters and caster brakes on both sides of the chair assembly.
5. Position the caster and caster brake with each bolt hole. Place the bolt through each bolt hole and tighten using the 3/16" (5mm) Allen wrench (see Figure 3).

Note: Ensure caster brake is positioned with one tab on the exterior of the caster.

6. Carefully tilt the scale back to an upright position on the floor.

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7. Adjust the feet to level the scale.
8. Connect the AC adapter to the power port in the display assembly. Do not plug the scale into a power source until the scale is fully assembled.

Note: The scale may be operated by an AC adapter (included), or by 6 C-cell batteries (not included). Using the battery option, unplug the AC adapter from the adapter jack and ensure that there are 6 C-cell batteries in the display assembly. The scale will automatically switch to battery power when the AC adapter is absent or not plugged in.

9. The scale is now assembled. Plug the AC adapter into a power source and / or insert batteries. (See page 9 for battery installation).

2595KL ASSEMBLY INSTRUCTIONS

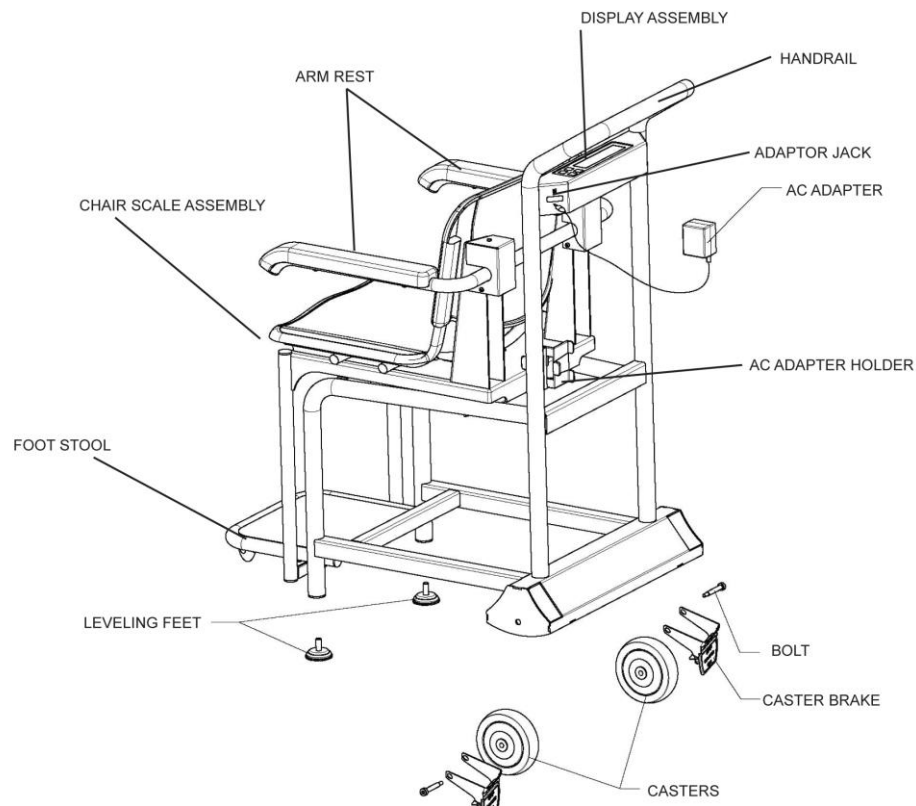
Figure 2. Assembly of the 2595KL

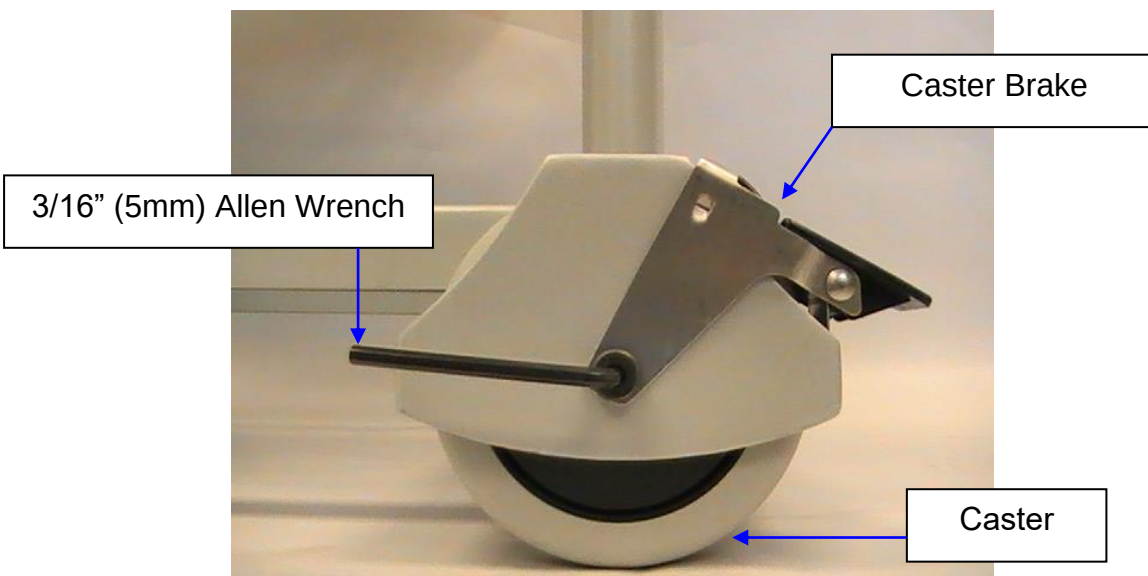
Figure 3

SET UP INFORMATION / BATTERIES

Set Up Information (Refer to Figures 4- 5)

1. Place 6 C-cell batteries in the battery holder (see “Replacing Batteries” below), or plug the AC adapter into the chair scale assembly and into the power source.
2. Press the **ON/OFF** button to turn the scale on. The display will show “START”, then “0.0”.
3. When the self-test function is complete:
 - The digital display reads “0.0”.
 - “LB” or “KG” is indicated on the right side of the display.
4. Place a weight not to exceed 600 lb (270 kg) on the scale. The digital display should blink “ ” until the weight calculation is complete.
5. Press the **LBS/KG** button to select the weight mode, either in kilograms or pounds.
6. Press the **REWEIGH** button; the digital display will read the weight on the chair scale assembly. Pressing the **REWEIGH** button will reweigh the weight that is still on the chair scale assembly.
7. Remove the weight from the chair scale assembly, the digital display will continue to show the weight of the item that was on the scale.
8. Press the **ZERO** button; the digital display will return to “0.0”.
9. Press the **ON/OFF** button to turn the chair scale assembly off. The display will show “OFF” and will then shut off.

Note: If the set up procedure failed, go to the troubleshooting instructions. If the problem is still not corrected, refer to qualified service personnel.

Installing / Replacing Batteries

1. Loosen the two plastic knobs found underneath the display assembly until the battery holder is released from the chair scale assembly and rests on the plastic knobs.
2. Carefully pull out the cable to provide access to the battery holder.
3. Raise the battery holder and simultaneously rotate it counterclockwise 1/4 turn. Carefully remove the holder from the chair scale assembly.
4. Install/replace the batteries with 6 new C-cell batteries.
5. Place the battery holder onto the plastic knobs by rotating it 1/4 turn clockwise.
6. Hold the battery holder close to the chair scale assembly and tighten the two plastic knobs to secure in place. Be careful not to damage the cable.

KG Lockout

The default weighing mode for this scale can be changed from LB/KG to KG only. To change to KG, turn the scale on and wait for the display to show “0.0”, then press and hold the **LBS/KG** button for 20 seconds until the scale beeps 6 times. The scale is now locked in the KG mode and the user cannot toggle between LB and KG. To disable the KG Lockout feature, press and hold the **LBS/KG** button for another 20 seconds until the scale beeps 6 times, indicating that the user can again toggle between pounds and kilograms.

SET UP INFORMATION / BATTERIES

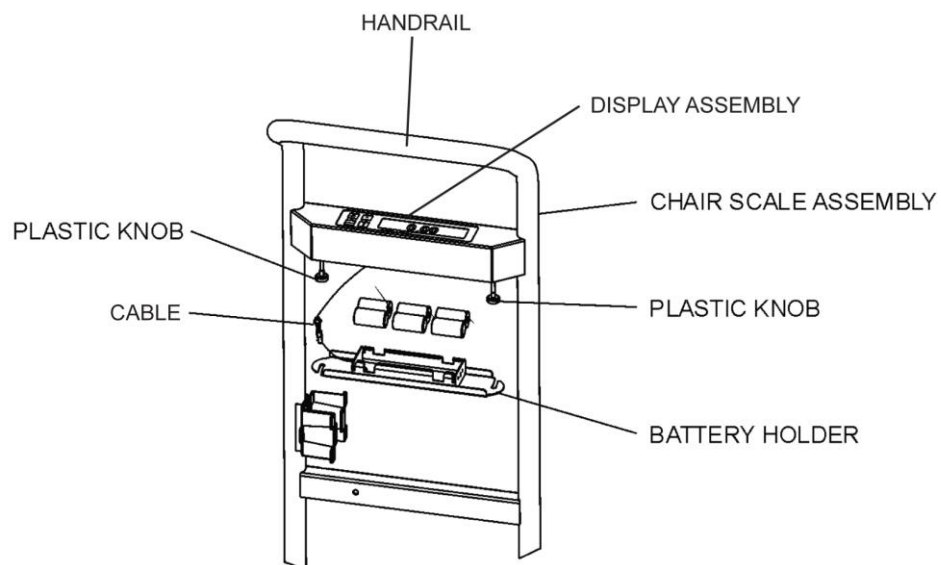
Figure 4. Replacing Batteries

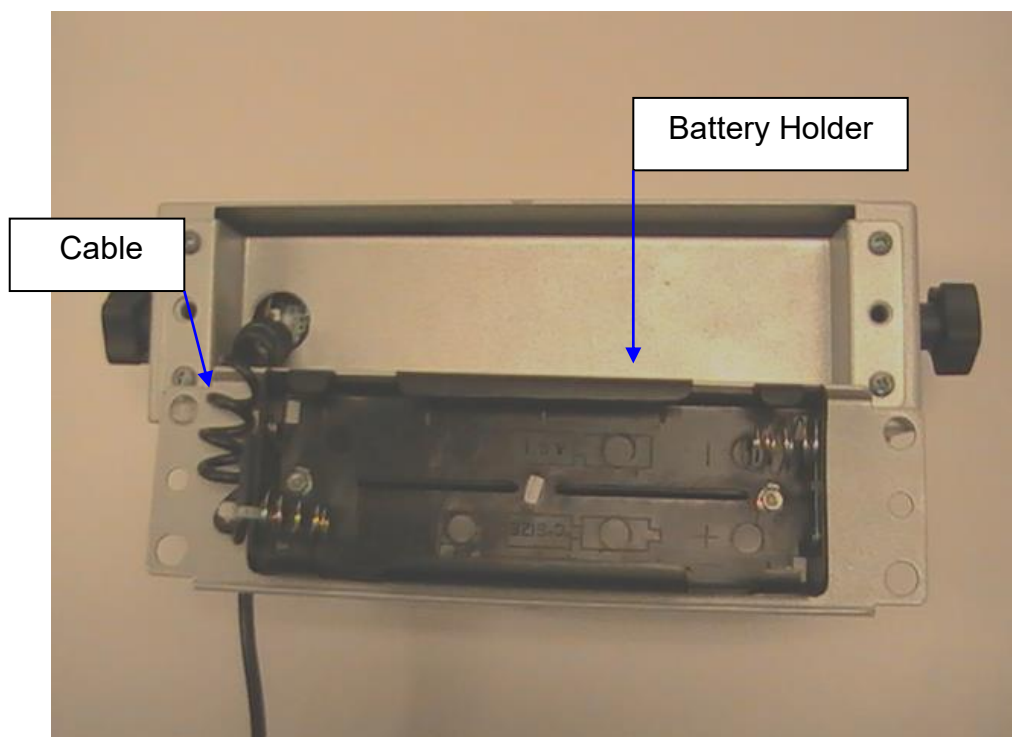
Figure 5

OPERATING INSTRUCTIONS

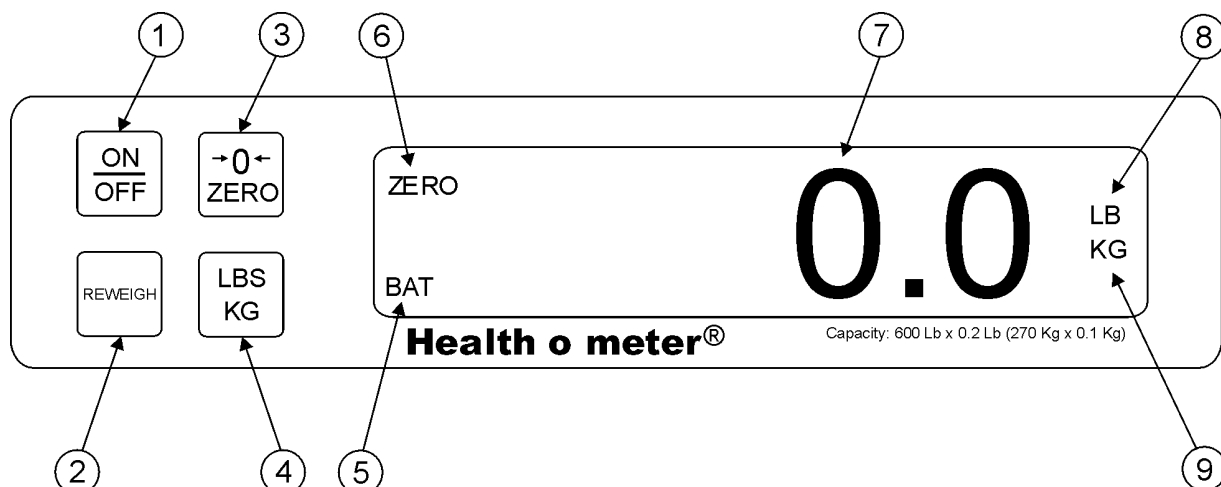
Figure 6. Control Panel

ITEM	DESCRIPTION	FUNCTION
1	ON/OFF button	Turns scale ON and OFF.
2	REWEIGH button	Allows repeated reweighing of person.
3	ZERO button	Zeros digital display prior to weighing. Enables the cancellation (tare) of weight if some object was placed on the scale.
4	LBS/KG button	Selects kilograms or pounds.
5	BAT indicator	When ON, indicates scale batteries are low. The batteries should be replaced.
6	ZERO indicator	When ON, indicates the scale is stable and ready for use.
7	Digital Display 6 Digit LCD	Displays weight in kilograms or pounds as shown by the "LB" or "KG" indicator on the right side of the display. When REWEIGH button is depressed, display will blink "====" until patient's weight is determined.
8	LB indicator	When ON, indicates displaying weight in pounds.
9	KG indicator	When ON, indicates displaying weight in kilograms.

OPERATING INSTRUCTIONS

1. Press the **ON/OFF** button to turn the scale on.
2. Wait until “0.0” appears on the digital display.
3. Position the patient onto the scale. The display will blink “” until the weight of the person is determined.
4. To reweigh the patient without asking the patient to stand up and sit on the scale again, press the **REWEIGH** button.
5. Assist the patient off of the scale. The digital display should continue to read the person’s weight until **REWEIGH** or **ZERO** button is pressed.

Note: When batteries are being used and the scale has been inactive for 2 minutes, the scale will turn itself off to conserve power. To restart the scale, press the **ON/OFF** button.

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5. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
 - mild soap and water solution
 - 70% isopropyl alcohol
 - solution with 1-5% hydrogen peroxide concentrationAfter cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure all scale parts are completely dry after cleaning.
6. Never use rough or abrasive materials to clean the scale, as these will damage the scale’s finish.
7. Do not submerge the scale in water or any other liquid.
8. Do not pour or spray fluids directly on the scale.

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SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	1. Defective battery 2. Faulty electrical outlet 3. Defective power supply	1. Replace batteries 2. Use a different outlet 3. Replace AC adapter
Questionable weight or the scale does not zero	1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration	1. Remove interfering object from the scale 2. Remove the patient from the scale. Zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value
The display shows "OL"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its limits
The display shows "UL"	The scale is in under-load condition	Make sure all seat parts and handles are installed in place and fixed properly
The display shows "LO BAT"	The battery is depleted	Replace batteries according to instructions

CALIBRATION PROCEDURE

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. The following calibration assumes the scale is set to pounds, and 200 lb is employed as the calibration standard.

Note: Only weights certified and traceable to national standards are to be used for calibration procedures.

Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the REWEIGH and ZERO buttons at the same time, while pressing the ON/OFF button to turn on the scale. "LOAD" will briefly appear on the display, followed by "200.0 LB" indicates you can calibrate. Release all three buttons.	"LOAD" followed by "200.0 LB"
Ensure there is no weight on the scale and press the REWEIGH button. "Clear" will flash on the display, followed by "-----". At this time the scale is calculating zero calibration. After zero calibration, the display will flash "PUT" followed by "200.0 LB".	"Clear" followed by "-----". Then "PUT" followed by "200.0 LB"
Place 200 lb on the scale. Then press REWEIGH . "CAL" will appear on the display, followed by "FAC" followed by 5 digits. The calibration factor should be in the range between 8000 and 10000. If the number is out of that range, contact Customer Service at 1-800-815-6615.	"CAL" followed by "FAC" followed by "XXXXX".
If the calibration factor is within range, press the REWEIGH button. "DONE" will appear on the display, followed by "START", followed by "0.0". Remove the weight from the scale.	"DONE" followed by "START", followed by "0.0".
Calibration procedure is complete.	

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