### Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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Date code can be found on the product label on the back of the display head.

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## Calibration

**This scale has been factory calibrated, and does not require calibration prior to use.** There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. Calibration of your scale is performed using 80 kilograms (KG) or 200 pounds (LB), according to the units of measure used upon entering into the calibration path.

If Everlock® is engaged the scale can only be calibrated in the locked unit of measure.

### Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold the **lb./kg** and **ZERO** buttons at the same time, and press and release the **ON** button to turn on the scale. Remain pressing the **lb./kg** and **ZERO** buttons until "CAL" appears on the display. Release all buttons. **Note:** The display will show "CAL" until the button combination is released.
2. Once the buttons are released the following message will appear : "– 2Ero SCALE –". With no weight on the platform, press the **ZERO** button.
3. The scale will zero and the following message will appear: "- PLACE 200 ON SCALE PrESS ON–" (if in pounds) or "– PLACE 80 ON SCALE PrESS ON–" (if in KG). You may toggle between pounds and kilograms using the **lb./kg** button. If Everlock® is engaged you will only be able to calibrate in the locked weight measurement unit.
4. Once the proper weight is placed on the platform, press the **ON** button. The message "– CAL –" will flash on the screen while the scale is calibrating.
  - a. If the weight is within calibration range, a four digit number will display on the screen. Remove the weight from the scale and the scale will restart. Note: the 4 digit calibration count will stay until the weight is removed.  
  
B. If the weight placed on the scale does not match the correct calibration standard (200 lb or 80 kg), the following message will be displayed: "– Out OF RANGE –". Calibration is not performed. Press **ZERO** button to exit the calibration mode. The scale will restart and go through the power up sequence to show 0.0. Power off the scale and begin the calibration process again.

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## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified weights. 50 / 100 / 200 pounds or 40 / 80 / 120 kg are employed as the calibration standards. The display must be in the desired unit of measure (LB or KG) before entering the calibration procedure.

**Caution:** To avoid injury, use extreme care when stacking weights on platform.

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

## Scale Calibration

<b>ACTION</b>	<b>DISPLAY SHOWS</b>
Press and hold the <b>lb./kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "UXX". (Two alphanumeric characters will appear next to "U".) Release all three buttons. Several alphanumeric characters will appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "UXX" then "XXXX"
Press the <b>ZERO</b> button. "C-50" (in LB mode) or "C-40" (in KG mode) will flash on the display, then the display will clear with only the unit of measure shown.	"C-50" (in LB mode) or "C-40" (in KG mode)
Place 50 lbs or 40 kg on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" (in LB mode) or "C-80" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-100" (in LB mode) or "C-80" (in KG mode)
Place an additional 50 lb on the scale, for a total of 100 lb or an additional 40 kg on the scale for a total of 80 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" (in LB mode) or "C-120" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-200" (in LB mode) or "C-120" (in KG mode)
Place an additional 100 lb on the scale for a total of 200 lbs or an additional 40 kg on the scale for a total of 120 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off. Calibration is complete and the scale is ready to use.	"XXXX" followed by "END" then "OFF"

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## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified weights. 50 / 100 / 200 / 400 lbs are employed as the calibration standards. The scale can only be calibrated in pounds (lb).

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## Scale Calibration

<b>ACTION</b>	<b>DISPLAY SHOWS</b>
Press and hold the <b>lb./kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "UXX". (Two alphanumeric characters will appear next to "U".) Release all three buttons. Several alphanumeric characters will appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "UXX" then "XXXX"
Press the <b>ZERO</b> button, and "C-50" will flash on the display, followed by several alphanumeric characters.	"C-50" then "XXXX"
Place 50 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-100" then "XXXX"
Place 100 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-200" then "XXXX"
Place 200 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-400" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-400" then "XXXX"
Place 400 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"

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## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified calibration weights. 50 / 100 / 200 / 400 lbs are employed as the calibration standards.

ACTION	DISPLAY SHOWS
Press and hold the <b>lb/kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "U13". Release all three buttons. Several alphanumeric characters shall appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "U 13" then "XXXX"
Press the <b>ZERO</b> button, and "C-50" will flash on the display, followed by several alphanumeric characters.	"C-50" then "XXXX"
Place 50 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-100" then "XXXX"
Place 100 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-200" then "XXXX"
Place 200 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-400" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-400" then "XXXX"
Place 400 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"

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