### Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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Date code can be found on the product label on the back of the display head.

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# Model 349KLX/349KGX

**CLICK THE BOX BELOW OR**  
**SKIP TO THE PAGE NOTED**

<b>Cautions and Warnings</b> p. 3	<b>Specifications / Definition of Symbols</b> p. 4	<b>Certifications / Disposal</b> p.5
<b>Assembly Instructions</b> p.6	<b>Powering the Scale / Battery Replacement</b> p.10	<b>Transport Instructions</b> p.11
<b>Set Up Instructions</b> p.12	<b>Everlock Permanent Unit Lock Instructions</b> p.14	<b>Operating Instructions</b> p.15
<b>Maintenance</b> p.17	<b>Calibration</b> p.18	<b>Troubleshooting</b> p.19
<b>Warranty</b> p.20	<b>Español</b> p.21	<b>Français</b> p.41

## CAUTIONS AND WARNINGS

**To prevent patient/caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.**

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not transport the scale with a patient or object on the scale.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.

### **Patient/caregiver safety**

- Patient should wear socks or other form of lightweight foot covering during weighing event.
- To prevent patient injury, the patient must be attended throughout the entire weighing event. Caregivers should ensure that the patient is stable and provide support as needed when getting onto and off of the scale.

### **When operating the scale with the AC adapter:**

- Operate this scale exclusively with the AC adapter provided by Health o meter Professional. Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- For use with AC adapter model ADPT40 (USA or Canada) or ADPT30 (International) (not included)
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

### **When operating the scale with batteries:**

- For accurate weighing, if the “LO” indicator activates, replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

**In no event shall Pelstar®, LLC be liable for damages or injuries whatsoever arising out of or connected with the assembly, use, or misuse of its products.**

# SPECIFICATIONS

## General

The Model 349KLX/349KGX remote display digital scale uses highly sophisticated microprocessor technology. Each scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

This scale is set up to use weighing technology that is able to determine the actual weight of a moving patient.

Weight can be displayed in pounds (lb) or in kilograms (kg). For privacy and convenience, the scale features a remote display head which may be held by the operator, placed on a table or other flat surface, or mounted on a wall. **This scale can be powered using an AC adapter, (not included), or by 6 AA batteries (included).**

## 349KLX Scale Specifications

Capacity and Resolution	349KLX: 400 lb x 0.2 lb (180 kg x 0.1 kg) 349KGX: 180 kg x 0.1 kg	
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE160714HKKK1-P Health o meter® Professional Model ADPT30 (Not Included) Input: 100-240V – 50/60Hz, 500mA Output: 9.0V - - - 0.5A  Or 6 AA Batteries (included)	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 32°F to 125°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	<b>Platform</b> Length: 12" (305 mm) Width: 12.5" (317.5 mm) Height: 1.875" (48 mm) Weight: 8.8 lb (4.0 kg)	<b>Display assembly</b> Length: 7.75" (197 mm) Width: 1.5" (15 mm) Height: 4.25" (108 mm) Weight: 12.8 oz (0.36 kg)

## CERTIFICATIONS / DISPOSAL

### Certification Descriptions

	<p>Pelstar®, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar®, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
	

### 349KLX Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or battery. Do not dispose of this device in the domestic waste stream.

# ASSEMBLY INSTRUCTIONS

## **Before Assembly**

Each 349KLX/349KGX remote display digital scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

## **Parts List**

### **Carton**

- (1) Scale platform
- (1) Display assembly
- (6) AA batteries
- (1) User instructions
- (1) Metal Stand
- (2) #8 Anchoring screws (4.2 mm nominal OD)
- (2) Drywall mounting anchors

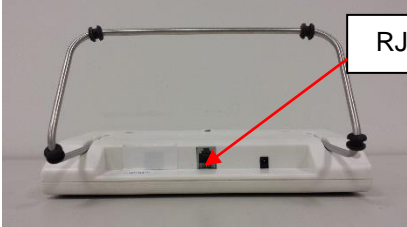
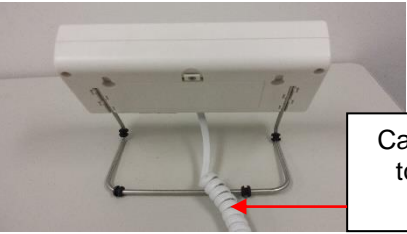
## **Tools Required**

Phillips head screwdriver (not included)

**Note: This scale has been factory calibrated, and does not require calibration prior to use.**

# DISPLAY ASSEMBLY TABLE TOP USE INSTRUCTIONS

## Assembling the Scale

<p>1. Carefully remove the scale platform and display assembly from the carton, and place on a flat, level, and dry surface.</p>	
<p>2. If using the display assembly on a desk or table, insert the metal stand into the slots on the back of the display.</p>	
<p>3. Connect the cable to the RJ Jack in the bottom of the display assembly. Ensure that the cable rests on top of the metal stand so the display assembly rests flat on the table. If using the scale on battery power the scale is now ready for use. If using the scale with the optional AC adapter see page 10 for set up instructions.</p>	   

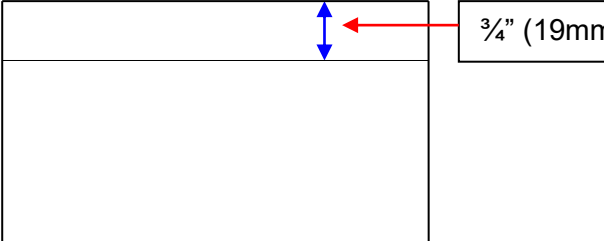
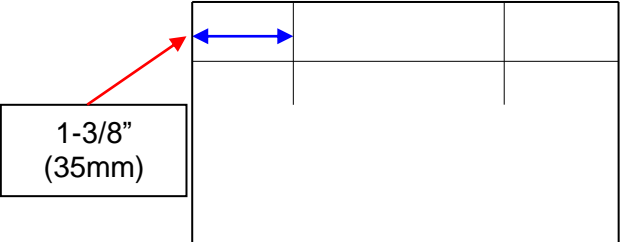
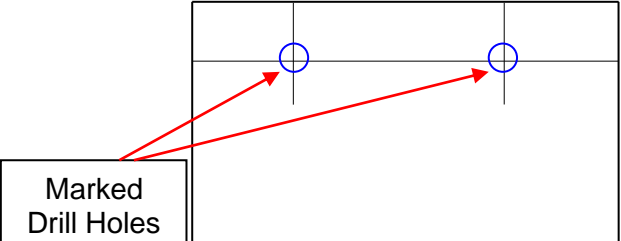
# DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS

## Required Parts

- (1) Display assembly
- (2) #8 Anchoring screws (4.2 mm nominal OD)
- (2) Mounting anchors, compatible with wall construction

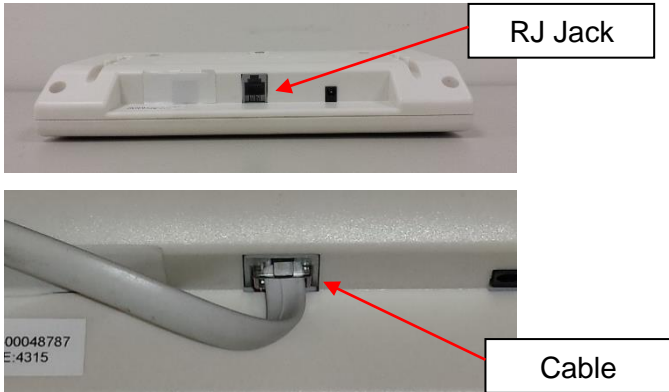
**Caution:** When wall mounting the display assembly, ensure that the mounting location is free of electrical lines, gas pipes, and other safety hazards. Ensure that wall is of construction that can support the display assembly.

**Note:** Remote head can be mounted to a wall up to 6 feet (1.83 m) from the base or it can be mounted on the display bracket (included) for placement on a table up to 6 feet (1.83 m) from the base.

<p>1. Position the display assembly on the wall, and carefully trace around the display.</p>	
<p>2. Mark a level horizontal line, 3/4" (19 mm) below the top of the display outline.</p>	
<p>3. Mark a vertical line 1-3/8" (35 mm) from each side of the display outline.</p>	
<p>4. At the horizontal and vertical intersection, carefully drill the appropriate size hole.</p>	


# DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS (CONT)

<p>5. Once the holes are marked, use a hammer or Phillips head screwdriver to set the 2 drywall anchors into the holes. Use a screwdriver to insert the drywall anchors completely into the wall. Screw the two mounting screws into the anchors. Adjust the screw head depth such that the screw heads protrude 1/8" (3.2 mm) from the wall.</p> <p><b>Warning: Extending the screw heads beyond 1/8" (3.2 mm) can damage the display assembly.</b></p>	
<p>6. Position the display assembly mounting holes onto the mounting screws, and carefully pull down on the display assembly to secure.</p>	
<p>7. Connect the cable to the RJ jack in the bottom of the display assembly. If using the scale on battery power the scale is now ready for use. If using the scale with the optional AC adapter see set up instructions.</p>	

## POWERING THE SCALE
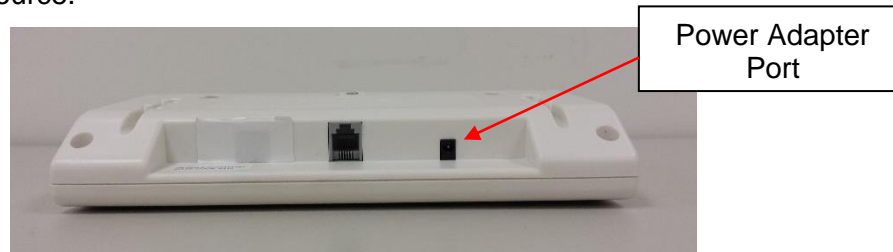
### Powering the Scale

If powering the scale via the optional AC adapter, plug the adapter into the display assembly and into a power source.


For battery operation, see Battery Installation below.

### Battery Installation

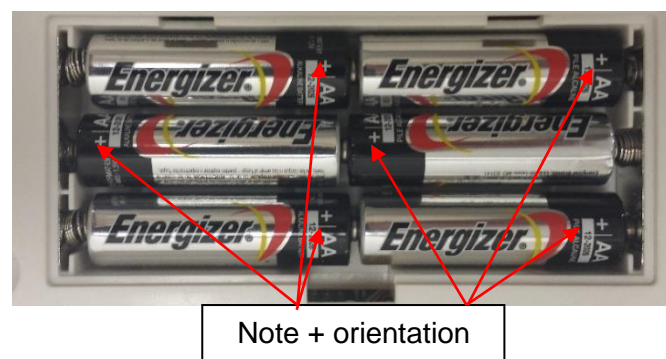1. Unplug the AC adapter from the power source.
2. Use a Phillips head screwdriver to remove the battery cover on the back of the display assembly.

3. Replace or install fresh batteries. Note the orientation that is shown and ensure the batteries are placed in the proper direction.

**Note:** Health o meter® Professional recommends using alkaline or lithium batteries, rated for 130°F (54°C) operation. Do not use zinc-carbon batteries.

4. Re-attach the battery cover to the display assembly.

**Warning:** If the scale will not be used for some time, remove the batteries to avoid a safety hazard. Disposal of batteries must be performed in accordance with the prevailing national, regional or local regulations that apply to you.

## TRANSPORTING THE SCALE WITH OPTIONAL CARRYING CASE

### Preparing the Scale for Transport

This scale can be easily placed into a carrying case (item # 64771) for storage, or transport from one room to another. To prepare the scale for transport, follow these steps:

Optional  
Carrying Case  
Item # 64771  
Sold Separately

1. Remove the display assembly and display bracket from its current position.
2. Unplug the AC adapter and cable from the display assembly, and remove the AC adapter from the outlet.
3. Carefully store the AC adapter, display bracket, and display assembly in carrying case to prevent damage during transport.
4. Lift scale platform carefully from the floor and place it into the carrying case. Position the scale platform so that the cable is facing up.

5. Close carrying case flap to cover the scale. Grasp the carrying case strap and carefully lift the case into the upright position. The scale can now be transported.

## SET UP INSTRUCTIONS

### **Enable / Disable Auto Off**

If powering the scale via the AC adapter, the display can be set to power off after a period of inactivity. The Auto Off default setting is “disabled”. Follow this procedure to enable or disable the Auto Off function.

Note: If powering the scale via batteries, the scale will always power off after a period of inactivity.

1. While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until “AF=d” or “AF=E” appears on the display, then release. Press the **lb./kg** button to change the setting.  
 AF=d Disables the Auto Off function  
 AF=E Enables the Auto Off function
2. Press **ZERO** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

### **Setting Auto Off Time**

When powering the scale via batteries the period of inactivity before the scale powers off can be set to a specific amount of time. The default setting for Auto Off time is 2 minutes. Follow this procedure to set the number of minutes of inactivity before power off.

1. While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until “AF=d” or “AF=E” appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until “AT” appears. Press the **lb./kg** button to change the setting.  
 At = 1 Scale will power off after 1 minute of inactivity  
 At = 2 Scale will power off after 2 minutes of inactivity  
 At = 3 Scale will power off after 3 minutes of inactivity
2. Choose the setting you prefer and press **ZERO** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

### **Setting Backlight Brightness**

This option allows the user to set the brightness of the backlight on the display. Users can choose from four levels of brightness or choose to disable the backlight. The brightness default setting is level 4 – the brightest level. Follow this procedure to set the brightness of the display or disable the backlight.

**Note:** If the scale is being powered by batteries, the backlight will dim to level 1 when the scale is in idle mode.

**Note:** High levels of brightness will utilize more battery power.

1. While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until “AF=d” or “AF=E” appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until “BL” appears. Press the **lb./kg** button to change the setting.  
 bL = 0 Disables the backlight  
 bL = 1 Lowest level of brightness  
 bL = 2 2<sup>nd</sup> lowest level of backlight brightness  
 bL = 3 2<sup>nd</sup> highest level of brightness  
 bL = 4 Highest level of backlight brightness
2. Choose the setting you prefer and press **ZERO** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

## SET UP INSTRUCTIONS (CONT)

### Changing the Weight Measuring Unit

1. Press the **ON** button to power the scale on.
2. Wait for "0.0" LB or "0.0" KG to appear on the display.
3. Press the **lb./kg** button to toggle to the desired weight measuring unit.

### Locking Unit of Measure

#### **(This feature is not available on KG models)**

The weight measuring unit (pounds/LB, kilograms/KG) can be locked to display weight only in the selected unit of measure. The unit of measure lock default setting is "unlocked". Follow this procedure to lock the **lb./kg** button.

**Note:** This lock is not permanent and can be unlocked by the user. For a permanent lock see the Everlock section on page 14.

**Note:** If Everlock<sup>®</sup> is engaged, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until "AF=d" or "AF=E" appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until "UL" appears. Press the **lb./kg** button to change the setting.

UL = U Allows the user to toggle weight measurement between LB and KG.

UL = L Locks the unit button so weight is only displayed in pounds (LB)

UL = 6 Locks the unit button so weight is only displayed in kilograms (KG)

1. Choose the setting you prefer and press **ZERO** to save your selection and exit the options mode.
2. The scale will restart. The display will show "UEr", followed by a version number, and then dashes. When the display shows "0.0" the scale is ready to use.

### Enabling Unit Display with Unit of Measure Locked

#### **(This feature is not available on KG models)**

Users can still view the alternate weight measuring unit by enabling the "Unit Display" option. Enabling the "Unit Display" option allows the user to press and hold the **lb./kg** button to briefly display the weight in the alternate weight measuring unit. If locked in kilograms or grams, Unit Display will allow you to view the weight in pounds. If locked in pounds, Unit Display will allow you to view the weight in kilograms. If locked in grams, Unit Display will allow you to view the weight in pounds. After releasing the **lb./kg** button the display will return to the locked weight measuring unit. The "Unit Display" default setting is "Disable". Follow this procedure to enable "Unit Display" when the weight measuring unit is locked. **Note:** If Everlock<sup>®</sup> is enabled, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until "AF=d" or "AF=E" appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until "UD" appears. Press the **lb./kg** button to change the setting.
  - UD = d Disables the **lb./kg** button so it will not display the other weight measuring unit.
  - UD = E Allows the user to view the other weight measuring unit by holding down the **lb./kg** button even if the weight measuring unit is locked. Enabling this function will only allow the user to view the other unit while pressing the **lb./kg** button; it will not unlock the weight measuring unit.
2. Choose the setting you prefer and press **ZERO** to save your selection and exit the Options mode.
3. The scale will restart. The display will show "UEr", followed by a version number, and then dashes. When the display shows "0.0" the scale is ready to use.

## SET UP INSTRUCTIONS (CONT)

### Setting Auto Zero

This option allows the user to select whether the scale will re-set itself to ZERO each time it is powered up or whether it will use the zero reading from the last time it was zeroed. The auto zero default setting is “enabled” Follow this procedure to enable or disable this feature.

- While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until “AF=d” or “AF=E” appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until “P2” appears. Press the **lb./kg** button to change the setting.
 

P2 = d     Disabling this feature means the scale does not attempt to find a zero when powered on, rather it will use the zero setting it had from its most recent use. Health o meter<sup>®</sup> Professional recommends that if the Disable option is selected, users should manually zero the scale before each weight measurement by pressing **ZERO**. Users will see dashes on the screen as the scale is finding zero.

P2 = E     Enabling this feature means the scale will zero itself when it is powered on. After each use, the scale will clear the weight of the patient and set itself back to the initial zero level it found on start-up. Health o meter<sup>®</sup> Professional recommends keeping this setting as Enable.
- Choose the setting you prefer and press **ZERO** to save your selection and exit the Options mode.
- The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

### Everlock<sup>®</sup>

**(349KLX only, this feature is not available on 349KGX)**

The Everlock<sup>®</sup> feature allows the measuring unit to be permanently locked on the scale, disabling the **lb./kg** button. The Everlock<sup>®</sup> default setting is “disabled”. Follow this procedure to engage Everlock<sup>®</sup>.

**Warning: This lock is permanent and cannot be reversed.**

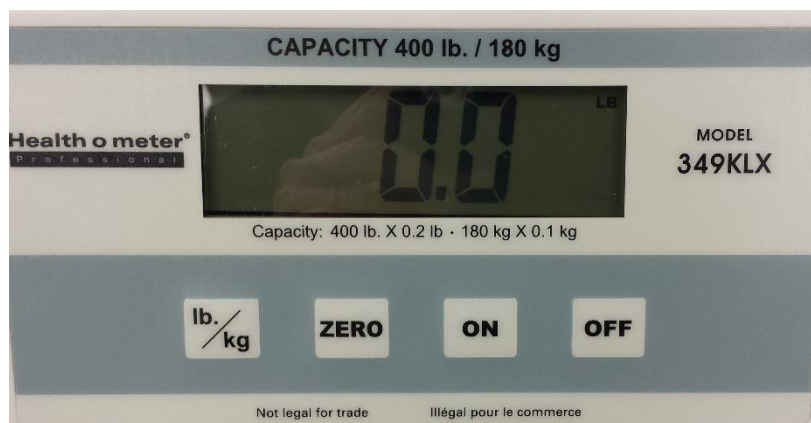
#### Notes:

- Press **ZERO** at any time to exit the Everlock<sup>®</sup> procedure.
  - Once Everlock<sup>®</sup> is engaged the UL and UD options will no longer be displayed when scrolling through the Options menu.
  - There is no method to undo Everlock<sup>®</sup> without sending the scale back to the manufacturer.
- While the scale is turned off, press and hold the **ZERO** button then press and release the **ON** button. Hold the **ZERO** button until “AF=d” or “AF=E” appears on the display, then release. Use the **ON** or **OFF** buttons to advance the display screen until “EL” appears. Press the **lb./kg** button to change the setting.
 

EL = d     Default setting with Everlock<sup>®</sup> disabled.

EL = E     Engages Everlock<sup>®</sup>. Once this option is selected, the display will prompt you to begin the locking procedure. If this is desired, follow the instructions below.
  - After Everlock<sup>®</sup> is selected, “PASS1” will appear on the display. This prompts the user to enter a password. Press the following keys in this sequence:  
**lb./kg, OFF, ON**
  - If entered correctly, “EL=L” (if scale was in LB mode) or “EL=6” (if scale was in KG mode) will appear on the display.
  - Press **lb./kg** to toggle between (L) Pounds and (6) Kilograms. When the desired unit is displayed, press the **ON** button and “PASS2” will appear on the display.
  - This will prompt the user to enter the second password. Press the following keys in this sequence:  
**OFF, lb./kg, ON**
  - If entered correctly, the scale will display “EL = L” or “EL = 6,” depending on the unit of measurement that was selected. Everlock<sup>®</sup> is now engaged.

# OPERATING INSTRUCTIONS

## Keypad

Function	Description
<b>ON</b>	Powers scale on. Press to Reweigh patient after weight locks.
<b>OFF</b>	Powers scale off. Press to Hold the weight measurement after the weight locks.
<b>ZERO</b>	Zeroes or tares the scale prior to a weighing.
<b>lb. / kg</b>	Toggles between pounds (lb) and kilograms (kg).

## Weighing a Patient

1. Make sure there is nothing on the weighing platform.
2. Press the **ON** button to power the scale on.
3. Wait until "0.0" LB or "0.0" KG appears on the display. Press the **ZERO** button to zero the scale, if necessary.
4. Assist the patient onto the scale. "LOCK" will appear and flash on the display along with a weight measurement. When a stable weight is determined "LOCK" will stay displayed along with the patient's weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.  
**Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
  - To reweigh the patient without having them step off the scale press the **ON** button after the weight locks.
  - To hold the locked weight, press the **OFF** button. To release the held weight press the **OFF** button again.
5. After recording the weight, have the patient step off the scale.
6. Press and hold the **OFF** button to power the scale off.

## OPERATING INSTRUCTIONS (CONT)

### Tare Function

When using this scale, the weight of an object can be subtracted from the total weight in order to determine the weight of the patient alone. The Tare Function (**ZERO** button) automatically performs this subtraction.

1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Place the object to be tared onto the weighing platform. The display will show a value for the objects weight.
4. With the object remaining on the scale, press the **ZERO** button. The word "TARE" will appear on the display, followed by "0.0" LB or "0.0" KG.  
Note: Tare function will only work on a locked weight.
5. Place the patient onto the scale. The scale will automatically deduct the weight of the object and only display the patient's weight.
6. After recording the weight, have the patient step off the scale.  
Note: With no weight on the scale, the display will show the tare value as a negative amount.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

### Child/Parent Function

The Parent/Child function on Health o meter® Professional scales provides the convenience of weighing a small child or infant while being held by a parent. Follow these simple steps to subtract the parent's weight from the total so only the child's weight is displayed.

1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Ask the parent to step onto the scale without the child. The display will show a value for the parent's weight.
4. Have the parent remain standing on the scale and press the **ZERO** button. The word "TARE" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the child into the parent's arms. The scale will automatically deduct the weight of the parent and only display the child's weight.
6. When the parent steps off of the scale, a negative weight will appear on the display.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

# MAINTENANCE

## **Maintenance**

The following pages provide instructions for maintenance, cleaning, calibration, and troubleshooting for the scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

**Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.**

1. Check overall appearance of the scale for obvious damage, wear, and tear.
2. Inspect the AC adapter for cord cracking or fraying, or for broken/bent prongs.

**Note:** This scale is an extremely sensitive weighing device. If the auto-off function is disabled, battery power will be consumed at a rapid rate.

## **Cleaning and Disinfecting**

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

**Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.**

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
  - mild soap and water solution
  - 70% isopropyl alcohol
  - solution with 1-5% hydrogen peroxide concentration

After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.

2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

## **Replacement Parts**

Replacement parts may be available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

<b>Part #</b>	<b>Description</b>
ADPT40	Power adapter (USA and Canada)
ADPT30	Power adapter (International)

## CALIBRATION

**This scale has been factory calibrated, and does not require calibration prior to use.** There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. Calibration of your scale is performed using 80 kilograms (KG) or 200 pounds (LB), according to the units of measure used upon entering into the calibration path.

If Everlock® is engaged the scale can only be calibrated in the locked unit of measure.

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

1. Press and hold the **lb./kg** and **ZERO** buttons at the same time, and press and release the **ON** button to turn on the scale. Remain pressing the **lb./kg** and **ZERO** buttons until "CAL" appears on the display. Release all buttons. **Note:** The display will show "CAL" until the button combination is released.
2. Once the buttons are released the following message will appear : "– 2Ero SCALE –". With no weight on the platform, press the **ZERO** button.
3. The scale will zero and the following message will appear: "- PLACE 200 ON SCALE PrESS ON–" (if in pounds) or "– PLACE 80 ON SCALE PrESS ON–" (if in KG). You may toggle between pounds and kilograms using the **lb./kg** button. If Everlock® is engaged you will only be able to calibrate in the locked weight measurement unit.
4. Once the proper weight is placed on the platform, press the **ON** button. The message "– CAL –" will flash on the screen while the scale is calibrating.
  - a. If the weight is within calibration range, a four digit number will display on the screen. Remove the weight from the scale and the scale will restart. Note: the 4 digit calibration count will stay until the weight is removed.  
  
B. If the weight placed on the scale does not match the correct calibration standard (200 lb or 80 kg), the following message will be displayed: "– Out OF RANGE –". Calibration is not performed. Press **ZERO** button to exit the calibration mode. The scale will restart and go through the power up sequence to show 0.0. Power off the scale and begin the calibration process again.

# TROUBLESHOOTING

## Troubleshooting

Before contacting service personnel, refer to the following instructions to check and correct any failures.

Symptom	Possible Cause	Corrective Action
Scale does not power on	<ol style="list-style-type: none"> <li>1. Dead batteries</li> <li>2. Faulty electrical outlet</li> <li>3. Bad power supply</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace batteries</li> <li>2. Use a different outlet</li> <li>3. Replace AC adapter</li> </ol>
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> <li>1. External object interfering with the scale</li> <li>2. The display did not show "0.0" before weighing</li> <li>3. Scale is not placed on a level floor</li> <li>4. Scale is out of calibration</li> </ol>	<ol style="list-style-type: none"> <li>1. Remove interfering object from the scale</li> <li>2. Ask the patient to get off the scale, zero the scale and begin weighing process again</li> <li>3. Place the scale on a level floor and begin weighing process again</li> <li>4. Check weight with known weight value</li> </ol>
Weighing is performed but the weighing process takes too long and the weight does not lock on the display	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (400 lb / 180 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The batteries are depleted	Replace batteries according to instructions
The display shows "UNdr" message	The weight on the scale is below the last zero value	Press the ZERO button to re-zero the scale

# WARRANTY

## Limited Warranty

### What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar®, LLC (see “To Get Warranty Service”, below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer’s sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar® dealers or retail stores selling Pelstar® products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

### What is Excluded?

**Your warranty does not cover normal wear of parts or damage resulting from any of the following:** negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

**To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase.** Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC  
Attention R/A# \_\_\_\_\_  
Return Department  
9500 West 55<sup>th</sup> St.  
McCook, IL 60525

### Extended Warranty Available\*

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility’s existing scale before its current warranty has expired. To learn more, visit [www.homscales.com/scalesurance/](http://www.homscales.com/scalesurance/) or contact your medical supply distributor.

\*Not available in all countries

9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

[www.homscales.com](http://www.homscales.com)

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar®, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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# SPECIFICATIONS

## General

The Model 349KLX remote display digital scale uses highly sophisticated microprocessor technology. Each scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

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## 349KLX Scale Specifications

Capacity and Resolution	400 lb x 0.2 lb (180 kg x 0.1 kg)	
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE160714HKKK1-P Health o meter® Professional Model ADPT30 (Not Included) Input: 100-240V – 50/60Hz, 500mA Output: 9.0V - - - 0.5A  Or 6 AA Batteries (included)	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 32°F to 125°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	<b>Platform</b> Length: 12" (305 mm) Width: 12.5" (317.5 mm) Height: 1.875" (48 mm) Weight: 8.8 lb (4.0 kg)	<b>Display assembly</b> Length: 7.75" (197 mm) Width: 1.5" (15 mm) Height: 4.25" (108 mm) Weight: 12.8 oz (0.36 kg)

## CERTIFICATIONS / DISPOSAL

### Certification Descriptions

	<p>Pelstar®, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar®, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
---	---

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# ASSEMBLY INSTRUCTIONS

## **Before Assembly**

Each 349KLX remote display digital scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

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
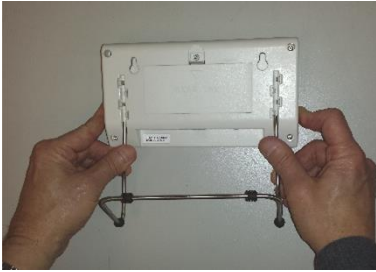
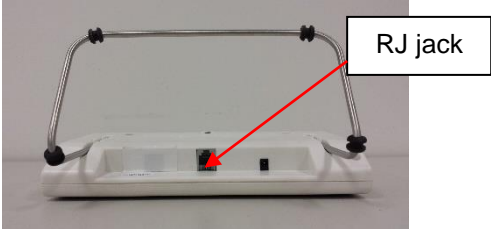
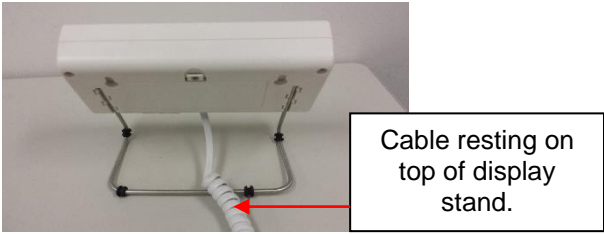
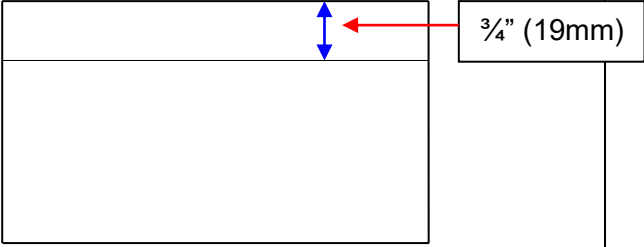
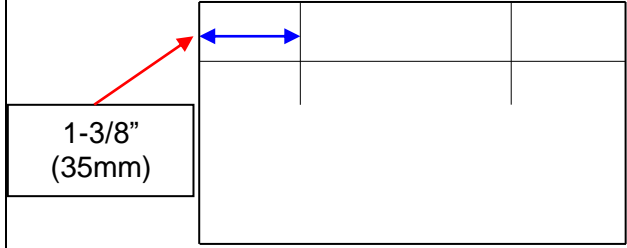
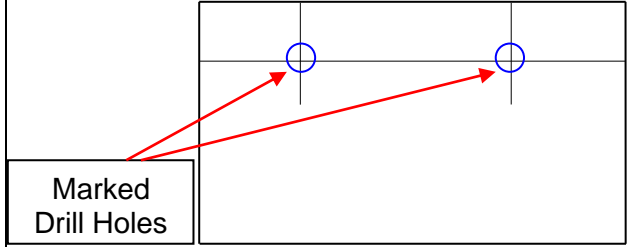
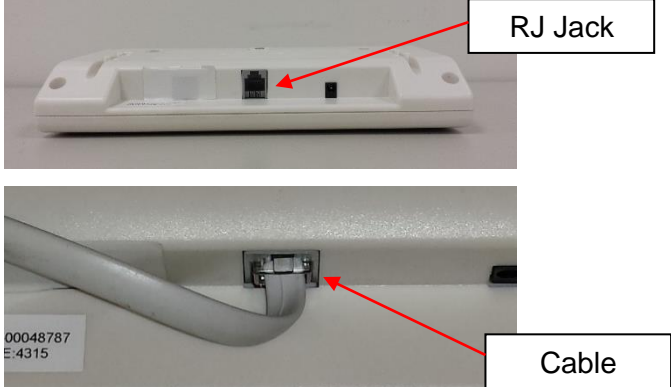
## DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS (CONT)

<p>5. Once the holes are marked, use a hammer or Phillips head screwdriver to set the 2 drywall anchors into the holes. Use a screwdriver to insert the drywall anchors completely into the wall. Screw the two mounting screws into the anchors. Adjust the screw head depth such that the screw heads protrude 1/8" (3.2 mm) from the wall.</p> <p><b>Warning: Extending the screw heads beyond 1/8" (3.2 mm) can damage the display assembly.</b></p>	
<p>6. Position the display assembly mounting holes onto the mounting screws, and carefully pull down on the display assembly to secure.</p>	
<p>7. Connect the cable to the RJ jack in the bottom of the display assembly. If using the scale on battery power the scale is now ready for use. If using the scale with the optional AC adapter see page 10 for set up instructions.</p>	

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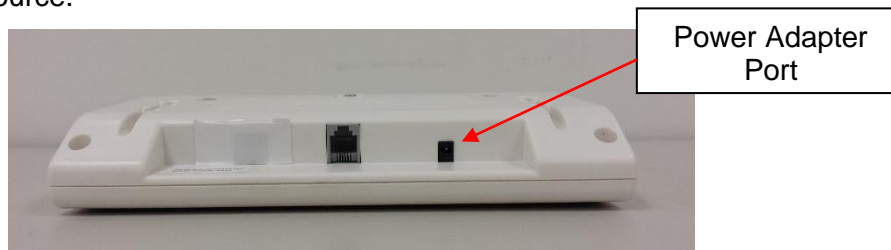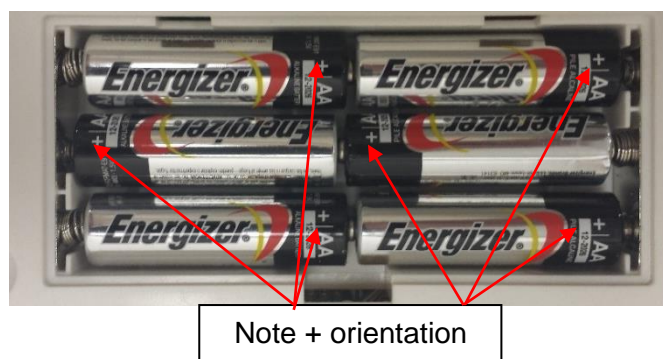
### Preparing a 349KLX Scale for Transport

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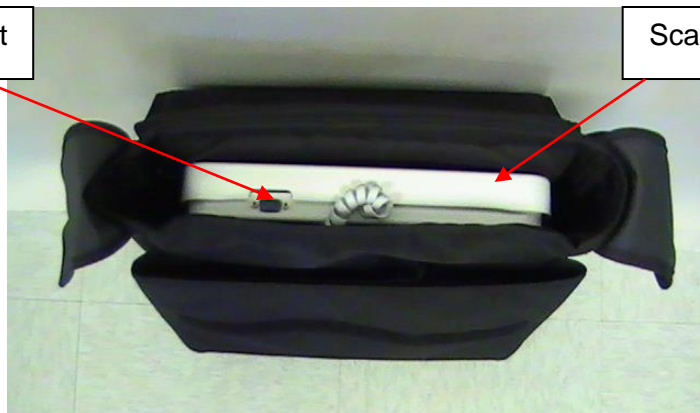
Optional  
Carrying Case  
Item # 64771  
Sold Separately

1. Remove the display assembly and display bracket from its current position.
2. Unplug the AC adapter and cable from the display assembly, and remove the AC adapter from the outlet.
3. Carefully store the AC adapter, display bracket, and display assembly in carrying case to prevent damage during transport.
4. Lift scale platform carefully from the floor and place it into the carrying case. Position the scale platform so that the RS-232 serial port and cable are facing up.

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
# OPERATING INSTRUCTIONS
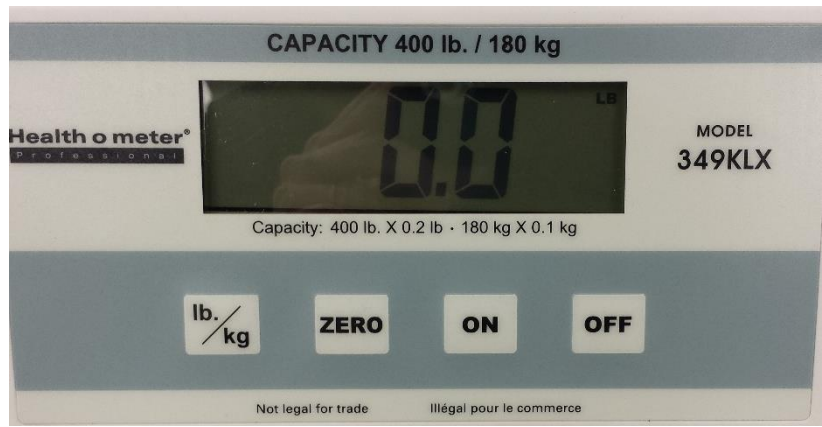
## Keypad

Function	Description
<b>ON</b>	Powers scale on.
<b>OFF</b>	Powers scale off.
<b>ZERO</b>	Zeroes or tares the scale prior to a weighing.
<b>lb. / kg</b>	Toggles between pounds (lb) and kilograms (kg).

### Weighing a Patient

1. Make sure there is nothing on the weighing platform.
2. Press the **ON** button to power the scale on.
3. Wait until "0.0" LB or "0.0" KG appears on the display.
4. Press the **ZERO** button to zero the scale, if necessary.
5. Have the patient step onto the scale. Wait for the weight reading to stabilize. When the weight stabilizes, "Lock" will appear on the display along with the patient's weight.  
**Note:** Depending on the movement of the patient on the scale, it may take approximately 3-4 seconds for the weight reading to stabilize.  
**Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
6. After recording the weight, have the patient step off the scale.
7. Press the **OFF** button to power the scale off.

## OPERATING INSTRUCTIONS (CONT)

### Auto-Off Function

As an energy saving feature, the scale automatically powers off after 2 minutes of inactivity. This function is enabled by default.

### Disable Auto-Off

1. While the scale is powered off, press and hold the **lb./kg** button.
2. While holding the **lb./kg** button, press the **ON** button to power the scale on.
3. "349" will briefly appear on the display followed by "A-OF" and then "0.0" lb appears.
4. Release the **lb./kg** and **ON** button.
5. Auto-off is now disabled, and the scale will remain on until the scale is manually powered off.

### Enable Auto-Off

1. While the scale is powered off, press and hold the **lb./kg** button.
2. While holding the **lb./kg** button, press the **ON** button to power the scale on.
3. "349" will briefly appear on the display, followed by "0.0" lb.
4. Release the **lb./kg** and **ON** button.
5. Auto-off is now enabled.

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### Locking the Weight Measuring Unit

The weight measuring unit can be locked to only measure in KG or LB. Follow the procedures below to lock the weight measuring unit.

#### KG Lock Instructions

1. Press the **ON** button to turn the scale on. Wait for "0.0" to appear on the display.
2. Press and hold the **lb./kg** button so that KG icon is displayed and continue holding the **lb./kg** button for approximately 10 seconds.
3. When the KG icon flashes, release the **lb./kg** button. The KG weight measuring unit is now locked and will remain locked even if the scale is turned off.

#### LB Lock Instructions

**Note:** To lock in LB weight mode, you must follow the instructions above to lock in KG weight mode first.

1. Once the scale is locked in KG mode, press and hold the **lb./kg** button for approximately 10 seconds until the LB icon flashes.
2. When the LB icon flashes, release the **lb./kg** button. The LB weight measuring unit is now locked and will remain locked even if the scale is turned off.

#### Unlocking the Weight Measuring Unit

**Note:** If your scale is locked in KG mode, you must first follow the instructions above to lock into LB mode, and then begin the disabling instructions below.

1. Press the **ON** button to turn the scale on. Wait for "0.0" to appear on the display.
2. Press and hold the **lb./kg** button until LB and KG icons *both* flash. When LB and KG are flashing, release the **lb./kg** button. The weight measuring unit is now unlocked; pressing the **lb./kg** button will allow user to toggle between LB and KG.

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1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Place the object to be tared onto the weighing platform. The display will show a value for the objects weight.
4. With the object remaining on the scale, press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the patient onto the scale. The scale will automatically deduct the weight of the object and only display the patient's weight.
6. With all the weight off the scale, "under" will be temporally displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

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1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Ask the parent to step onto the scale without the child. The display will show a value for the parent's weight.
4. Have the parent remain standing on the scale and press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the child into the parent's arms. The scale will automatically deduct the weight of the parent and only display the child's weight.
6. When the parent steps off of the scale, "under" will be temporally displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

# MAINTENANCE

## **Maintenance**

The following pages provide instructions for maintenance, cleaning, calibration, and troubleshooting for the 349KLX scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

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# CALIBRATION

## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified weights. 50 / 100 / 200 pounds or 40 / 80 / 120 kg are employed as the calibration standards. The display must be in the desired unit of measure (LB or KG) before entering the calibration procedure.

**Caution:** To avoid injury, use extreme care when stacking weights on platform.

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## Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the <b>lb./kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "UXX". (Two alphanumeric characters will appear next to "U".) Release all three buttons. Several alphanumeric characters will appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "UXX" then "XXXX"
Press the <b>ZERO</b> button. "C-50" (in LB mode) or "C-40" (in KG mode) will flash on the display, then the display will clear with only the unit of measure shown.	"C-50" (in LB mode) or "C-40" (in KG mode)
Place 50 lbs or 40 kg on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" (in LB mode) or "C-80" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-100" (in LB mode) or "C-80" (in KG mode)
Place an additional 50 lb on the scale, for a total of 100 lb or an additional 40 kg on the scale for a total of 80 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" (in LB mode) or "C-120" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-200" (in LB mode) or "C-120" (in KG mode)
Place an additional 100 lb on the scale for a total of 200 lbs or an additional 40 kg on the scale for a total of 120 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off. Calibration is complete and the scale is ready to use.	"XXXX" followed by "END" then "OFF"

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<b>Symptom</b>	<b>Possible Cause</b>	<b>Corrective Action</b>
Scale does not power on	<ol style="list-style-type: none"> <li>1. Dead batteries</li> <li>2. Faulty electrical outlet</li> <li>3. Bad power supply</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace batteries</li> <li>2. Use a different outlet</li> <li>3. Replace AC adapter</li> </ol>
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> <li>1. External object interfering with the scale</li> <li>2. The display did not show "0.0" before weighing</li> <li>3. Scale is not placed on a level floor</li> <li>4. Scale is out of calibration</li> </ol>	<ol style="list-style-type: none"> <li>1. Remove interfering object from the scale</li> <li>2. Ask the patient to get off the scale, zero the scale and begin weighing process again</li> <li>3. Place the scale on a level floor and begin weighing process again</li> <li>4. Check weight with known weight value</li> </ol>
Weighing is performed but the weighing process takes too long and the weight does not lock on the display	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (400 lb / 180 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The batteries are depleted	Replace batteries according to instructions

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**9500 West 55th St. McCook, IL 60525-7110 USA**

**1-800-638-3722 or 1-708-377-0600**

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:**

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## Model 349KLX

**CLICK THE BOX BELOW OR  
SKIP TO THE PAGE NOTED**

**Cautions and  
Warnings  
p. 113**

**Specifications /  
Definition of  
Symbols  
p. 114**

**Certifications /  
Disposal  
p.115**

**Assembly  
Instructions  
p.116**

**Powering the  
Scale / Battery  
Replacement  
p.120**

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### **When operating the scale with the optional AC adapter:**

- Operate this scale exclusively with the AC adapter provided by Health o meter Professional. Use of an unspecified adapter will void the warranty and can pose a serious safety hazard.
- For use with AC adapter model ADPT40
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

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In no event shall Pelstar, LLC be liable for damages or injuries whatsoever arising out of or connected with the assembly, use, or misuse of its products.

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This scale is set up to use weighing technology that is able to determine the weight of a moving patient.

Weight can be displayed in pounds (lb) or in kilograms (kg). For privacy and convenience, the scale features a remote display head which may be held by the operator, placed on a table or other flat surface, or mounted on a wall. **This scale can be powered using an AC adapter, model ADPT40 (not included), or by 6 AA batteries (included).**

## 349KLX Scale Specifications

Capacity and Resolution	400 lb x 0.2 lb (180 kg x 0.1 kg)	
Power Requirements	120 VAC, 60Hz - DC adapter (model ADPT40) or 6 AA batteries.	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 32°F to 125°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	<b>Platform</b> Length: 12" (305 mm) Width: 12.5" (318 mm) Height: 1.875" (48 mm) Weight: 8.8 lb (4.0 kg)	<b>Display assembly</b> Length: 7.75" (197 mm) Width: 1.5" (15 mm) Height: 4.25" (108 mm) Weight: 12.8 oz (0.36 kg)

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## DISPLAY ASSEMBLY TABLE TOP USE INSTRUCTIONS

### Assembling the 349KLX Scale

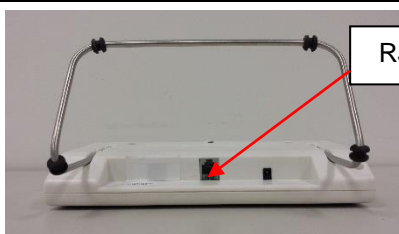
1. Carefully remove the scale platform and display assembly from the carton, and place on a flat, level, and dry surface.

2. If using the display assembly on a desk or table, insert the metal stand into the slots on the back of the display.

3. Connect the cable to the RJ Jack in the bottom of the display assembly. Ensure that the cable rests on top of the metal stand so the display assembly rests flat on the table. If using the scale with batteries the scale is now ready for use. If using the scale with the optional AC adapter, see page 10 for set up instructions.

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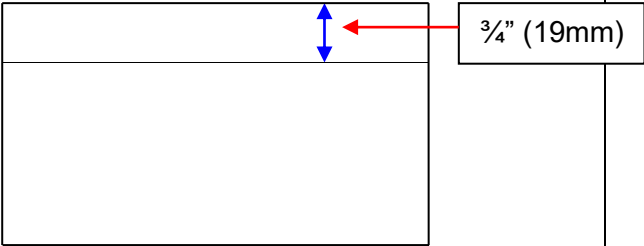
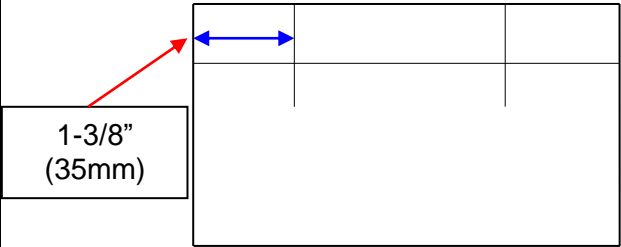
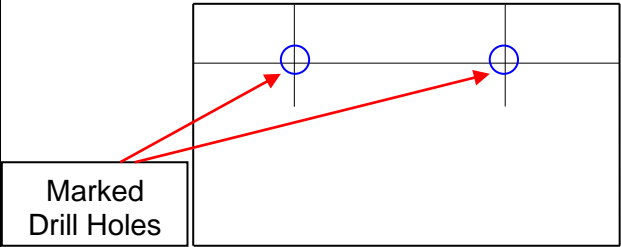
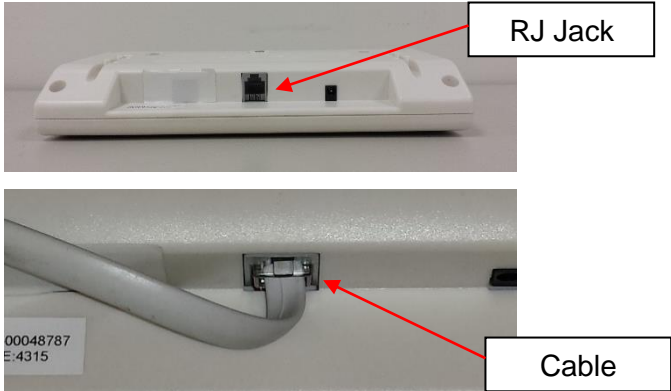
<p>1. Position the display assembly on the wall, and carefully trace around the display.</p>	
<p>2. Mark a level horizontal line, 3/4" (19 mm) below the top of the display outline.</p>	
<p>3. Mark a vertical line 1-3/8" (35 mm) from each side of the display outline.</p>	
<p>4. At the horizontal and vertical intersection, carefully drill the appropriate size holes.</p>	


# DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS (CONT)

<p>5. Once the holes are marked, use a hammer or Phillips head screwdriver to set the 2 drywall anchors into the holes. Use a screwdriver to insert the drywall anchors completely into the wall. Screw the two mounting screws into the anchors. Adjust the screw head depth so the screw heads protrude 1/8" (3.2 mm) from the wall. <b>Warning: Extending the screw heads beyond 1/8" (3.2 mm) can damage the display assembly.</b></p>	
<p>6. Position the display assembly mounting holes onto the mounting screws, and carefully pull down on the display assembly to secure.</p>	
<p>7. Connect the cable to the RJ jack in the bottom of the display assembly. If using the scale on battery power, the scale is now ready for use. If using the scale with the optional AC adapter, see page 10 for set up instructions.</p>	

## POWERING THE SCALE
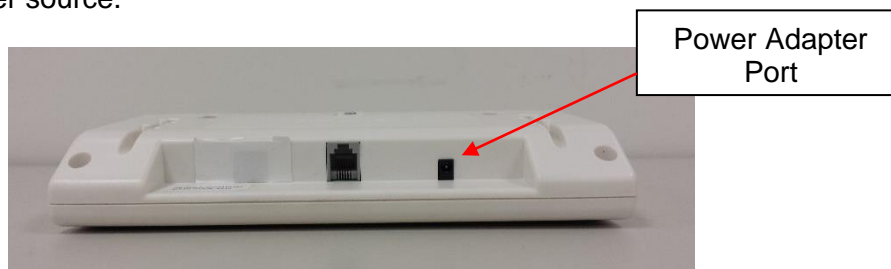
### Powering the Scale


1. If powering the scale via the optional AC adapter, plug the adapter into the display assembly and into a power source.

2. For battery operation, see Battery Installation below.

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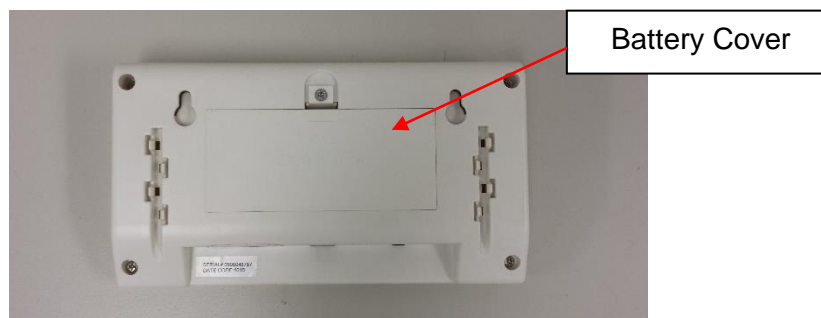### Preparing a 349KL Scale for Transport

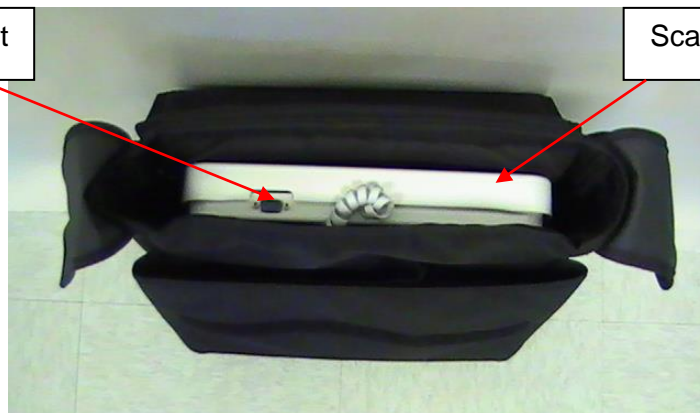
This scale can be easily placed into a carrying case (item # 64771) for storage, or transport from one location to another. To prepare the scale for transport, follow these steps:

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## Weighing a Patient

1. Make sure there is nothing on the weighing platform.
2. Press the **ON** button to power the scale on.
3. Wait until "0.0" LB or "0.0" KG appears on the display.
4. Press the **ZERO** button to zero the scale, if necessary.
5. Have the patient step onto the scale. Wait for the weight reading to stabilize.  
**Note:** Depending on the movement of the patient on the scale, it may take approximately 3-4 seconds for the weight reading to stabilize.  
**Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
6. After recording the weight, have the patient step off the scale.
7. Press the **OFF** button to power the scale off.

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### **Changing the Weight Measuring Unit**

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### **Locking the Weight Measuring Unit**

**Note:** The weight measuring unit can be locked to only measure in KG or LB.

1. Press the **ON** button to power the scale on.
2. Wait for "0.0" LB or "0.0" KG to appear on the display.
3. Press and hold the **lb./kg** button until the unit of measure begins to flash and the selected mode will be locked.
4. Release the **lb./kg** button and the scale will operate in the selected mode.
5. Repeat steps 3-4 to change the weighing mode to a different unit of measure as the default for use.

### **Connection to Welch Allyn® Spot Vital Signs® Lxi Monitor.**

**Warning:** When connecting to another medical device, the power adapter must not be used. Disconnect the power adapter from the scale and use only the battery power. This ensures proper electrical isolation.

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1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Place the object to be tared onto the weighing platform. The display will show a value for the object's weight.
4. With the object remaining on the scale, press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the patient onto the scale with the tared item. The scale will automatically deduct the weight of the object and only display the patient's weight.
6. With all the weight off the scale, "UNDER" will be temporarily displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

### Child/Parent Function

The Parent/Child function on Health o meter® Professional scales provides the convenience of weighing a small child or infant while being held by a parent. Follow these simple steps to display the child's weight.

1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Ask the parent to step onto the scale without the child. The display will show the parent's weight.
4. Have the parent remain standing on the scale and press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the child into the parent's arms. The scale will display the child's weight.
6. When the parent steps off of the scale, "under" will be temporarily displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

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**Caution: Disconnect the scale from the AC power source prior to cleaning the unit.**

1. Clean all external surfaces with a clean damp cloth. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solutions.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

# CALIBRATION

## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified weights. 50 / 100 / 200 / 400 lbs are employed as the calibration standards. The scale can only be calibrated in pounds (lb).

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**Note:** Only weights certified and traceable to national standards should be used for calibration procedures.

## Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the <b>lb./kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "UXX". (Two alphanumeric characters will appear next to "U".) Release all three buttons. Several alphanumeric characters will appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "UXX" then "XXXX"
Press the <b>ZERO</b> button, and "C-50" will flash on the display, followed by several alphanumeric characters.	"C-50" then "XXXX"
Place 50 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-100" then "XXXX"
Place 100 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-200" then "XXXX"
Place 200 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-400" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-400" then "XXXX"
Place 400 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"

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<b>Symptom</b>	<b>Possible Cause</b>	<b>Corrective Action</b>
Scale does not power on	<ol style="list-style-type: none"><li>1. Dead batteries</li><li>2. Faulty electrical outlet</li><li>3. Bad power supply</li></ol>	<ol style="list-style-type: none"><li>1. Replace batteries</li><li>2. Use a different outlet</li><li>3. Replace AC adapter</li></ol>
Questionable weight or the scale does not zero	<ol style="list-style-type: none"><li>1. External object interfering with the scale</li><li>2. The display did not show "0.0" before weighing</li><li>3. Scale is not placed on a level floor</li><li>4. Scale is out of calibration</li></ol>	<ol style="list-style-type: none"><li>1. Remove interfering object from the scale</li><li>2. Ask the patient to get off the scale, zero the scale and begin weighing process again</li><li>3. Place the scale on a level floor and begin weighing process again</li><li>4. Check weight with known weight value</li></ol>
Weighing is performed but the weighing process takes too long and the weight does not lock on the display	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (400 lb / 180 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The batteries are depleted	Replace batteries according to instructions

# WARRANTY

## Limited Warranty

### **What does the Warranty Cover?**

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

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Pelstar, LLC  
 Attention R/A# \_\_\_\_\_  
 Return Department  
 9500 West 55<sup>th</sup> St.  
 McCook, IL 60525

### **Extended Warranty Available**

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit [www.homscales.com/scalesurance/](http://www.homscales.com/scalesurance/) or contact your medical supply distributor.

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**Cautions and  
Warnings  
p. 164**

**Specifications /  
Definition of  
Symbols  
p. 165**

**Certifications /  
Disposal  
p.166**

**Assembly  
Instructions  
p.167**

**Powering the  
Scale / Battery  
Replacement  
p.173**

**Transport  
Instructions  
p.175**

**Operating  
Instructions  
p.177**

**Maintenance  
p.181**

**Calibration  
p.182**

**Troubleshooting  
p.183**

**Warranty  
p.184**

**Español  
p.185**

**Français  
p.207**

## CAUTION AND WARNING

- To prevent injury and damage to your scale, please follow these instructions very carefully.
- Assemble and operate the scale per the enclosed user instructions.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.
- Do not transport the scale with any weight on the platform.
- Do not exceed recommended weight limit of 400 lb / 180 kg for this scale.
- Do not use in the presence of flammable or explosive materials.
- For accurate weighing, if the “Lo” indicator activates, replace the battery or switch to the AC adapter as soon as possible.
- If the scale becomes damaged, it should not be operated until properly serviced. All repairs should only be performed by authorized Health o meter® Professional service personnel.
- When replacing / inserting the battery, be sure to use a new 9V battery.
- To prevent patient injury, the patient must be attended to throughout the entire weighing event.

### **When operating this scale with the optional AC adapter:**

- Operate this scale exclusively with the AC adapter provided by Health o meter® Professional. Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- For use with AC adapter model ADPT40.
- Prior to using this scale, inspect the AC adapter cord for cracking or fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.
- Unplug the AC adapter, and carefully store both the AC adapter and adapter cord before moving the scale.
- If the scale is being connected to another medical device, the power adapter must not be used. Disconnect the power adapter from the scale and use only the battery power. This ensures proper electrical isolation.

## SPECIFICATIONS FOR THIS SCALE

### General

The Model 349KLX remote display digital scale uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

This scale is set up to use weighing technology in order to determine the actual weight of a moving patient.

Weight can be displayed in pounds (lb) or in kilograms (kg). For privacy and convenience in use, the scale features a remote display head which may be held by the operator, placed on a table or other flat surface, or mounted on a wall. **This scale can be powered using an AC adapter, model ADPT40 (not included), or by 9V battery (included).**

### 349KLX Scale Specifications

Capacity and Resolution	400 lb x 0.2 lb (180 kg x 0.1 kg)	
Power Requirements	120 VAC, 60Hz - 9V DC adapter (model ADPT40) or 9V battery	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 32°F to 125°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	<b>Platform</b> Length: 12" (30.5 cm) Width: 12.5" (30.5 cm) Height: 1.875" (4.8 cm) Weight: 8.8 lb (4.0 kg)	<b>Display assembly</b> Length: 6.3" (16.0 cm) Width: 1.3" (3.4 cm) Height: 3.25" (8.3 cm) Weight: 0.3 lb (0.14 kg)

## CERTIFICATIONS / DISPOSAL

### Certification Descriptions

	<p>Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
---	---

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## **Parts List**

### **Carton**

- (1) Scale platform
- (1) Display assembly
- (1) Display bracket
- (1) 9V battery
- (1) User instructions
- (2) #8 Anchoring screws (4.2 mm nominal OD)
- (2) Mounting anchors, compatible with wall construction

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
## 349KLX ASSEMBLY INSTRUCTIONS (CONTINUED)

### **Assembling the 349KLX Scale (Refer to Figures 1- 4 on pages 8-11)**


1. Carefully remove the scale platform and display assembly from the carton, and place the scale platform and display assembly on a flat, level, and dry surface (see Figure 3 on page 9).
2. Connect the cable to the display assembly RJ jack (see Figures 2 & 3 on pages 8 & 9).
3. Remove the display bracket from the carton (see Figure 1 on page 8).
4. Remove the two small screws from plate B (see Figure 1 on page 8).
5. While holding plate A, slide the tab of plate B up and under the center tab on the back of plate A (see Figure 1 on page 8).
6. Align plate A and plate B, and ensure that the screw holes are lined up (see Figure 1 on page 8).
7. Insert the two small screws into the front of plate A and through the holes in plate B (see Figure 1 on page 8).
8. Using a Phillips head screwdriver, tighten the screws to secure plate A to plate B (see Figure 1 on page 8).
9. Slide the two mounting studs on the display bracket into the two mounting holes in the back of the display assembly (see Figures 1 & 4 on pages 8 & 11). The display bracket is now ready for use.

# 349KLX ASSEMBLY INSTRUCTIONS (CONTINUED)

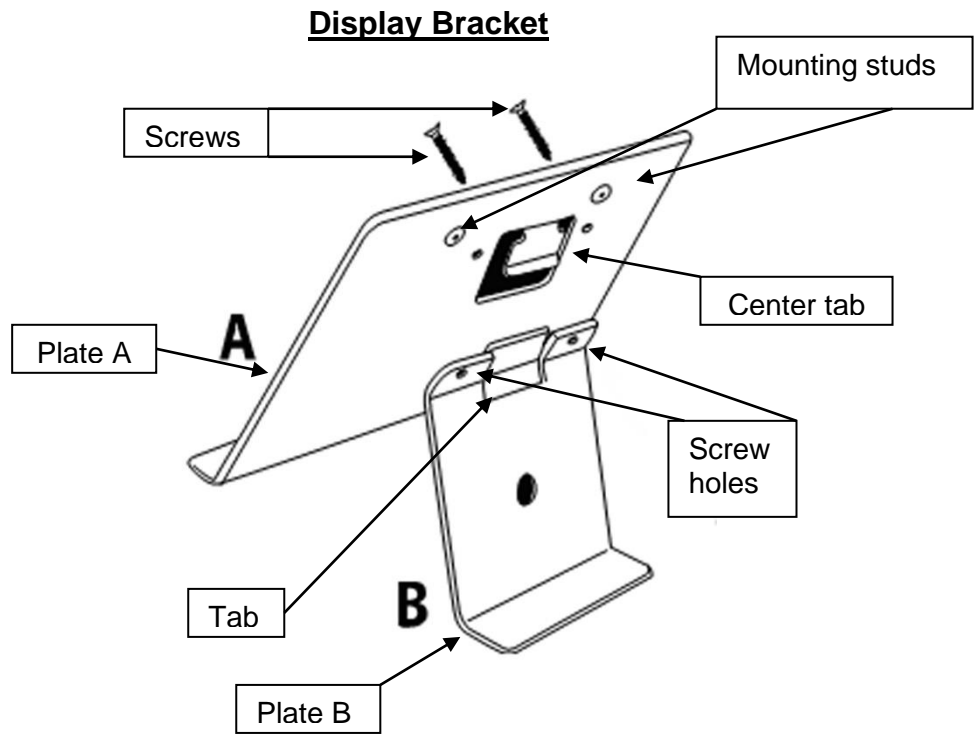
**Figure 1**

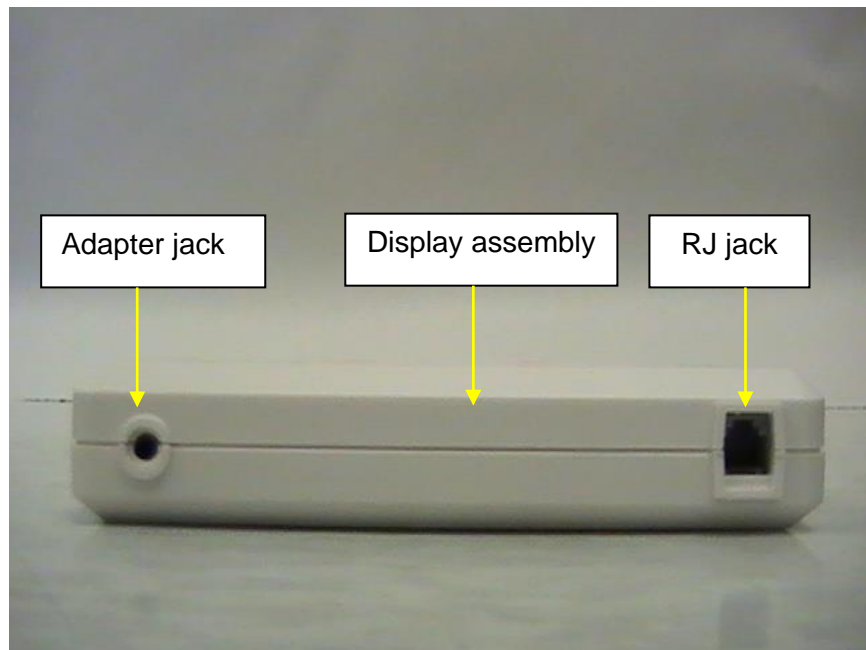**Figure 2**

# 349KLX ASSEMBLY INSTRUCTIONS (CONTINUED)

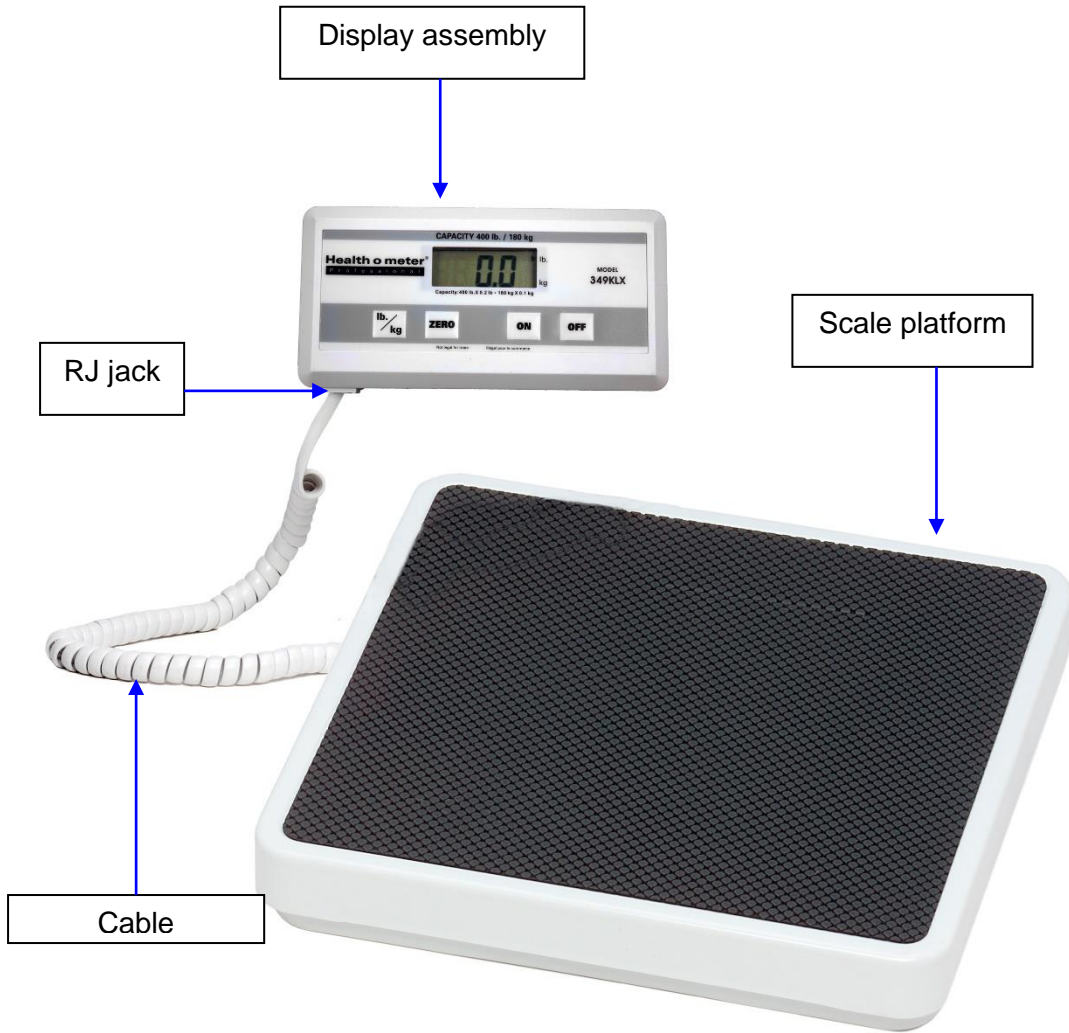**Figure 3**

## DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS

### **Display Assembly Wall Mounting Instructions (Refer to Figure 4 on page 11)**

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1. Position the display assembly on the wall, and carefully trace around the display outline.
2. Mark a level horizontal line,  $\frac{3}{4}$ " (19 mm) below the top of the display outline.
3. Mark a vertical line  $2\text{-}\frac{1}{4}$ " (57.2 mm) from each side of the display outline.
4. Use the appropriate plastic or metal anchors for the type of wall construction.
5. At the horizontal and vertical intersection, carefully drill the appropriate size hole.
6. Insert the anchors into the wall, and screw the two mounting screws into the anchors. Adjust the screw head depth such that the screw heads protrude  $\frac{1}{8}$ " (3.2 mm) from the wall.


**Warning:** Extending the screw heads beyond  $\frac{1}{8}$ " (3.2 mm) can damage the display assembly.

7. Position the display assembly mounting holes onto the mounting screws, and carefully pull down on the display assembly to secure.

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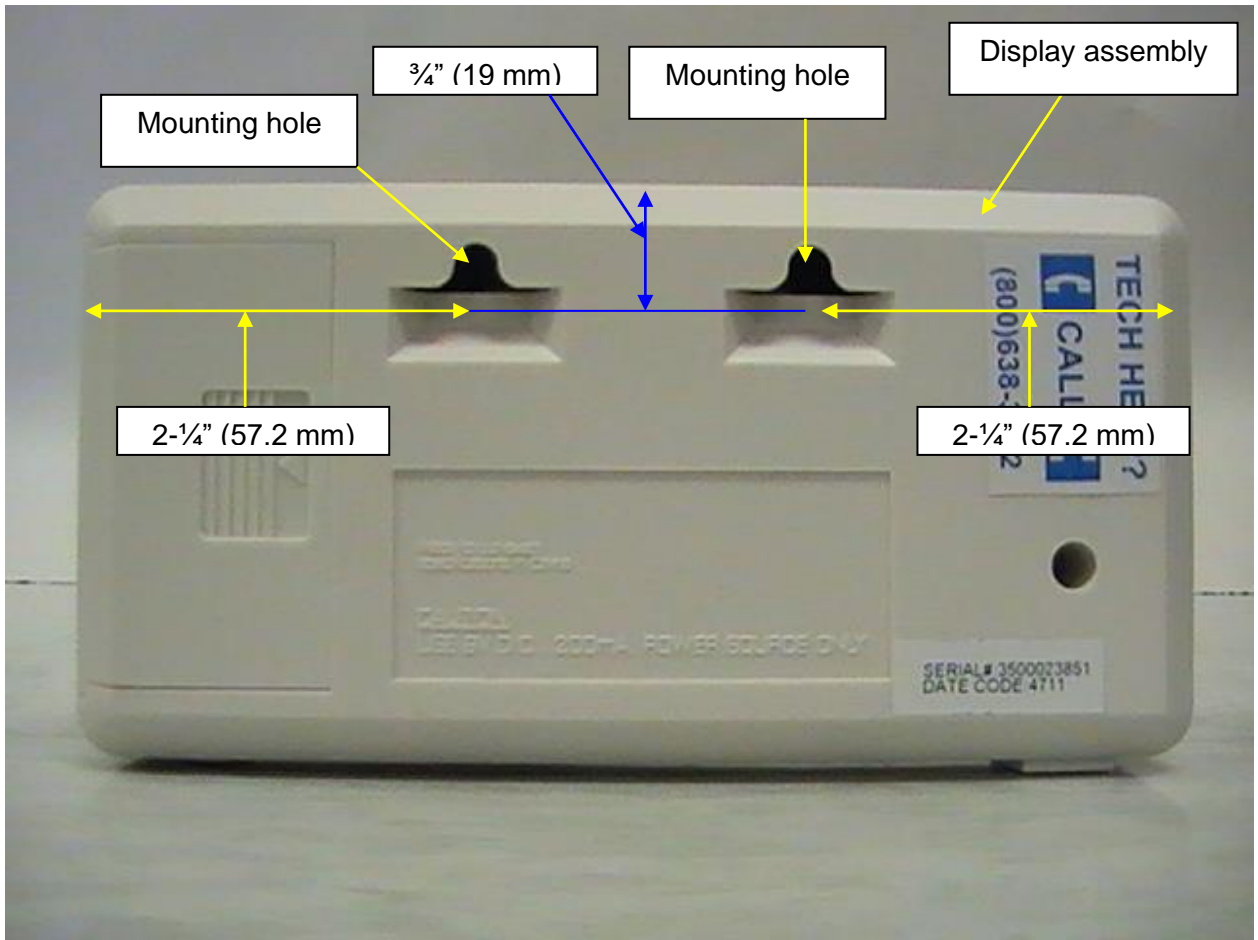### Preparing the 349KLX Scale for Use

#### Replacing / Inserting Battery

1. Unplug the AC adapter from the power source.
2. Remove the battery cover on the side of the display assembly by sliding the battery cover out (see Figure 5 on page 13).
3. Replace or install the battery (Figure 6 on page 13).

**Note:** Health o meter® Professional recommends using an alkaline or lithium battery, rated for 130°F (54°C) operation. Do not use zinc-carbon battery.

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**Warning:** If the scale will not be used for some time, remove the battery to avoid a safety hazard. Disposal of battery must be performed in accordance with the prevailing national, regional or local regulations which apply to you.

#### Optional AC Adapter


1. Remove the protective plastic film covering the display (the film says “FACTORY CALIBRATED”).
2. Place battery into the battery holder (see Figure 5 on page 13).
3. Connect the optional AC adapter into adapter jack on the display assembly (see Figure 2 on page 8).

**Warning:** To avoid a safety hazard, use AC adapter model ADPT40.


4. Plug the scale’s AC adapter into the power source.
5. Ensure all objects are removed from the scale’s assembly, and then press the **ON** button to power the scale on. The display will show “349”, then “0.0” lb. Press the **lb. /kg** button to select the weight mode (lb or kg).
6. When the AC adapter is disconnected from the scale, the scale will switch to battery power if battery is installed.

**Note:** If the set up procedure failed, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

# SETUP / BATTERY (CONTINUED)

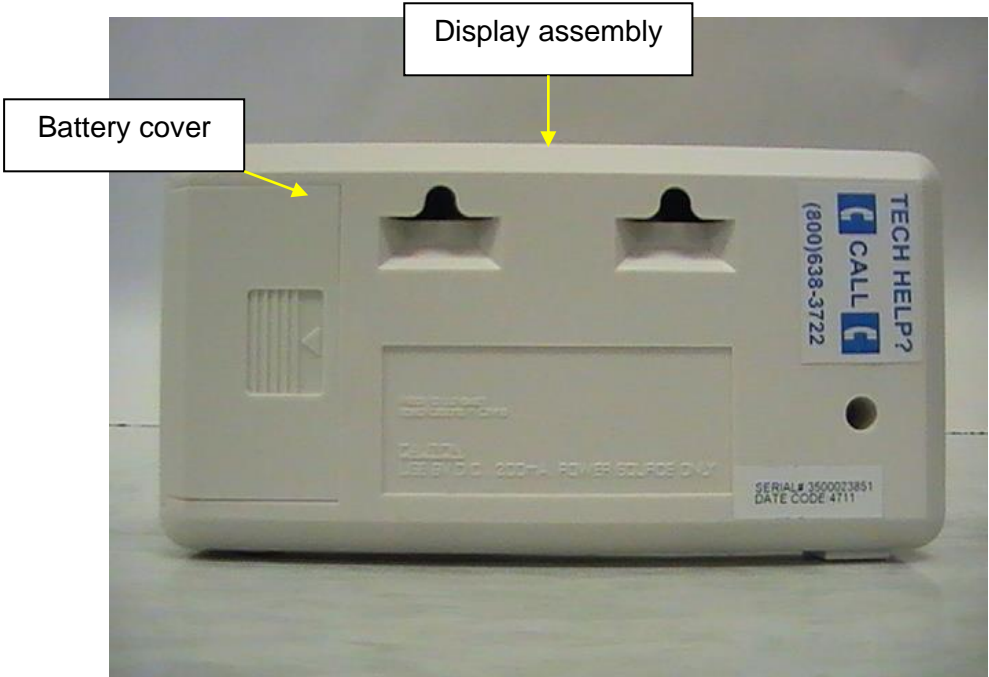**Figure 5**

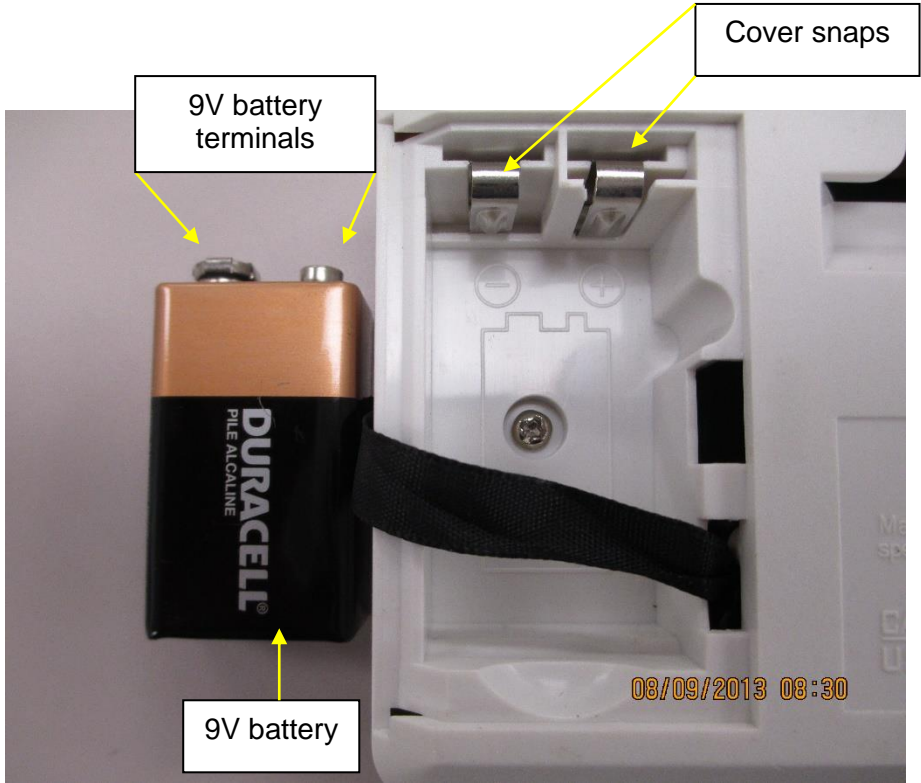**Figure 6**

# TRANSPORT

## Preparing a 349KL Scale for Transport

This scale can be easily placed into carrying case (item # 64771) for storage, or transport from one room to another. This scale does not include the carrying case item # 64771, and must be acquired separately (see Figure 7 below). To prepare the scale for transport, follow these steps:

1. Remove the display assembly and display bracket from its current position.
2. Unplug the AC adapter and cable from the display assembly, and remove the AC adapter from the outlet.
3. Carefully store the AC adapter, display bracket, and display assembly in carrying case to prevent damage during transport (see Figure 8 on page 15).
4. With the scale platform on the floor, lift scale platform carefully and place scale platform into the carrying case. Position scale platform such that the RS-232 serial port and cable are facing up (see Figure 8 on page 15).
5. Close carrying case flap over the scale (see Figure 7 below).
6. Grasp the carrying case strap and lift carefully into the upright position. Position the strap over a shoulder (see Figure 9 on page 15). The scale can now be transported.

## Required Parts


- (1) Display assembly
- (1) Scale platform
- (1) Display bracket
- (1) Carrying case (item # 64771)

**Note:** Carrying case item # 64771 is not included.

**Figure 7**

# TRANSPORT (CONTINUED)

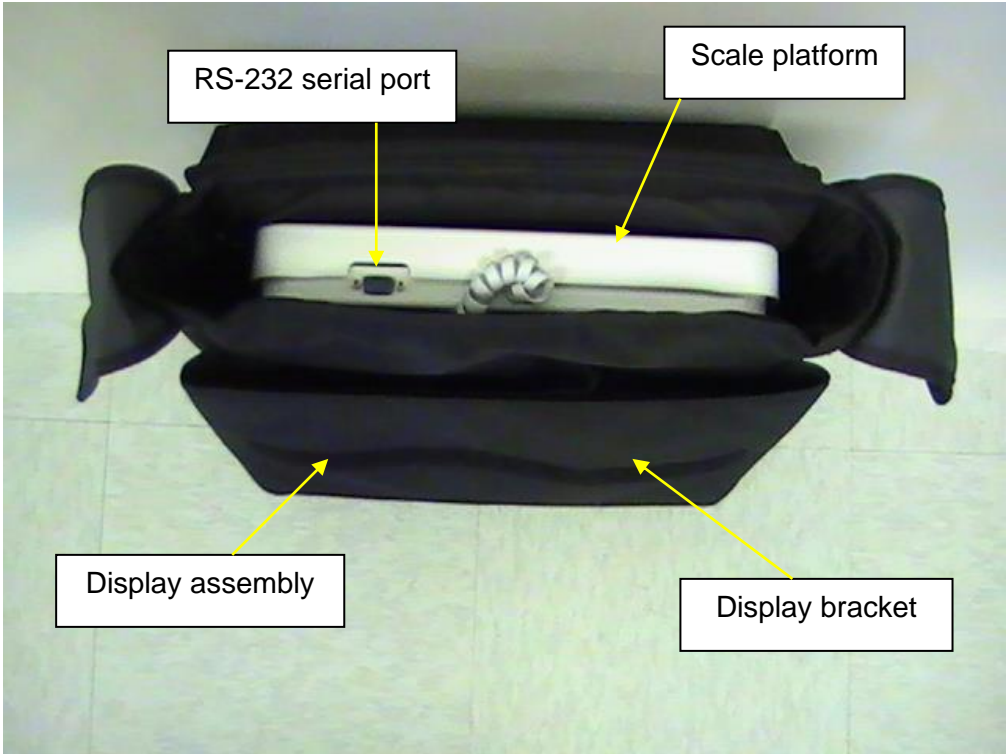**Figure 8**

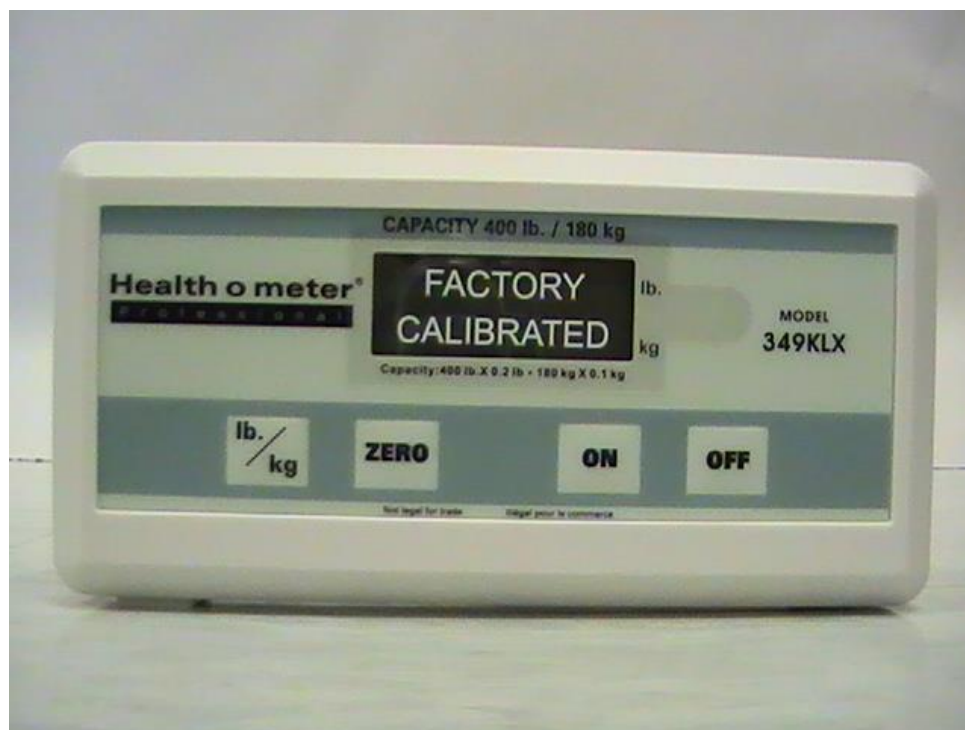
**Figure 9**

## QUICK START / OPERATING INSTRUCTIONS

### Keypad

**Figure 10**

Function	Description
<b>ON</b>	Powers scale on.
<b>OFF</b>	Powers scale off.
<b>ZERO</b>	Tares the scale prior to weighing (0.0).
<b>lb. / kg</b>	Toggles between pounds (lb) and kilograms (kg).

## QUICK START / OPERATING INSTRUCTIONS (CONT)

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**Note:** Ensure cable is attached to the display assembly RJ jack, and make sure there is no object on the scale platform.

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### Units of weight:

1. Press the **ON** button to power the scale on.
2. Wait for "0.0" lb to appear on the display.
3. Press and release the **lb./kg** button to operate the scale in the selected mode. The selected units of measure will appear on the right side of display.
4. Repeat step 3 to toggle the weighing mode to a different unit of measure.

### KG and LB Lock Out

**Note:** The default weighing mode for this scale can be changed from LB/KG to KG only or LB only.

## QUICK START / OPERATING INSTRUCTIONS (CONT)

### To Change Weighing Mode:

1. Press the **ON** button to power the scale on.
2. Wait for “0.0” LB or “0.0” KG to appear on the display.
3. Press and hold the **lb./kg** button until the unit of measure begins to flash and the selected mode will be locked.
4. Release the **lb./kg** button and the scale will operate in the selected mode.
5. Repeat steps 3-4 to change the weighing mode to a different unit of measure as the default for use.

### Weighing a Patient

**Note:** When this scale is powered on, it will always default to pounds (lb) or kilograms (kg).

1. Make sure there is no object on the scale platform.
2. Press the **ON** button to power the display assembly on.
3. Wait until the “0.0” lb appears on the display.
4. Press the **ZERO** button to zero the scale, if necessary.
5. Have the patient step onto the scale. Wait for the weight reading to stabilize.

**Note:** Depending on the movement of the patient on the scale, it may take approximately 3-4 seconds for the weight reading to stabilize.

**Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**


6. After the weight measurement has been taken and recorded, have the patient step off the scale.
7. Press the **OFF** button to power the scale off.

### Connection to Another Medical Device (Optional)

**Warning:** If the scale is being connected to another medical device, the power adapter must not be used. Disconnect the power adapter from the scale and use only the battery power. This ensures proper electrical isolation.

The 349KLX scale is able to connect to other medical devices, such as the Welch Allyn® Spot Vital Signs® Lxi Monitor. To connect to another medical device, use the RS-232 serial port, located on the scale platform.

## QUICK START / OPERATING INSTRUCTIONS (CONT)

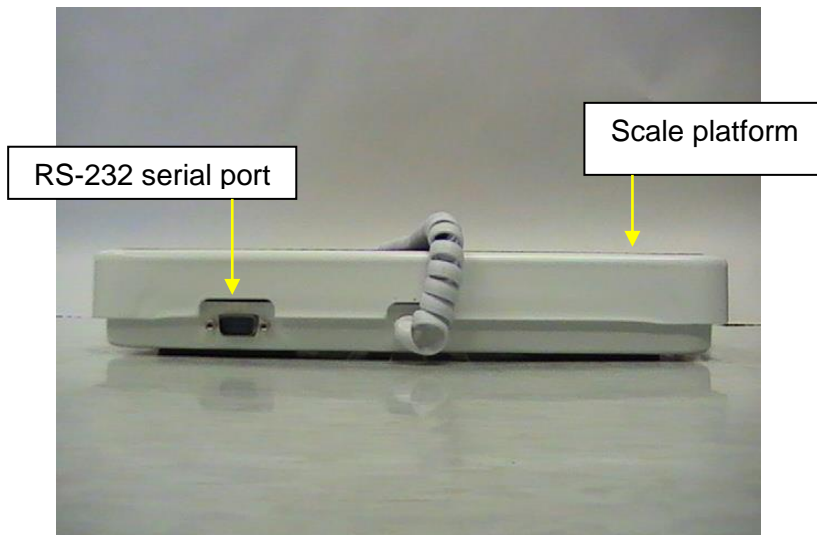**Figure 11**

### **Zero Function**

When using this scale, the weight of an object, can be zeroed out allowing additional weight to be added and displayed. The zero function automatically performs this subtraction.

**Note:** Ensure cable is attached to the display assembly RJ jack, and make sure there is no object on the scale platform.

### **Zero (Push Button)**

1. With all weight off the scale, press the **ON** button to power on the scale.
2. Wait until the "0.0" lb or appears on the display.
3. Place the object on the scale. The display will show a value for the object's weight.
4. Press the **ZERO** button. The word "TARE" will appear on the display assembly, followed by "0.0" lb.
5. Place additional item (object) on the platform and only the weight of the additional item will be displayed.
6. When you remove the item from the platform the displayed will return to "0.0" lb

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1. Check overall appearance of the total scale for any obvious damage, wear, and tear.
2. Inspect the AC adapter for cord cracking or fraying, or for broken/bent prongs.

**Note:** This scale is an extremely sensitive weighing device. If the auto-off function is disabled, battery power will be consumed.

## Cleaning

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

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1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solutions.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

# CALIBRATION

## Calibration

**This scale has been factory calibrated, and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified calibration weights. 50 / 100 / 200 / 400 lbs are employed as the calibration standards.

## Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the <b>lb/kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "U13". Release all three buttons. Several alphanumeric characters shall appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "U 13" then "XXXX"
Press the <b>ZERO</b> button, and "C-50" will flash on the display, followed by several alphanumeric characters.	"C-50" then "XXXX"
Place 50 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-100" then "XXXX"
Place 100 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-200" then "XXXX"
Place 200 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-400" will appear on the display, followed by several alphanumeric characters.	"XXXX" followed by "C-400" then "XXXX"
Place 400 lbs on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"

# TROUBLESHOOTING

## Troubleshooting

Before contacting service personnel, refer to the following instructions to check and to correct any failures.

<b>Symptom</b>	<b>Possible Cause</b>	<b>Corrective Action</b>
Scale does not power on	<ol style="list-style-type: none"><li>1. Dead battery</li><li>2. Faulty electrical outlet</li><li>3. Bad power supply</li></ol>	<ol style="list-style-type: none"><li>1. Replace battery</li><li>2. Use a different outlet</li><li>3. Replace AC adapter</li></ol>
Questionable weight or the scale does not zero	<ol style="list-style-type: none"><li>1. External object interfering with the scale</li><li>2. The display did not show "0.0" before weighing</li><li>3. Scale is not placed on a level floor</li><li>4. Scale is out of calibration</li></ol>	<ol style="list-style-type: none"><li>1. Remove interfering object from the scale</li><li>2. Ask the patient to get off the scale, zero the scale and begin weighing process again</li><li>3. Place the scale on a level floor and begin weighing process again</li><li>4. Check weight with known weight value</li></ol>
Weighing is performed but the weighing process takes too long and the weight does not lock on the displayed	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (400 lb / 180 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The battery is discharged	Replace battery according to instructions

# WARRANTY

## Limited Warranty

### What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

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9500 West 55<sup>th</sup> St.

McCook, IL 60525

If your scale is not covered by warranty, or has been damaged, an estimate of replacement costs will be provided to you for approval, prior to replacing.

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Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

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© Pelstar®, LLC 2023
**PELSTAR, LLC**
Health o meter® est une marque de commerce enregistré de Sunbeam Products, Inc. Utilisée sous licence.
Les produits Health o meter® Professional sont manufacturés, conçus, et possédés par Pelstar, LLC.