BABYHR Mounting Instructions for 522KL/522KG

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA
© Pelstar, LLC 2018

www.homscales.com

weigh**easier**®

Health o meter®

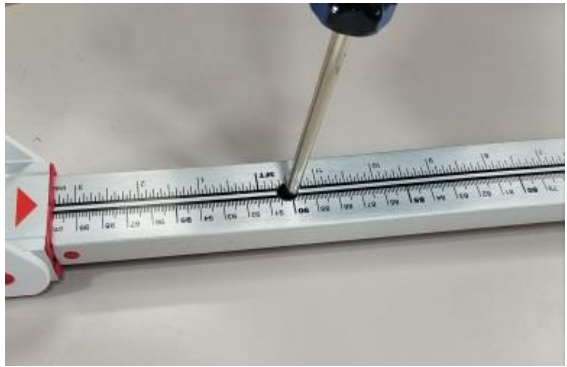
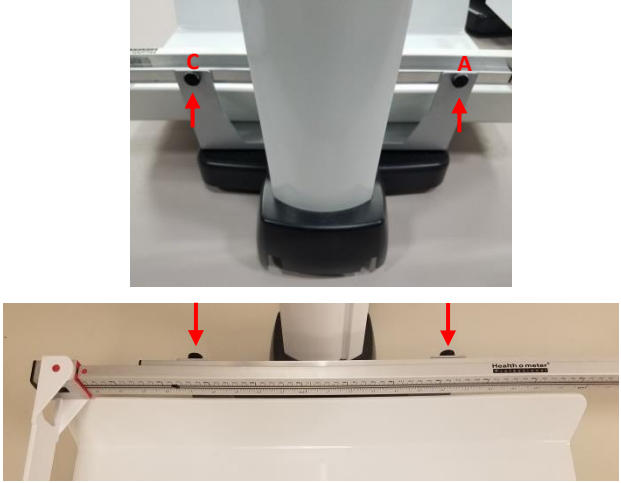
P r o f e s s i o n a l

Thank you for purchasing this Health o meter® Professional product. Please read these instructions carefully and keep it for easy reference or training.

Notes:

- If mounting the height rod onto a scale with the weighing tray already attached start at step 1.
- If mounting the height rod on a brand new scale, do not attach the weighing tray during scale assembly. Begin height rod mounting at step 2 below.
- Ensure that the scale is off during the height rod mounting process. If the scale was powered on during the process, be sure to turn the scale off and power on again before weighing the first patient.
- If the height rod needs to be removed, ensure the scale is off before removal and off before replacing the height rod onto the scale.

<p>1. Remove the screw cover caps on the weighing tray. Next, using a screwdriver, remove the two screws to detach the weighing tray from the scale platform. Retain the screws and caps for use in step 3.</p>	
<p>2. Obtain the mounting bracket and align the holes in the bracket with the holes in the scale platform, as shown.</p>	
<p>3. Obtain the weighing tray and align the holes in the tray with the holes in the platform and mounting bracket. Obtain the two weighing tray screws included with your scale or that were removed in step 1. Insert the screws through the weighing tray and mounting bracket and into the scale platform. Using a screwdriver, attach the weighing tray to the scale platform. Obtain the screw cover caps included with your scale or removed in step 1. Replace the caps over the screws on the tray.</p>	
<p>4. Obtain the silver support rod from the packaging. Lay the support rod on a flat surface with the side with four holes facing up. Place the end with the most holes on the left side.</p>	

<p>5. Flip the support rod over so the side with the four holes is facing down. Ensure that the end of the support rod with the most holes remains on the left side.</p>	
<p>6. Place the BABYHR onto the support rod aligning the two holes at each end. Ensure the end of the height rod with the measuring starting point is on the left side. Obtain the two flat head screws from the packaging and insert them through the height rod into the support rod. Tighten completely using a Phillips head screwdriver. A magnetic screwdriver is recommended.</p> <p>Note: The packaging includes an extra flat head screw. Only two screws are needed.</p>	
<p>7. Align the support rod with height rod onto the mounting bracket on the scale. Ensure that the starting point of the height rod is flush with the side of the scale. Obtain the two thumb screws from the packaging. Insert the thumb screws through holes A and C on the support rod and into the mounting bracket. Completely hand tighten.</p>	
<p>8. Ensure that the stationary paddle of the height rod is flush with the left side of the weighing tray. Mounting is now complete.</p>	

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter[®] Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the product at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 West 55th Street
McCook, IL 60525

PELSTAR, LLC
9500 West 55th St. McCook, IL 60525-7110 USA
1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR PRODUCT FOR WARRANTY COVERAGE AT:
www.homscases.com

Health o meter[®] is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter[®] Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter[®] Professional product features or specifications without notice.

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