Model 549KL / 549KG

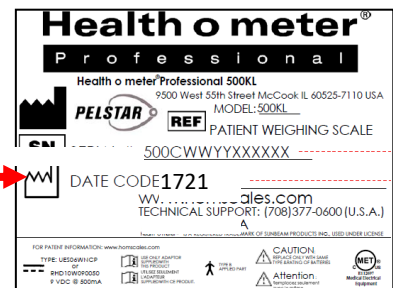
Find the date code on your scale's product label and follow this chart to select the version specific calibration instructions for your scale.

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<u>Date Codes</u>	<u>Link/Page #</u>
4423, all date codes ending in 24	Click here or skip to the next page
For all other date codes not shown above, click here or skip to page 3	

Date code can be found on the product label on the back of the display head.

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CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. A 5kg weight is required for calibration. If the scale is Everlocked into pounds, the calibration procedure will still require use of a 5kg weight.

Note: Avoid scale placement near heating/cooling vents, heating appliances, areas of high airflow or direct sunlight. Calibration must be done in a controlled environment.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Power on the scale. Press and hold **UNIT**. "Opt" will appear on the display, remaining holding the button until "CAL" appears. Release **UNIT** when "CAL" appears.
2. Once the buttons are released the following message will display, "St60", "St59" etc., until the display counts down to zero. This allows the load cells time to settle before calibration starts. The **UNIT** button can be pressed at any time to cancel the countdown, however this is not recommended.
3. Once the time has elapsed, "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ON/TARE/OFF**.
4. The scale will zero and "PLACE 5 ON SCALE" will scroll across the screen.
5. Place a certified 5kg weight on the center of the weighing tray, then press **UNIT**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a five digit number will be displayed for approximately 5 seconds then display the calibration weight. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard (5 kg), "Out OF RANGE" will appear on the display. Calibration is not performed. Remove the calibration weight and press **UNIT** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

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CALIBRATION – 549KL

Note: Only scale models with a date code of 4317 or greater can be calibrated.

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated.

For model 549KL, calibration is performed in either pounds or kilograms. In LB mode, 40 pounds are required for calibration, applied in 10 lb and 20 lb increments. In KG mode, 20 kilograms are required for calibration, applied in 5 kg and 10 kg increments.

Before beginning the calibration procedure ensure the desired calibration unit of measure is selected on the display (LB or KG). The unit of measure does not need to be locked the calibration procedure.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

549KL Calibration Procedure – in LB Mode

Note: To calibrate in pounds, LB must be the selected unit on the display before starting the calibration procedure.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the ON/TARE/OFF button and the kg/lb button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the ON/TARE/OFF button. “C10” will flash on the display.	“C10”
Place 10 pounds on the scale. After stabilization, “C20” will flash on the display.	“C20”
Place an additional 10 pounds on the scale, for a total of 20 lb. After stabilization “C40” will flash on the display.	“C40”
Place an additional 20 lb on the scale, for a total of 40 lb. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

549KL Calibration Procedure – in KG Mode

Note: To calibrate in kilograms, KG must be the selected unit on the display before starting the calibration procedure.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the ON/TARE/OFF button and the kg/lb button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the ON/TARE/OFF button. “C5” will flash on the display.	“C5”
Place 5 kg on the scale. After stabilization, “C10” will flash on the display.	“C10”
Place an additional 5 kg on the scale, for a total of 10 kg. After stabilization “C20” will flash on the display.	“C20”
Place an additional 10 kg on the scale, for a total of 20 kg. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

CALIBRATION – 549KG

Note: Only scale models with a date code of 4317 or greater can be calibrated.

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated.

Calibration is performed using kilograms (KG). 20 kilograms are required for calibration, applied in 5 kg and 10 kg increments.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the ON/TARE/OFF button and the CAL button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the ON/TARE/OFF button. “C5” will flash on the display.	“C5”
Place 5 kg on the scale. After stabilization, “C10” will flash on the display.	“C10”
Place an additional 5 kg on the scale, for a total of 10 kg. After stabilization “C20” will flash on the display.	“C20”
Place an additional 10 kg on the scale, for a total of 20 kg. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

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