## Model 549KL / 549KG

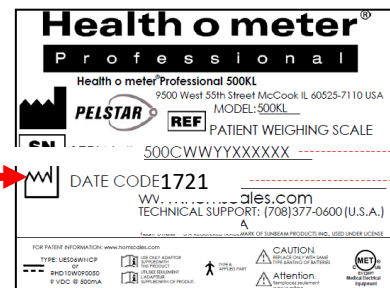
**Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.**

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<u>Date Codes</u>	<u>Link/Page #</u>
4423, all date codes ending in 24	Click <a href="#">here</a> or skip to the next page
For all other date codes not shown above, click <a href="#">here</a> or skip to page 20	

Date code can be found on the product label on the back of the display head.

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**Model 549KL / 549KG**

**CLICK THE BOX BELOW OR**  
**SKIP TO THE PAGE NOTED**

<b>Cautions and Warnings</b> p. 2	<b>Specifications / Definition of Symbols</b> p. 3	<b>Set Up Instructions</b> p.4
<b>Everlock® Permanent Unit Lock Instructions</b> p.5	<b>Operating Instructions</b> p.6	<b>Battery Installation / Maintenance</b> p. 7
<b>Calibration</b> p. 8	<b>Warranty</b> p. 9	<b>Español</b> p.10

## CAUTIONS AND WARNINGS

### **Intended use**

The Health o meter® Professional 549KL and 549KG scales were intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh infants that are laying or sitting on the tray. Do not modify the product or use it for anything other than its intended purpose.

### **To prevent injury and damage to your scale, please follow these instructions very carefully.**

- Assemble and operate the scale per the enclosed user instructions.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not transport the scale with a patient or object on the tray.
- Do not exceed the weight capacity specified for this scale.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- If “Lo” appears on the display, replace the batteries or switch to the AC adapter as soon as possible.

### **Patient/caregiver safety**

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- To prevent falls or injury, place the scale securely far from the edge of the counter or table that is used to hold the scale.

**In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.**

# SPECIFICATIONS

## General

The Health o meter® Professional 549KL / 549KG scale uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

## Scale Specifications

Capacity and Resolution	549KL: 44 lb / 0.5 oz; 20 kg / 10 g 549KG: 20 kg / 10 g	
Power Requirements	9V battery included	
Environmental	Operating temperatures: 50°F to 104°F (10°C to 50°C) Storage temperatures: 32°F to 104°F (0°C to 50°C) Atmospheric Pressure: 500hPa – 1060hPa Humidity: 10% - 95% RH	
Physical Dimensions	<b>Tray Size:</b> Width: 19 1/4" (489 mm) Depth: 13" (330 mm) Height: 2 5/8" (67 mm)	<b>Product Footprint:</b> Width: 19 3/8" (492 mm) Depth: 13 1/2" (343 mm) Height: 4 3/8" (111 mm)

## Certification Descriptions

 	<p>Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
--	---

## Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations that apply to you for disposal of electronic waste or batteries. Do not dispose of this device in the domestic waste stream.

## SET UP INSTRUCTIONS

To enter the Options mode, press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPT” appears, then release the button. When released, “Aft” will appear on the display indicating the scale is in Options mode.

### Auto Off Time

The scale is designed to power off after a period of inactivity. This period of time can be set to a specific time of 1, 2, or 3 minutes. The default setting for Auto Off time is 2 minutes. Follow this procedure to set the number of minutes of inactivity before power off.

1. Press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPT” appears, then release the button. When released, “Aft” will appear on the display. Press the **ON/TARE/OFF** button to change the Auto Off time.  
Aft = 1 Scale will power off after 1 minute of inactivity  
Aft = 2 Scale will power off after 2 minutes of inactivity  
Aft = 3 Scale will power off after 3 minutes of inactivity
2. Choose the setting you prefer then press and hold the **UNIT** button until “Stor” appears on the screen.
3. After “Stor” appears, release the **UNIT** button and the scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0” the scale is ready to use.

### Auto Hold

The Auto Hold function allows the scale to continue displaying the weight measurement after the patient is removed from the weighing tray. The Auto Hold setting can be adjusted to display the weight for 0, 30, or 60 seconds. The default setting is 30 seconds. Follow this procedure to adjust the Auto Hold setting.

1. Press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPT” appears, then release the button. When released, “Aft” will appear on the display. Press **UNIT** again to advance the display screen until “AHd” appears. Press **ON/TARE/OFF** to change the setting.  
AHd = 0 Weight measurement will not be held on the display after the patient is removed  
AHd = 1 Weight measurement will hold on the display for 30 seconds  
AHd = 2 Weight measurement will hold on the display for 60 seconds
2. Choose the setting you prefer then press and hold **UNIT** until “Stor” appears on the display.
3. After “Stor” appears, release **UNIT** and the scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0” the scale is ready to use.

### Locking Unit of Measure (LB or KG) (KL versions only)

The weight measuring unit (pounds / LB or kilograms / KG) can be locked to display weight only in the selected unit of measure. The unit of measure lock default setting is “unlocked”. Follow this procedure to lock the **UNIT** button. **Note:** If Everlock® is engaged, this feature will not appear in the Options mode.

1. Press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPT” appears, then release the button. When released, “Aft” will appear on the display. Press **UNIT** again to advance the display screen until “Unt” appears. Press **ON/TARE/OFF** to change the setting.  
Unt = U Allows the user to toggle weight measurement between LB and KG.  
Unt = L Locks the unit button so weight is only displayed in pounds (LB)  
Unt = † Locks the unit button so weight is only displayed in kilograms (KG)
2. Choose the setting you prefer then press and hold **UNIT** until “Stor” appears on the screen.
3. After “Stor” appears, release **UNIT** and the scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0” the scale is ready to use.

## SET UP INSTRUCTIONS

### **Enabling Unit View with Unit of Measure Locked**

If the scale is locked per the instructions above, users can still view the alternate weight measuring unit by enabling the “Unit View” option. Enabling the “Unit View” option allows the user to press the **UNIT** button to briefly display the weight in the alternate weight measuring unit. After releasing the **UNIT** button the display will return to the locked weight measuring unit. The “Unit View” default setting is “Disable”. Follow this procedure to enable “Unit View” when the weight measuring unit is locked. **Note:** If Everlock® is enabled, this feature will not appear in the Options mode.

1. Press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPt” appears, then release the button. When released, “Aft” will appear on the display. Press **UNIT** again to advance the display screen until “UdP” appears. Press **ON/TARE/OFF** to change the setting.
  - UdP = d Disables the **UNIT** button so it will not display the other weight measuring unit.
  - UdP = E Allows the user to view the other weight measuring unit by holding down the **UNIT** button even if the weight measuring unit is locked. Enabling this function will only allow the user to view the other unit while pressing the **UNIT** button; it will not unlock the weight measuring unit.
2. Choose the setting you prefer then press and hold **UNIT** until “Stor” appears on the screen.
3. After “Stor” appears, release **UNIT** and the scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0” the scale is ready to use.

### **Everlock® (KL versions only)**

The Everlock® feature allows the measuring unit to be permanently locked on the scale, disabling the **UNIT** button. The Everlock® default setting is “disabled”. Follow this procedure to engage Everlock®.

**Warning: This lock is permanent and cannot be reversed.**

#### **Notes:**

- Press and hold **ON/TARE/OFF** at any time to exit the Everlock® procedure.
  - Once Everlock® is engaged the Unt and UdP options will no longer be displayed when scrolling through the Options menu.
  - There is no method to undo Everlock® without sending the scale back to the manufacturer.
1. Press **ON/TARE/OFF** to power on the scale. When the display shows “0” press and hold **UNIT** until “OPt” appears, then release the button. When released, “Aft” will appear on the display. Press **UNIT** again to advance the display screen until “ELC” appears. Press **ON/TARE/OFF** to change the setting.
    - ELC =d Default setting with Everlock® disabled.
    - ELC = E Engages Everlock®. Once this option is selected, the display will prompt you to begin the locking procedure. If this is desired, follow the instructions below.
  2. After Everlock® is selected, “PASS1” will appear on the display. This prompts the user to enter a password. Press the following keys in this sequence:  
**ON/TARE/OFF, UNIT, ON/TARE/OFF**
  3. If entered correctly, “ELC=L” (if scale was in LB mode) or “ELC=6” (if scale was in KG mode) will appear on the display.
  4. Use the **UNIT** button to toggle between (L) Pounds and (6) Kilograms. When the desired unit is displayed, press **ON/TARE/OFF** and “PASS2” will appear on the display.
  5. This will prompt the user to enter the second password. Press the following keys in this sequence:  
**UNIT, ON/TARE/OFF, UNIT**
  6. If entered correctly, the scale will display “ELC = E”.
  7. Press and hold **UNIT** until “Stor” appears on the display.
  8. After “Stor” appears, release **UNIT** and the scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0” the scale is ready to use.

**Warning: Do not power off the scale until **UNIT** is pressed to save the Everlock option, the scale has restarted and the display has returned to “0.0”.**

# OPERATING INSTRUCTIONS

## Weighing a Patient

1. Make sure there is nothing on the weighing tray. Press **ON/TARE/OFF** to turn on the scale. The scale is ready for use when “0” appears on the display.
2. Place the baby on the scale so that its weight is evenly distributed. The scale will begin to display weight information and will lock on the patient’s weight when the stable weight of the patient is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight. When a stable weight is calculated, the weight reading will flash on the display then remain steady when locked. The weight reading will remain on the display until the baby is removed from the scale. **Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
3. To see the patient’s weight in another unit of measurement, press and hold **UNIT**. If the scale is in LB mode, it will convert the weight to KG. If the scale is in KG mode, it will convert the weight to LB. Note: This function is not available on KG models or on KL models with Everlock® engaged or with the “Unit View” setting disabled. Note: “Unit View” is only available when the patient is still on the scale, it does not work during Auto Hold.
4. Remove the baby from the scale. The weight will be held on the display for a period of time if the Auto Hold setting is enabled. To clear a held weight press **ON/TARE/OFF**.
5. To turn off the scale, press and hold **ON/TARE/OFF** for approximately three seconds.  
**Note:** If the display shows “Err” you have exceeded the capacity of the scale.

**Note:** If “Lo” appears on the display, it indicates that the battery power is low. It does NOT indicate a problem with the weight measurement.

## Tare Function

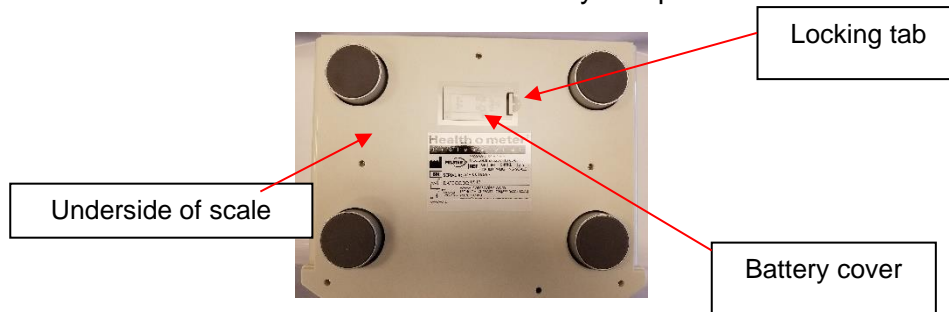
The tare function allows you to place other items on the scale while weighing the baby without affecting the baby’s weight reading.

1. Make sure there is nothing on the weighing tray. Press **ON/TARE/OFF** to turn on the scale. The scale is ready for use when “0” appears on the display.
2. Place the object you want to tare on the scale. The display will show a value for the object’s weight.
3. With the object remaining on the scale, press **ON/TARE/OFF**. “tArE” will appear on the display then the display will return to zero.
4. Keep the object on the scale and place the baby on the scale. The scale will automatically deduct the weight of the object and only display the baby’s weight.
6. Remove the object and the baby from the tray. The weight will be held on the display for a period of time if the Auto Hold setting is enabled. To clear a held weight press **ON/TARE/OFF**.
5. The display will show “Undr”. Press **ON/TARE/OFF** to clear the tared weight and return the scale to zero.

# BATTERY INSTALLATION / MAINTENANCE

## Battery Installation

1. Turn the scale over to access the battery compartment on the underside of the scale. Push down on the tab and lift the cover off of the battery compartment.

2. Replace or install a new 9V battery. Health o meter® Professional recommends using an alkaline or lithium battery, rated for 130°F (54°C) operation. Do not use zinc-carbon batteries. Reattach the battery compartment door.

**Note:** If “Lo” appears on the display, it indicates that the battery power is low. It does NOT indicate a problem with the weight measurement.

**Warning:** If the scale will not be used for some time, remove the battery to avoid a safety hazard. Disposal of batteries must be performed in accordance with the prevailing national, regional or local regulations that apply to you.

## Maintenance

The following pages provide instructions for maintenance, cleaning, and calibrating your scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

Before first use, or after long periods of non-use, check the scale for proper operation and function. Check overall appearance of the total scale for any obvious damage, wear, and tear. If the scale does not operate correctly, refer to qualified service personnel.

**Note:** This scale is an extremely sensitive weighing device. Items such as clothing or towels placed or dropped on the scale may cause the scale to activate. In order to prevent battery depletion, do not store or leave anything on the scale.

## Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

**Caution:** Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
  - mild soap and water solution
  - 70% isopropyl alcohol
  - solution with 1-5% hydrogen peroxide concentrationAfter cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.
2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

## CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. A 5kg weight is required for calibration. If the scale is Everlocked into pounds, the calibration procedure will still require use of a 5kg weight.

**Note: Avoid scale placement near heating/cooling vents, heating appliances, areas of high airflow or direct sunlight. Calibration must be done in a controlled environment.**

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

1. Power on the scale. Press and hold **UNIT**. "Opt" will appear on the display, remaining holding the button until "CAL" appears. Release **UNIT** when "CAL" appears.
2. Once the buttons are released the following message will display, "St60", "St59" etc., until the display counts down to zero. This allows the load cells time to settle before calibration starts. The **UNIT** button can be pressed at any time to cancel the countdown, however this is not recommended.
3. Once the time has elapsed, "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ON/TARE/OFF**.
4. The scale will zero and "PLACE 5 ON SCALE" will scroll across the screen.
5. Place a certified 5kg weight on the center of the weighing tray, then press **UNIT**. "CAL" will flash on the screen while the scale is calibrating.
  - A. If the weight is within calibration range, a five digit number will be displayed for approximately 5 seconds then display the calibration weight. Remove the weight from the scale and the scale will restart.
  - B. If the weight placed on the scale does not match the correct calibration standard (5 kg), "Out OF RANGE" will appear on the display. Calibration is not performed. Remove the calibration weight and press **UNIT** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

# WARRANTY

## **Limited Warranty**

### **What does the Warranty Cover?**

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see “To Get Warranty Service”, below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer’s sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

### **Who is Covered?**

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

### **What is Excluded?**

**Your warranty does not cover normal wear of parts or damage resulting from any of the following:** negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

**To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase.** Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# \_\_\_\_\_

Return Department

9500 West 55<sup>th</sup> Street

McCook, IL 60525

## **Extended Warranty Available**

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility’s existing scale before its current warranty has expired. To learn more, visit [www.homscal.es.com/scalesuranc/e/](http://www.homscal.es.com/scalesuranc/e/) or contact your medical supply distributor.

\*Not available in all countries.

**PELSTAR®**, LLC

**9500 West 55<sup>th</sup> St. McCook, IL 60525-7110 USA**

**1-800-638-3722 or 1-708-377-0600**

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

[www.homscal.es.com](http://www.homscal.es.com)

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter®

Professional products are manufactured, designed, and owned by Pelstar®, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Pelstar, LLC  
Attention R/A# \_\_\_\_\_  
Return Department  
9500 West 55<sup>th</sup> Street  
McCook, IL 60525

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Health o meter® Professional 549KL

Health o meter® Professional 549KG

Thank you for purchasing this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

## TABLE OF CONTENTS

Cautions and Warnings..... 22

Specifications..... 23

Operating Instructions..... 24

Battery Installation / Maintenance ..... 25

Calibration 549KL ..... 26

Calibration 549KG ..... 27

Warranty ..... 28

**Note: This scale has been factory calibrated, and does not require calibration prior to use.**

Please register your scale for warranty coverage at:  
[www.homscales.com](http://www.homscales.com)

For User Instructions updates and revisions please go to:  
[www.homscales.com](http://www.homscales.com)

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**To prevent injury and damage to your scale, please follow these instructions very carefully.**

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## Certification Descriptions

	<p>Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
---	---

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# OPERATING INSTRUCTIONS

## Weighing a Patient

1. Make sure there is nothing on the weighing tray. Press the **ON/TARE/OFF** button to turn on the scale. "88.88" will appear on the display followed by zeroes.
2. Place the baby on the scale so that its weight is evenly distributed. The scale will begin to display weight information and will lock on the patient's weight when the stable weight of the patient is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight. When a stable weight is calculated, the weight reading will flash on the display three times. The weight reading will remain on the display until the baby is removed from the scale. **Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
3. Remove the baby from the scale.
4. To turn off the scale, press and hold the **ON/TARE/OFF** button for approximately three seconds.

**Note:** If the display shows "Err" you have exceeded the capacity of the scale.

**Note:** If "Lo" appears on the display, it indicates that the battery power is low. It does NOT indicate a problem with the weight measurement.

## Tare Function

The tare function allows you to place other items on the scale while weighing the baby without affecting the baby's weight reading.

1. Make sure there is nothing on the weighing tray. Press the **ON/TARE/OFF** button to turn on the scale. "88.88" will appear on the display followed by zeroes.
2. Place the object you want to tare on the scale. The display will show a value for the object's weight.
3. With the object remaining on the scale, press the **ON/TARE/OFF** button. "TARE" will appear on the display then the scale will return to zero.
4. Keep the object on the scale and place the baby on the scale. The scale will automatically deduct the weight of the object and only display the baby's weight.

**Note:** If the scale does not return to zero when you remove the object and baby from the tray, press the **ON/TARE/OFF** button to return the scale to zero.

## LB/KG Lock (549KL ONLY)

The weight measuring unit can be locked to only measure in LB or KG.

### KG Lock Instructions

1. Make sure there is nothing on the weighing tray. Press the **ON/TARE/OFF** button to turn on the scale. "88.88" will appear on the display followed by zeroes.
2. Press the **kg/lb** button so the KG icon is displayed and continue holding the **kg/lb** button for approximately 10 seconds.
3. When the KG icon flashes, release the **kg/lb** button. The KG weight measuring unit is now locked and will remain locked even if the scale is turned off.

### LB Lock Instructions

1. Make sure there is nothing on the weighing tray. Press the **ON/TARE/OFF** button to turn on the scale. "88.88" will appear on the display followed by zeroes.
2. Press the **kg/lb** button so the LB icon is displayed and continue holding the **kg/lb** button for approximately 10 seconds.
3. When the LB icon flashes, release the **kg/lb** button. The LB weight measuring unit is now locked and will remain locked even if the scale is turned off.

### Disabling LB/KG Lock

1. Make sure there is nothing on the weighing tray. Press the **ON/TARE/OFF** button to turn on the scale. "88.88" will appear on the display followed by zeroes.
2. Press and hold the **kg/lb** button until LB and KG icons *both* flash on the display. When both LB and KG icons are flashing, release the **kg/lb** button. The weight measuring unit is now unlocked; pressing the **kg/lb** button will allow user to toggle between LB and KG.

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## CALIBRATION – 549KL

**Note: Only scale models with a date code of 4317 or greater can be calibrated.**

**This scale has been factory calibrated, and does not require calibration prior to use.** If required, the scale can be calibrated.

For model 549KL, calibration is performed in either pounds or kilograms. In LB mode, 40 pounds are required for calibration, applied in 10 lb and 20 lb increments. In KG mode, 20 kilograms are required for calibration, applied in 5 kg and 10 kg increments.

Before beginning the calibration procedure ensure the desired calibration unit of measure is selected on the display (LB or KG). The unit of measure does not need to be locked the calibration procedure.

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### **549KL Calibration Procedure – in LB Mode**

**Note: To calibrate in pounds, LB must be the selected unit on the display before starting the calibration procedure.**

ACTION	DISPLAY SHOWS
With the scale off, press and hold the <b>ON/TARE/OFF</b> button and the <b>kg/lb</b> button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the <b>ON/TARE/OFF</b> button. “C10” will flash on the display.	“C10”
Place 10 pounds on the scale. After stabilization, “C20” will flash on the display.	“C20”
Place an additional 10 pounds on the scale, for a total of 20 lb. After stabilization “C40” will flash on the display.	“C40”
Place an additional 20 lb on the scale, for a total of 40 lb. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

### **549KL Calibration Procedure – in KG Mode**

**Note: To calibrate in kilograms, KG must be the selected unit on the display before starting the calibration procedure.**

ACTION	DISPLAY SHOWS
With the scale off, press and hold the <b>ON/TARE/OFF</b> button and the <b>kg/lb</b> button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the <b>ON/TARE/OFF</b> button. “C5” will flash on the display.	“C5”
Place 5 kg on the scale. After stabilization, “C10” will flash on the display.	“C10”
Place an additional 5 kg on the scale, for a total of 10 kg. After stabilization “C20” will flash on the display.	“C20”
Place an additional 10 kg on the scale, for a total of 20 kg. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

## CALIBRATION – 549KG

**Note: Only scale models with a date code of 4317 or greater can be calibrated.**

**This scale has been factory calibrated, and does not require calibration prior to use.** If required, the scale can be calibrated.

Calibration is performed using kilograms (KG). 20 kilograms are required for calibration, applied in 5 kg and 10 kg increments.

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ACTION	DISPLAY SHOWS
With the scale off, press and hold the <b>ON/TARE/OFF</b> button and the <b>CAL</b> button at the same time. When “CAL” appears on the display, release both buttons. Numbers will appear indicating you can begin calibration.	“CAL” followed by numbers
When the calibration numbers stabilize, press the <b>ON/TARE/OFF</b> button. “C5” will flash on the display.	“C5”
Place 5 kg on the scale. After stabilization, “C10” will flash on the display.	“C10”
Place an additional 5 kg on the scale, for a total of 10 kg. After stabilization “C20” will flash on the display.	“C20”
Place an additional 10 kg on the scale, for a total of 20 kg. After stabilization, “END” will appear on the display and the scale will automatically turn off. The scale has been successfully calibrated.	“END”

# WARRANTY

## **Limited Warranty**

### **What does the Warranty Cover?**

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

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McCook, IL 60525

## **Extended Warranty Available**

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit [www.homscases.com/scalesurance/](http://www.homscases.com/scalesurance/) or contact your medical supply distributor.

\*Not available in all countries.

**PELSTAR® , LLC**

**9500 West 55th St. McCook, IL 60525-7110 USA**

**1-800-638-3722 or 1-708-377-0600**

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:**

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© Pelstar®, LLC 2023
Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar®, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.