BEFORE USING SCALE

1. Remove the Battery Isolation Tab:

Carefully remove the isolating tab located in the battery compartment.

2. Place on Flat Surface:

Immediately after removing the tab, set the scale on a flat, level floor.

3. Ignore Initial Displayed Weight:

If the scale displays approximately 3.7 kg or 8.2 lb, this is expected and can be disregarded. Allow the scale to automatically turn OFF after about 10 seconds.

4. Initialize the Scale:

Lightly tap the scale to power it ON. You will see “8888” scroll across the display, followed by “0.0”. This indicates the scale has initialized and the zero point has been set.

5. Ready for Use:

The scale is ready when “0.0” appears on the display. Always perform this initialization process whenever the scale is moved.

Note: If the display does not activate upon tapping, check the battery connection and refer to the battery replacement instructions below.

6. Select Weight Unit (Optional):

By default, the 880KL displays weight in pounds. To change the unit, use a pen or stylus to slide the unit switch located on the bottom of the scale to your preferred unit.

OPERATING YOUR SCALE

1. Lightly tap the scale to power it ON. Scale is initialized using vibration. Standing on top of the platform will not initialize the scale.
2. Scale is ready when “0.0” appears on the display.
3. Step on scale platform and stand still for approximately 5 seconds to lock in the weight measurement.
4. Weight measurement will flash on screen. Step off the scale.
5. Screen will hold and display the last captured weight for 10 seconds and then scale will power OFF.

WARNING INDICATORS

Lo Replace battery

ERR Weight exceeds maximum capacity of 500lb / 220kg

ADVICE FOR USE AND CARE

- Placing your scale on a flat, level floor will ensure the greatest accuracy and consistency.
- Clean your scale with a damp cloth. Do not use chemical cleaning agents.
- Do not allow your scale to become saturated with water as this can damage the electronics.
- Treat your scale with care – it is a precision instrument. Do not drop it or jump on it. It is not shock proof.

TROUBLESHOOTING GUIDE

If you experience any difficulty in the use of your scale:

- Check to make sure the batteries are correctly fitted.
- Make sure the scale is on a flat, level surface and not touching a wall.
- When you use the scale, if there is no display at all or if “Lo” is displayed – try new batteries.
- For technical assistance, contact Customer Service at 1-800-815-6615.

CHANGING THE BATTERIES

- **IMPORTANT:** Each time the batteries are changed, the unit may display approximately 3.7 kg or 8.2 lb. This is expected and can be disregarded. Please place the scale on a flat, level floor, wait 10 seconds until it turns OFF, then turn it ON again and verify that it displays '0.0' before starting any weight measurement.
- When the symbol "Lo" appears, the batteries need to be replaced.
- Use **ONLY** the recommended CR2032 3.0 V Lithium button type batteries. 2 batteries are required for use.
- Remove battery cover on the underside of the scale by pushing down on the tab and lifting the cover off the unit.
- Make sure that the new batteries are inserted with the positive polarity up! Ensure that the batteries are securely fitted into place. Replace battery cover.
- Remove the batteries if the scale will not be used for extended periods of time.
- Batteries contain substances that may be harmful to the environment and human health.
- **WARNING:** Discard old batteries carefully, out of reach of young children. Swallowing a battery may be fatal. If a battery is swallowed, contact the hospital immediately to have it removed.

This symbol indicates that batteries should not be disposed of in the normal household waste but should be taken to an appropriate local recycling facility when one is available.

Never throw batteries in a fire or attempt to open outer casings, they may explode!

LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alterations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

9500 West 55th St.

McCook, IL 60525

If your scale is not covered by warranty, or has been damaged, an estimate of replacement costs will be provided to you for approval, prior to replacing.

PELSTAR, LLC

9500 West 55th St • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2025