
Model 2842KL

**Large Platform Digital Scale
with Remote Display**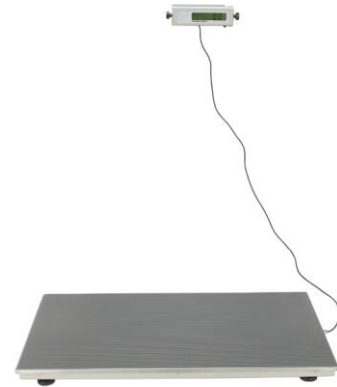

Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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<u>Date Codes</u>	<u>Link/Page #</u>
3625, 4925, all date codes ending in 26	Click here for skip to page 2
For all other date codes not shown above, click here or skip to page 5	

Date code can be found on the product label on the back of the display head.

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CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. 200lb or 80 kg of certified weight is required for calibration.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold **HOLD/RELEASE**, then press and release the  button to power on the scale. Remain pressing **HOLD/RELEASE** until "CAL" appears on the display. Release all buttons.
2. "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ZERO/TARE**.
3. The scale will zero and "PLACE 200 ON SCALE PrESS HOLD" in LB mode, or "PLACE 80 ON SCALE PrESS HOLD" in KG mode. Press **UNIT** to toggle between units.
4. Place 200lb or 80 kg of certified weight on the center of the seat, then press **HOLD/RELEASE**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a four digit number will be displayed. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard, "Out OF RANGE" will appear on the display. Calibration is not performed. Press **ZERO/TARE** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

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CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. The following calibration assumes the scale is set to pounds, and 200 lb is employed as the calibration standard.

Note: Only weights certified and traceable to national standards are to be used for calibration procedures.

Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the REWEIGH and ZERO buttons at the same time, while pressing the ON/OFF button to turn on the scale. "LOAD" will briefly appear on the display, followed by "200.0 LB" indicates you can calibrate. Release all three buttons.	"LOAD" followed by "200.0 LB"
Ensure there is no weight on the scale (except for the rubber mat) and press the REWEIGH button. "Clear" will flash on the display, followed by "-----". At this time the scale is calculating zero calibration. After zero calibration, the display will flash "PUT" followed by "200.0 LB".	"Clear" followed by "-----". Then "PUT" followed by "200.0 LB"
Place 200 lb on the scale. Then press REWEIGH . "CAL" will appear on the display, followed by "FAC" followed by 5 digits. The calibration factor should be in the range between 8000 and 10000. If the number is out of that range, contact Customer Service at 1-800-815-6615.	"CAL" followed by "FAC" followed by "X.XXXX".
If the calibration factor is within range, press the REWEIGH button. "DONE" will appear on the display, followed by "START", followed by "0.0". Remove the weight from the scale.	"DONE" followed by "START", followed by "0.0".
Calibration procedure is complete.	