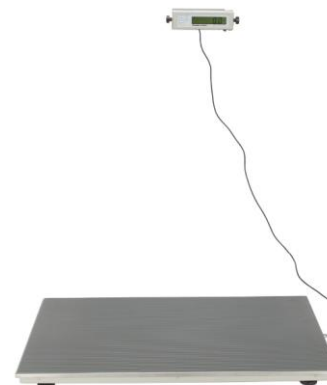
Model 2842KL

**Large Platform Digital Scale
with Remote Display**

Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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<u>Date Codes</u>	<u>Link/Page #</u>
3625, 4925, all date codes ending in 26	Click here for skip to page 2
For all other date codes not shown above, click here or skip to page 55	

Date code can be found on the product label on the back of the display head.

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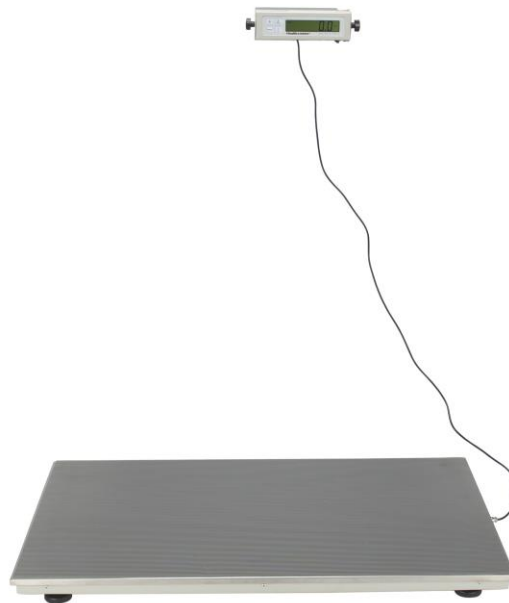
Health o meter®

P r o f e s s i o n a l

Large Platform Digital Scale Model 2842KL

ISO13485:2016
CERTIFIED

ISO 9001:2015
CERTIFIED

USER INSTRUCTIONS

PELSTAR®, LLC 9500 West 55th St. McCook, IL 60525-7110 USA www.homscales.com

Health o meter® Professional 2842KL

© Pelstar®, LLC 2026

weigh**easier**®

Thank you for your purchase of this Health o meter[®] Professional scale. Please read this manual carefully and keep it for easy reference or training.

TABLE OF CONTENTS

CAUTIONS AND WARNINGS.....

3

SPECIFICATIONS

4

CERTIFICATIONS/DISPOSAL.....

5

ASSEMBLY INSTRUCTIONS

6

SET UP

9

OPERATING INSTRUCTIONS.....

13

MAINTENANCE

15

TROUBLESHOOTING

16

CALIBRATION

17

WARRANTY

18

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:
www.homscales.com

For User Instructions updates and revisions please go to:
www.homscales.com

CAUTIONS AND WARNINGS

INTENDED USE

This Health o meter® Professional scale is intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh patients who are safely positioned on the platform. Do not modify the product or use it for anything other than its intended purpose.

To prevent patient/caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not transport the scale with a patient or object on the scale.
- Do not wheel or pull the scale down stairs, doing so may damage internal parts.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.

Patient/caregiver safety

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer.
- To prevent patient injury, the patient must be attended to throughout the entire weighing event. Caregivers should ensure that the patient is stable and provide support as needed when getting onto and off of the scale.

When operating the scale with the AC adapter:

- Operate this scale exclusively with the AC adapter provided by Health o meter Professional. Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- For use with AC adapter model ADPT31 (USA or Canada) or ADPT30 (International)
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

When operating the scale with batteries:

- If the "LOBAT" indicator activates, replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS

Health o meter Professional's Large Platform Digital Scale model 2842KL uses highly sophisticated microprocessor technology. Each scale is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

- Weighing is performed by motion-sensing technology.
- The patient's weight can be displayed in pounds or in kilograms.
- The scale includes a specially designed removable rubber mat for easy cleaning.
- The unit can be operated using its AC power adapter or by 6 C-cell batteries.

Scale Specifications

Capacity and Resolution:	600 lb x 0.2 lb (270 kg x 0.1 kg)
Power Requirements:	Adapter Model No. RHD10W090050 or No. RHD090020 Health o meter® Professional Model (ADPT31) (Included) Input: 100-240VAC, 50/60Hz, 500mA Output: 9.0V $\overline{\text{---}}$ 0.5A Or 6 C Batteries (not included)
Environmental:	Operating Temperature: 50°F to 104°F / 10°C to 40° C Storage Temperature: 32°F to 122°F (0°C to 50°C) Maximum Humidity: 85% RH in use or in storage
Platform Dimensions:	22 1/4" (w) x 42" (d) x 2 1/2" (h) / 565mm (w) x 1067mm (d) x 64mm (h)

Definition of Symbols

Attention: Consult Accompanying Documents

Caution

Do not dispose of this device in the domestic waste stream.

Type B Applied Parts

DC Power Supply

Temperature Limit

Manufacturer

Catalogue Number

Batch Code

Serial Number

CERTIFICATIONS / DISPOSAL

Certification Descriptions:

 	Pelstar, LLC has been officially certified as a manufacturer of Health o meter® Professional medical devices to be in compliance with stringent ISO requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
--	--

Scale Disposal:

Do not dispose of this device in domestic waste. This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal. For scale assistance or telephone support, please call Health o meter® Professional Customer Service (available on normal business days) at (+1) 800-815-6615 or (+1) 708-377-0600.

ASSEMBLY INSTRUCTIONS

Before Assembly

Each 2842KL Large Platform Digital Scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

Parts List

Carton

- (1) Base assembly
- (1) Display head
- (1) Display assembly bracket
- (4) Base assembly feet
- (1) Small white box containing AC adapter
- (2) Display assembly screws and washers
- (1) Rubber mat

ASSEMBLY INSTRUCTIONS (CONT.)

1. Place the base assembly upside down on a flat, steady surface.	
2. Screw the four feet into the base assembly. Turn each foot completely into the base assembly and then back out 1 1/2 turns. Carefully turn the base assembly right side up. Adjust the feet as necessary for a level, stable platform. Do not drop the scale as this may damage the load cells.	
3. Plug the display cable connector into the base cable connector.	
4. Optional step to secure excess cable: Remove the screws on the side of the platform that include black clamps. Open the clamp and route the cable into the clamp. Replace the screw with the cable onto the side of the platform.	
5. Place the rubber mat in the center area of the base assembly platform.	
6. If mounting the display head onto a wall see the Display Head Wall Mounting instructions on the next page. If wall mounting the display, complete this step after the display assembly bracket has been mounted to the wall. If using as on a desk or table, place the display head into the display bracket and screw in the two display assembly screws.	

ASSEMBLY INSTRUCTIONS (CONT.)

Display Assembly Wall Mounting Instructions

Caution: When wall mounting the display assembly, ensure that the mounting location is free of electrical lines, gas pipes, and other safety hazards. Ensure that wall is of construction that can support the display assembly.

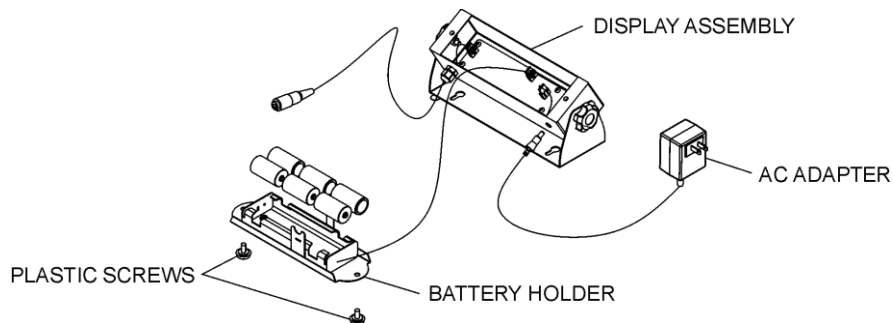
Note: Display assembly can be mounted to a wall up to 6 feet (1.83 m) from the base.

1. If display head is already attached to the display assembly bracket, remove the display assembly screws at the side of the display head and set them aside.
2. Position the display assembly bracket on the wall at the desired location and mark the mounting bracket hole locations.
3. Use suitable plastic or metal anchors for the type of wall construction and carefully drill the appropriate sized holes.
4. Insert the anchors into the wall, and screw the two mounting screws into the anchors. Adjust the screw head depth such that the screw heads protrude 1/8" (3.2 mm) from the wall.
5. Position the display assembly mounting holes onto the mounting screws and pull down on the display assembly to secure the bracket over the screws.
6. Re-attach the display head to the display assembly bracket using the two display assembly screws.

Required Parts Not Included With This Scale

- (2) # 10-12 x 1 in. Screws
- (2) Mounting anchors, compatible with wall construction

Battery Installation

1. If connected, disconnect the scale from the AC power source.
2. Unscrew two plastic screws and carefully remove the battery holder from the display assembly.
3. Take the batteries from the battery holder, replace the batteries with new ones.
4. Position the battery holder into the display assembly and reinstall two plastic screws.

SET UP INSTRUCTIONS

To enter the Options mode, push and hold the **UNIT** button while pressing and releasing the  button.

Auto Off Function

When powering the scale via the AC adapter, the display can be set to power off after a period of inactivity. The Auto Off default setting is “disabled”. Follow this procedure to enable or disable the Auto Off function. **Note:** When powering the scale via batteries, the Auto Off function is automatically enabled.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Press **UNIT** again to change the setting.
AOF = d Disables the Auto Off function
AOF=E Enables the Auto Off function
2. Press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Setting Auto Off Time

When powering the scale via batteries or if Auto Off is Enabled for the AC adapter, the period of inactivity before the scale powers off can be set to a specific amount of time. The default setting for Auto Off time is 2 minutes. Follow this procedure to set the number of minutes of inactivity before power off.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “AFt” setting appears. Press **UNIT** to change the Auto Off time.
AFt = 1 Scale will power off after 1 minute of inactivity
AFt = 2 Scale will power off after 2 minutes of inactivity
AFt = 3 Scale will power off after 3 minutes of inactivity
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Auto Hold

The Auto Hold function allows the scale to continue displaying the weight measurement after the patient is removed from the weighing tray/seat. The Auto Hold setting can be adjusted to display the weight for 1, 30, or 60 seconds. The default setting is 30 seconds. Follow this procedure to adjust the Auto Hold setting.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=F” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “AHd” setting appears. Press **UNIT** to change the setting.
AHd = 0 Weight measurement will hold on the display for 1 second
AHd = 1 Weight measurement will hold on the display for 30 seconds
AHd = 2 Weight measurement will hold on the display for 60 seconds
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS (CONT.)

Setting Sound Options

The display can be set to stay silent or beep while operating the scale. The default setting for Sound is “enabled”. Follow this procedure to enable or disable sound.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “Snd” setting appears. Press **UNIT** to change the setting.
Snd = E Enables sound so the scale beeps when keys are pressed.
Snd = d Disables sound so the scale stays silent when keys are pressed.
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Locking Unit of Measure (LB or KG) (KL versions only)

The weight measuring unit (pounds / LB or kilograms / KG) can be locked to display weight only in the selected unit of measure. The unit of measure lock default setting is “unlocked”. Follow this procedure to lock the **UNIT** button. **Note:** If Everlock® is engaged, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “Unt” setting appears. Press **UNIT** button to change the setting.
Unt = U Allows the user to toggle weight measurement between LB and KG.
Unt = L Locks the unit button so weight is only displayed in pounds (LB)
Unt = | Locks the unit button so weight is only displayed in kilograms (KG)
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Enabling Unit View with Unit of Measure Locked

If the scale is locked per the instructions above, users can still view the alternate weight measuring unit by enabling the “Unit View” option. Enabling the “Unit View” option allows the user to press the **UNIT** button to briefly display the weight in the alternate weight measuring unit. After releasing the **UNIT** button the display will return to the locked weight measuring unit. The “Unit View” default setting is “Disable”. Follow this procedure to enable “Unit View” when the weight measuring unit is locked. **Note:** If Everlock® is enabled, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “UdP” setting appears. Press **UNIT** to change the setting.
UdP = d Disables the **UNIT** button so it will not display the other weight measuring unit.
UdP = E Allows the user to view the other weight measuring unit by holding down the **UNIT** button even if the weight measuring unit is locked. Enabling this function will only allow the user to view the other unit while pressing the **UNIT** button; it will not unlock the weight measuring unit.

SET UP INSTRUCTIONS (CONT.)

2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Setting Auto Zero

This option allows the user to select whether the scale will re-set itself to ZERO each time it is powered up or whether it will use the zero reading from the last time it was zeroed. The auto zero default setting is “enabled”. Follow this procedure to enable or disable this feature

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “PU2” setting appears. Press **UNIT** to change the setting.

PU2 = d Disabling this feature means the scale does not attempt to find a zero when powered on, rather it will use the zero setting it had from its most recent use. Health o meter® Professional recommends that if the Disable option is selected, users should manually zero the scale before each weight measurement by pressing **ZERO / TARE**. Users will see dashes on the screen as the scale is finding zero.

PU2 = E Enabling this feature means the scale will zero itself when it is powered on. After each use, the scale will clear the weight of the patient and set itself back to the initial zero level it found on start-up. Health o meter® Professional recommends keeping this setting as Enable.
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Setting Weight Lock Option

This option allows the user to choose whether the scale locks onto a weight. The default setting is for the scale to lock. **Note:** if patient movement causes a change in weight more than 5 lb / 2.27 kg then the scale will not lock regardless of the setting. Follow this procedure to change the setting.

1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until the “nLC” setting appears. Press **UNIT** to change the setting.

nLC = E Weight measurement will not lock
nLC = d Weight measurement will lock and the LOCK icon will be displayed
2. Choose the setting you prefer and press **HOLD/RELEASE** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS (CONT.)

Everlock® (KL versions only)

The Everlock® feature allows the measuring unit to be permanently locked on the scale, disabling the **UNIT** button. The Everlock® default setting is “disabled”. Follow this procedure to engage Everlock®.

Warning: This lock is permanent and cannot be reversed.

Notes:

- Press **HOLD/RELEASE** at any time to exit the Everlock® procedure.
 - Once Everlock® is engaged the Unt and UdP options will no longer be displayed when scrolling through the Options menu.
 - There is no method to undo Everlock® without sending the scale back to the manufacturer.
1. While the scale is turned off, press and hold **UNIT** then press and release the  button. Hold **UNIT** until “AOF=d” or “AOF=E” appears on the display, then release. Use **ZERO/TARE** to advance or the  button to scroll back the display screen until “ELC=d” appears. This is the default setting with Everlock® disabled. Press **UNIT** to start the Everlock procedure. .
 2. “PASS1” will appear on the display. This prompts the user to enter a password. Press the following keys in this sequence:
, **ZERO/TARE**, **UNIT**
 3. If entered correctly, “ELC=L” (if scale was in LB mode) or “ELC=6” (if scale was in KG mode) will appear on the display.
 4. Use the **ZERO/TARE** button to toggle between (L) Pounds and (6) Kilograms. When the desired unit is displayed, press **HOLD/RELEASE** and “PASS2” will appear on the display.
 5. This will prompt the user to enter the second password. Press the following keys in this sequence:
ZERO/TARE, **UNIT**,  (**Warning: Entering this password will engage Everlock**)
 6. If entered correctly, the scale will display “ELC = L” or “ELC = 6,” depending on the unit of measurement that was selected. Everlock® is now engaged.
 7. Press **HOLD/RELEASE** to save your selection and exit the options mode.
 8. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Warning: Do not power off the scale until **HOLD/RELEASE** is pressed to save the Everlock® option, the scale has restarted and the display has returned to “0.0”.

OPERATING INSTRUCTIONS

Keypad

DESCRIPTION	FUNCTION
/ REWEIGH	Powers scale on and off. Allows repeated weighing of the patient while remaining on the scale.
ZERO/TARE	Zeroes out or tares the scale prior to a weighing.
HOLD/RELEASE	Holds the displayed weight until the button is pressed again to release the value. Will recall the previously locked weight if pressed when nothing is on the scale.
UNIT	Selects kilograms or pounds.

Weighing a Patient

Note: When this scale is turned on, it will always default to the settings and units last used (LB or KG). The scale will not lock on a weight less than 5 lb / 2.27 kg.

1. Make sure there is nothing on the weighing seat. Press to power on the scale. The scale is ready for use when "0" appears on the display.
2. Assist the patient onto the scale. "LOCK" will appear and flash on the display along with a weight measurement. When a stable weight is determined "LOCK" will stay displayed along with the patient's weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight. **Note:** if patient movement causes a change in weight more than 5 lb / 2.27 kg then the scale will not lock

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

3. Assist the patient off of the scale. The weight will be held on the display for a period of time if the Auto Hold setting is enabled. The weight can also be held by pressing **HOLD/RELEASE**. To clear a held weight, press **HOLD/RELEASE**. After the weight measurement has been taken and is recorded, remove the patient from the scale. Or press the / **REWEIGH** button to reweigh the patient without removing them from the scale.
4. To power off the scale, press and hold for approximately three seconds.
Note: If the display shows "- - - - -" you have exceeded the capacity of the scale.

Weight Recall

Note: Weight will only recall if scale has remained powered on.

1. To recall the last weight measured, press **HOLD/RELEASE**.
2. To clear the display of the recalled weight, press **HOLD/RELEASE** again.

OPERATING INSTRUCTIONS

Zero Function

Zeroes the scale prior to a weight measurement. The **ZERO/TARE** button is also used in the Tare function outlined below.

Tare Function

When using this scale, the weight of an object, such as a portable oxygen tank, can be subtracted from the total weight in order to determine the weight of the patient alone.

Weighing a Patient and Taring an Unknown Weight:

The minimum weight that can be tared is 5 lb / 2.27 kg. **Note:** The scale must lock onto a weight before it can be tared.

1. Make sure there is nothing on the weighing tray. Press  to power on the scale. The scale is ready for use when "0" appears on the display.
2. Place the object to be tared on the scale. The display will show a value for the object's weight.
3. Press **ZERO/TARE**. The word "TARE" will appear on the display, and the scale will return to zero.
4. Keep the object on the platform and place the baby on the scale. The scale will automatically deduct the weight of the object and only display the patient's weight.
5. When you remove the patient and the item from the tray, a negative value will show on the display. Press **ZERO/TARE** to clear the tared value.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

Weighing a Patient and Taring a Known Weight (Pre-Tare):

Pre-Tare can be performed for items between .2 lb and 200 lb (0.1 kg and 91 kg).

1. Make sure there is nothing on the weighing tray. Press  to power on the scale. The scale is ready for use when "0" appears on the display.
2. Press and hold **ZERO/TARE** until the word "TARE" appears on the display.
3. The tare icon will flash along with "0.0" LB or "0.0" KG prompting you to enter the weight to be tared. Use the  and **ZERO/TARE** buttons to increase or decrease the weight. Pounds will change in 0.2 lb increments and KG will change in 0.1 kg increments.
4. Once the desired tare value is set, lock the value by pressing **HOLD/RELEASE**. The tare icon will stop flashing and remain on, and the entered amount will show as a negative.
5. Weigh the patient following the weighing procedure outlined on the previous page. The scale will automatically deduct the tare value and only display the patient's weight.
6. To clear the tare value, press the **ZERO/TARE** button.

MAINTENANCE

Maintenance

The following pages provide instructions for maintenance, cleaning, calibration, and troubleshooting for the scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check the overall appearance of the scale for obvious damage, wear, and tear.
2. If applicable, inspect the AC adapter cord for cracking, fraying, and for broken/bent prongs.

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
 - mild soap and water solution
 - 70% isopropyl alcohol
 - solution with 1-5% hydrogen peroxide concentration

After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.

2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel. Should you have further questions, call Health o meter® Professional Scales Customer Service at 1-800-815-6615.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply	1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a stable surface 4. Scale is out of calibration	1. Remove interfering object from the scale 2. Remove patient, zero the scale and begin weighing process again 3. Place the scale on a stable surface and begin weighing process again 4. Check weight with known weight value
The display shows "- - - - -"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its capacity of 600 lb / 270 kg
The display shows "uNdEr"	The scale is in under-load condition	Make sure the rubber mat is placed on the base. Or press ZERO to clear a previously tared weight.
The display shows "LO BAT"	The batteries are low	Replace batteries according to instructions

CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. 200lb or 80 kg of certified weight is required for calibration.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold **HOLD/RELEASE**, then press and release the  button to power on the scale. Remain pressing **HOLD/RELEASE** until "CAL" appears on the display. Release all buttons.
2. "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ZERO/TARE**.
3. The scale will zero and "PLACE 200 ON SCALE PrESS HOLD" in LB mode, or "PLACE 80 ON SCALE PrESS HOLD" in KG mode. Press **UNIT** to toggle between units.
4. Place 200lb or 80 kg of certified weight on the center of the seat, then press **HOLD/RELEASE**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a four digit number will be displayed. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard, "Out OF RANGE" will appear on the display. Calibration is not performed. Press **ZERO/TARE** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the product at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 West 55th Street
McCook, IL 60525

PELSTAR® , LLC

9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY AT:

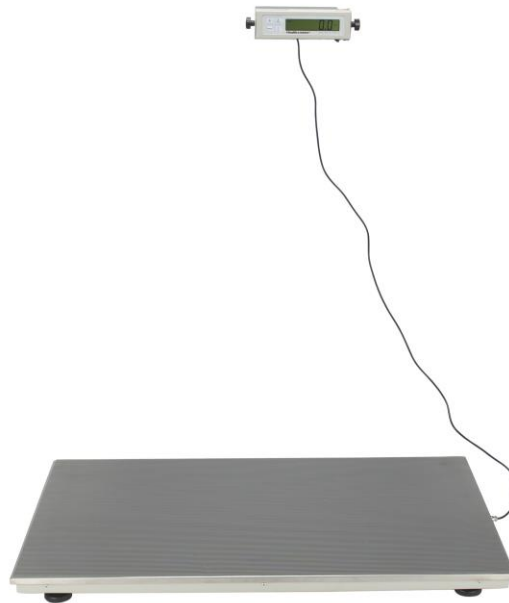
www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar®, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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PELSTAR®, LLC 9500 West 55th St. McCook, Illinois 60525-7110 E.U.A. www.homscales.com
Health o meter® Professional 2842KL

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www.homscales.com

For User Instructions updates and revisions please go to:

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PELSTAR®, LLC 9500 West 55th St. McCook, IL 60525-7110 USA www.homscales.com
Health o meter® Professional 2842KL

© Pelstar®, LLC 2021

weigh**easier**®

Thank you for your purchase of this Health o meter® Professional scale. Please read this manual carefully and keep it for easy reference or training.

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SCALE SPECIFICATIONS

Capacity and Resolution:	600 lb x 0.2 lb (270 kg x 0.1 kg)
Power Requirements:	Adapter Model No. RHD10W090050 or No. RHD090020 Health o meter® Professional Model (ADPT31) (Included) Input: 100-240VAC, 50/60Hz, 500mA Output: 9.0V $\overline{\text{---}}$ 0.5A Or 6 C Batteries (not included)
Environmental:	Operating Temperature: 50°F to 95°F / 10°C to 35° C Storage Temperature: 32°F to 122°F (0°C to 50°C) Maximum Humidity: 85% RH in use or in storage
Platform Dimensions:	22 1/4" (w) x 42" (d) x 2 1/2" (h) / 565mm (w) x 1067mm (d) x 64mm (h)

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(2) # 10-12 x 1 in. Screws

(2) Mounting anchors, compatible with wall construction

SET UP

1. Plug the scale's AC adapter into the display assembly and into the power source. If using batteries instead of the AC adapter, see "Replacing Batteries" section on the next page for battery installation instructions.
2. Press the **ON/OFF** button to turn the scale on. The display will show "start" and then "0.0".
3. When the self-test function is complete, the digital display should read "0.0".
4. Place a weight [not to exceed 600 lb (270 kg)] on the scale. The digital display will blink "-----" until the weight calculation is complete.
5. Press the **LBS/KG** button to select the weight mode, either in kilograms or pounds.
6. Press the **REWEIGH** button; the digital display should again read the weight of the item on the scale.
7. Remove the weight from the scale. The digital display should continue to read the weight of the item that was on the scale.
8. Press the **ZERO** button; the digital display should return to zero.
9. Turn the scale on using **ON/OFF** button.

NOTE: If the set up procedure failed, contact Health o meter Professional Scales Customer Service at 1-800-815-6615.

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OPERATING INSTRUCTIONS

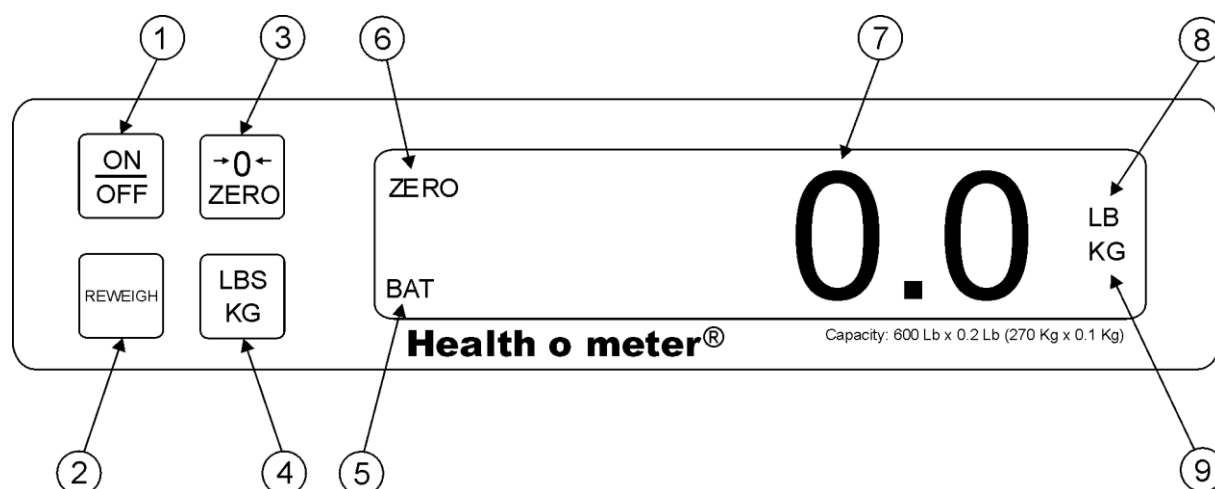
Figure 2. Control Panel

ITEM	DESCRIPTION	FUNCTION
1	ON/OFF button	Turns scale ON and OFF.
2	REWEIGH button	Allows repeated weighing of the patient without stepping off the scale.
3	ZERO button	Zeros digital display prior to weighing. Enables the cancellation (tare) of weight of mat or other accessories on the scale.
4	LBS/KG button	Selects kilograms or pounds.
5	BAT indicator	When ON, indicates scale batteries are low. The batteries should be replaced.
6	ZERO indicator	When ON, indicates the scale is stable and ready for use.
7	Digital Display 6 Digit LCD	Displays patient weight in kilograms or pounds as shown by the "LB" or "KG" indicator on the right side of the display. When REWEIGH button is depressed, display will blink "-----" until patient's weight is determined.
8	LB indicator	When ON, indicates displaying weight in pounds.
9	KG indicator	When ON, indicates displaying weight in kilograms.

OPERATING INSTRUCTIONS (CONT.)

1. Press the **ON/OFF** button to turn the scale on.
2. Wait until “0.0” appears on the digital display.
3. Place the patient on the scale. The display will blink “=====” until the weight of the patient is determined.
4. If you wish to reweigh the patient without removing them from the scale, press the **REWEIGH** button.
5. Remove the patient from the scale. The digital display will continue to read the patient’s weight until the **REWEIGH** or **ZERO** button is pressed.

NOTE: After 2 minutes of inactivity, the scale will turn off to conserve power. To restart the scale, press the **ON/OFF** button. (If it is powered by adapter, auto off function is disabled.)

Tare Function

When using this scale, the weight of an object, such as a blanket, can be subtracted from the total weight in order to determine the weight of the patient alone. Ensure that the edges of the object are on the scale and not touching the floor.

1. With all weight off the scale, press the **ON/OFF** button to turn on the scale.
2. Wait until the “0.0” appears on the display.
3. Place the object to be tared on the platform. The display will blink “=====” until the weight of the object is determined.
4. Do not remove the object. Then press the **ZERO** button.
5. Place the patient on the scale. The scale will automatically deduct the tare weight of the object from the gross weight of the patient and item together.
6. The weight of the patient will appear on the scale.
7. When the patient and the object are removed from the scale, the display will show “UL”.
8. To clear the tared weight, press the **ZERO** button.

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SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply	1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a stable surface 4. Scale is out of calibration	1. Remove interfering object from the scale 2. Remove patient, zero the scale and begin weighing process again 3. Place the scale on a stable surface and begin weighing process again 4. Check weight with known weight value
The display shows "OL"	The load on the scale exceeds the capacity	Remove the excess weight and use the scale according to its capacity of 600 lb / 270 kg
The display shows "UL"	The scale is in under-load condition	Make sure the rubber mat is placed on the base. Or press ZERO to clear a previously tared weight.
The display shows "LO BAT"	The batteries are low	Replace batteries according to instructions

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. The following calibration assumes the scale is set to pounds, and 200 lb is employed as the calibration standard.

Note: Only weights certified and traceable to national standards are to be used for calibration procedures.

Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the REWEIGH and ZERO buttons at the same time, while pressing the ON/OFF button to turn on the scale. "LOAD" will briefly appear on the display, followed by "200.0 LB" indicates you can calibrate. Release all three buttons.	"LOAD" followed by "200.0 LB"
Ensure there is no weight on the scale (except for the rubber mat) and press the REWEIGH button. "Clear" will flash on the display, followed by "-----". At this time the scale is calculating zero calibration. After zero calibration, the display will flash "PUT" followed by "200.0 LB".	"Clear" followed by "-----". Then "PUT" followed by "200.0 LB"
Place 200 lb on the scale. Then press REWEIGH . "CAL" will appear on the display, followed by "FAC" followed by 5 digits. The calibration factor should be in the range between 8000 and 10000. If the number is out of that range, contact Customer Service at 1-800-815-6615.	"CAL" followed by "FAC" followed by "X.XXXX".
If the calibration factor is within range, press the REWEIGH button. "DONE" will appear on the display, followed by "START", followed by "0.0". Remove the weight from the scale.	"DONE" followed by "START", followed by "0.0".
Calibration procedure is complete.	

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