# Health o meter®

P r o f e s s i o n a l

## Model 349KLX Remote Display Digital Scale

## User Instructions

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

[www.homscales.com](http://www.homscales.com)

Health o meter® Professional 349KLX

© Pelstar®, LLC 2019

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

## TABLE OF CONTENTS

Cautions and Warnings.....	3
Specifications .....	4
Certifications / Disposal.....	5
Assembly Instructions .....	6
Display Assembly Table Top Use Instructions .....	7
Display Assembly Wall Mount Instructions.....	8
Powering the Scale .....	10
Transporting the Scale with Optional Carrying Case.....	11
Operating Instructions .....	12
Maintenance.....	15
Calibration .....	16
Troubleshooting .....	17
Warranty.....	18

**Note: This scale has been factory calibrated, and does not require calibration prior to use.**

Please register your scale for warranty coverage at:

[www.homscales.com/product-support/technical-support/product-warranty-registration](http://www.homscales.com/product-support/technical-support/product-warranty-registration)

For User Instructions updates and revisions please go to:

[www.homscales.com](http://www.homscales.com)

## CAUTIONS AND WARNINGS

**To prevent patient/caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.**

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not transport the scale with a patient or object on the scale.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.

### **Patient/caregiver safety**

- Patient should wear socks or other form of lightweight foot covering during weighing event.
- To prevent patient injury, the patient must be attended throughout the entire weighing event. Caregivers should ensure that the patient is stable and provide support as needed when getting onto and off of the scale.

### **When operating the scale with the AC adapter:**

- Operate this scale exclusively with the AC adapter provided by Health o meter Professional. Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- For use with AC adapter model ADPT40 (USA or Canada) or ADPT30 (International) (not included)
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.

### **When operating the scale with batteries:**

- For accurate weighing, if the “LO” indicator activates, replace the batteries or switch to the AC adapter as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

**In no event shall Pelstar®, LLC be liable for damages or injuries whatsoever arising out of or connected with the assembly, use, or misuse of its products.**

# SPECIFICATIONS

## General

The Model 349KLX remote display digital scale uses highly sophisticated microprocessor technology. Each scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

This scale is set up to use weighing technology that is able to determine the actual weight of a moving patient.

Weight can be displayed in pounds (lb) or in kilograms (kg). For privacy and convenience, the scale features a remote display head which may be held by the operator, placed on a table or other flat surface, or mounted on a wall. **This scale can be powered using an AC adapter, (not included), or by 6 AA batteries (included).**

## 349KLX Scale Specifications

Capacity and Resolution	400 lb x 0.2 lb (180 kg x 0.1 kg)	
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE160714HKKK1-P Health o meter® Professional Model ADPT30 (Not Included) Input: 100-240V – 50/60Hz, 500mA Output: 9.0V - - - 0.5A  Or 6 AA Batteries (included)	
Environmental	Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 32°F to 125°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	<b>Platform</b> Length: 12" (305 mm) Width: 12.5" (317.5 mm) Height: 1.875" (48 mm) Weight: 8.8 lb (4.0 kg)	<b>Display assembly</b> Length: 7.75" (197 mm) Width: 1.5" (15 mm) Height: 4.25" (108 mm) Weight: 12.8 oz (0.36 kg)

# CERTIFICATIONS / DISPOSAL

## Certification Descriptions

	<p>Pelstar®, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar®, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
---	---

## 349KLX Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or battery. Do not dispose of this device in the domestic waste stream.

# ASSEMBLY INSTRUCTIONS

## **Before Assembly**

Each 349KLX remote display digital scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

## **Parts List**

### **Carton**



- (1) Scale platform
- (1) Display assembly
- (6) AA batteries
- (1) User instructions
- (1) Metal Stand
- (2) #8 Anchoring screws (4.2 mm nominal OD)
- (2) Drywall mounting anchors

## **Tools Required**

Phillips head screwdriver (not included)

# DISPLAY ASSEMBLY TABLE TOP USE INSTRUCTIONS

## Assembling the 349KLX Scale
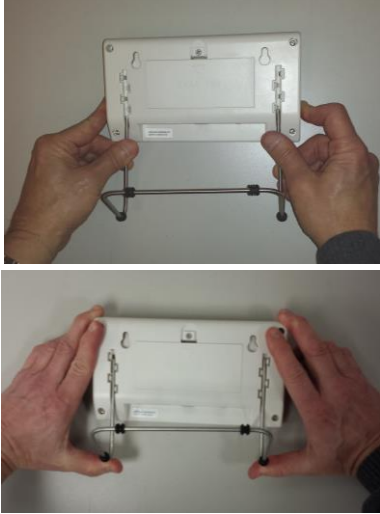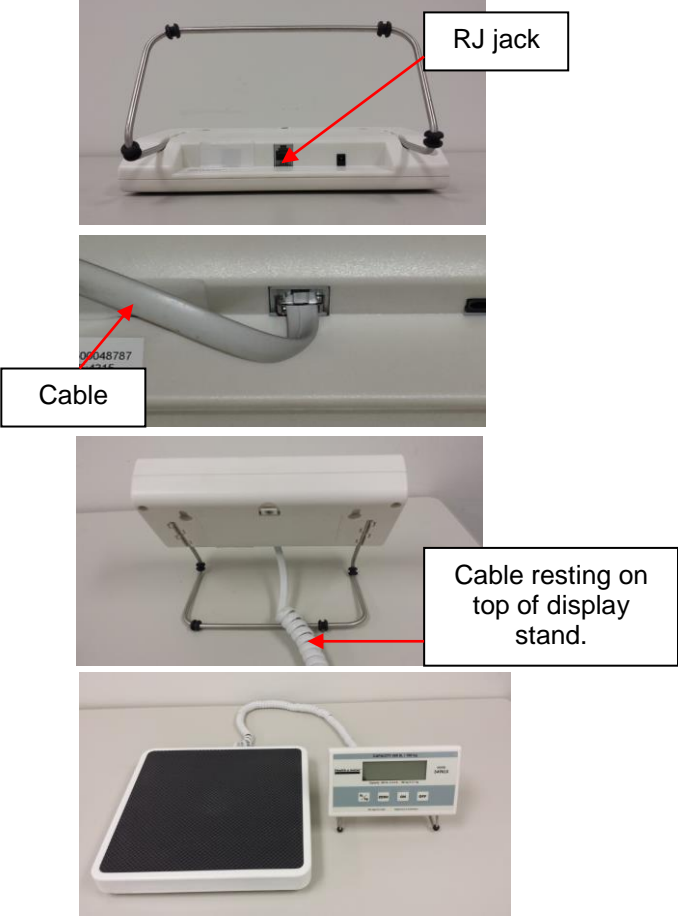
<p>1. Carefully remove the scale platform and display assembly from the carton, and place on a flat, level, and dry surface.</p>	
<p>2. If using the display assembly on a desk or table, insert the metal stand into the slots on the back of the display.</p>	
<p>3. Connect the cable to the RJ Jack in the bottom of the display assembly. Ensure that the cable rests on top of the metal stand so the display assembly rests flat on the table. If using the scale on battery power the scale is now ready for use. If using the scale with the optional AC adapter see page 10 for set up instructions.</p>	

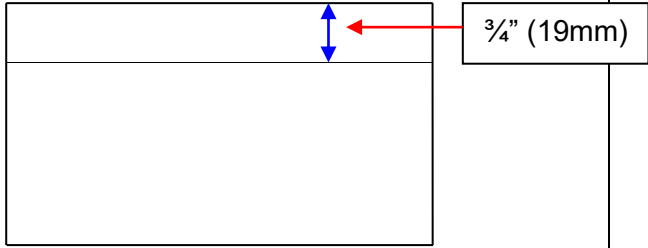
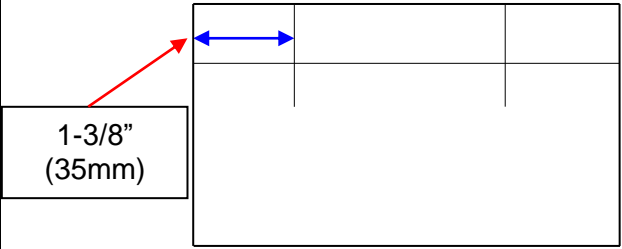
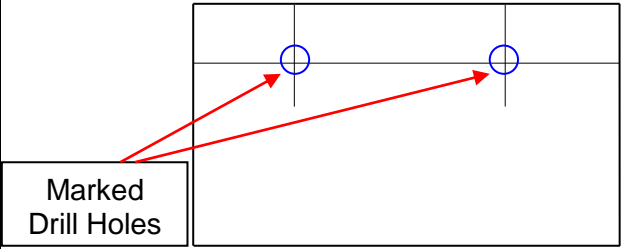
# DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS

**Required Parts**

- (1) Display assembly
- (2) #8 Anchoring screws (4.2 mm nominal OD)
- (2) Mounting anchors, compatible with wall construction

**Caution:** When wall mounting the display assembly, ensure that the mounting location is free of electrical lines, gas pipes, and other safety hazards. Ensure that wall is of construction that can support the display assembly.

**Note:** Remote head can be mounted to a wall up to 6 feet (1.83 m) from the base or it can be mounted on the display bracket (included) for placement on a table up to 6 feet (1.83 m) from the base.

<p>1. Position the display assembly on the wall, and carefully trace around the display.</p>	
<p>2. Mark a level horizontal line, 3/4" (19 mm) below the top of the display outline.</p>	
<p>3. Mark a vertical line 1-3/8" (35 mm) from each side of the display outline.</p>	
<p>4. At the horizontal and vertical intersection, carefully drill the appropriate size hole.</p>	

# DISPLAY ASSEMBLY WALL MOUNT INSTRUCTIONS (CONT)

5. Once the holes are marked, use a hammer or Phillips head screwdriver to set the 2 drywall anchors into the holes. Use a screwdriver to insert the drywall anchors completely into the wall. Screw the two mounting screws into the anchors. Adjust the screw head depth such that the screw heads protrude 1/8" (3.2 mm) from the wall.



**Warning: Extending the screw heads beyond 1/8" (3.2 mm) can damage the display assembly.**

6. Position the display assembly mounting holes onto the mounting screws, and carefully pull down on the display assembly to secure.


7. Connect the cable to the RJ jack in the bottom of the display assembly. If using the scale on battery power the scale is now ready for use. If using the scale with the optional AC adapter see page 10 for set up instructions.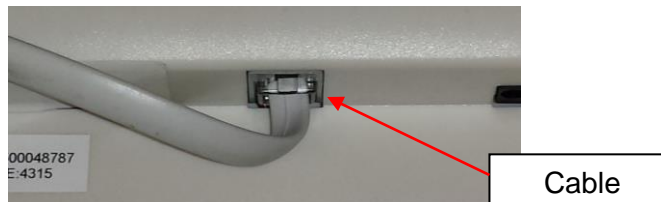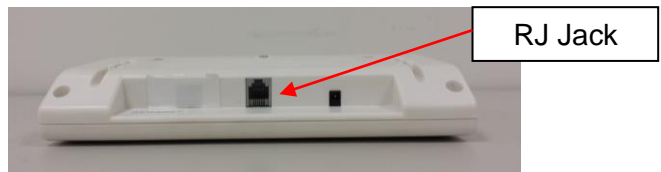

# POWERING THE SCALE

## Powering the Scale

If powering the scale via the optional AC adapter, plug the adapter into the display assembly and into a power source.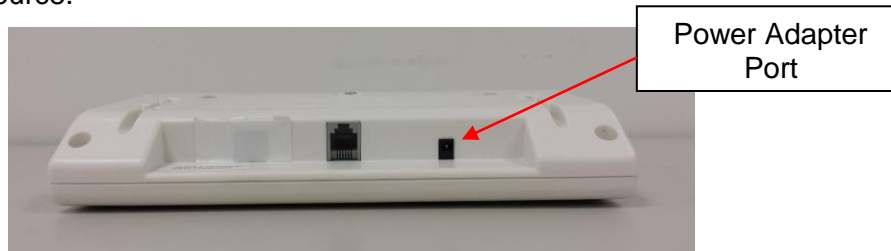

## Connection to Welch Allyn® Spot Vital Signs® Lxi Monitor.

**Warning:** When connecting to another medical device, the power adapter must not be used. Disconnect the power adapter from the scale and use only the battery power. This ensures proper electrical isolation.


For battery operation, see Battery Installation below.

## Battery Installation

1. Unplug the AC adapter from the power source.
2. Use a Phillips head screwdriver to remove the battery cover on the back of the display assembly.

3. Replace or install fresh batteries. Note the orientation that is shown and ensure the batteries are placed in the proper direction.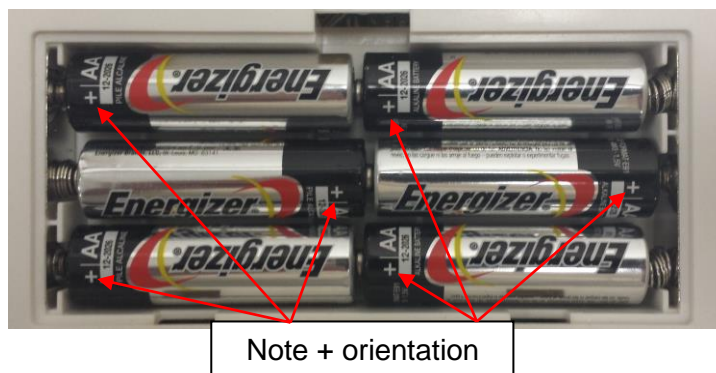

**Note:** Health o meter® Professional recommends using alkaline or lithium batteries, rated for 130°F (54°C) operation. Do not use zinc-carbon batteries.

4. Re-attach the battery cover to the display assembly.

**Warning:** If the scale will not be used for some time, remove the batteries to avoid a safety hazard. Disposal of batteries must be performed in accordance with the prevailing national, regional or local regulations that apply to you.

# TRANSPORTING THE SCALE WITH OPTIONAL CARRYING CASE

## **Preparing a 349KLX Scale for Transport**


This scale can be easily placed into a carrying case (item # 64771) for storage, or transport from one room to another. To prepare the scale for transport, follow these steps:

Optional  
Carrying Case  
Item # 64771  
Sold Separately

1. Remove the display assembly and display bracket from its current position.
2. Unplug the AC adapter and cable from the display assembly, and remove the AC adapter from the outlet.
3. Carefully store the AC adapter, display bracket, and display assembly in carrying case to prevent damage during transport.
4. Lift scale platform carefully from the floor and place it into the carrying case. Position the scale platform so that the RS-232 serial port and cable are facing up.

RS-232 serial port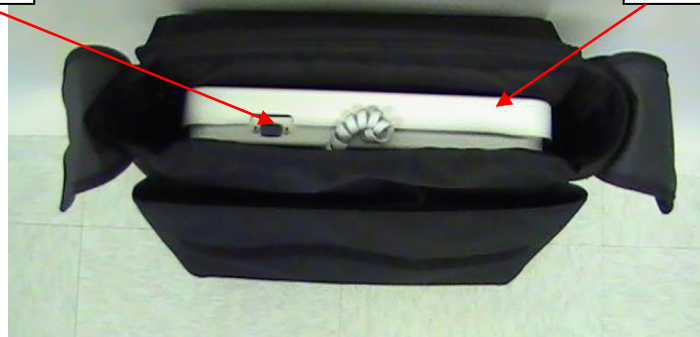

Scale platform

5. Close carrying case flap to cover the scale. Grasp the carrying case strap and carefully lift the case into the upright position. The scale can now be transported.

# OPERATING INSTRUCTIONS

## Keypad

Function	Description
<b>ON</b>	Powers scale on.
<b>OFF</b>	Powers scale off.
<b>ZERO</b>	Zeroes or tares the scale prior to a weighing.
<b>lb. / kg</b>	Toggles between pounds (lb) and kilograms (kg).

## Weighing a Patient

1. Make sure there is nothing on the weighing platform.
2. Press the **ON** button to power the scale on.
3. Wait until “0.0” LB or “0.0” KG appears on the display.
4. Press the **ZERO** button to zero the scale, if necessary.
5. Have the patient step onto the scale. Wait for the weight reading to stabilize. When the weight stabilizes, “Lock” will appear on the display along with the patient’s weight.  
**Note:** Depending on the movement of the patient on the scale, it may take approximately 3-4 seconds for the weight reading to stabilize.  
**Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.**
6. After recording the weight, have the patient step off the scale.
7. Press the **OFF** button to power the scale off.

# OPERATING INSTRUCTIONS (CONT)

## Auto-Off Function

As an energy saving feature, the scale automatically powers off after 2 minutes of inactivity. This function is enabled by default.

## Disable Auto-Off

1. While the scale is powered off, press and hold the **lb./kg** button.
2. While holding the **lb./kg** button, press the **ON** button to power the scale on.
3. "349" will briefly appear on the display followed by "A-OF" and then "0.0" lb appears.
4. Release the **lb./kg** and **ON** button.
5. Auto-off is now disabled, and the scale will remain on until the scale is manually powered off.

## Enable Auto-Off

1. While the scale is powered off, press and hold the **lb./kg** button.
2. While holding the **lb./kg** button, press the **ON** button to power the scale on.
3. "349" will briefly appear on the display, followed by "0.0" lb.
4. Release the **lb./kg** and **ON** button.
5. Auto-off is now enabled.

## Changing the Weight Measuring Unit

1. Press the **ON** button to power the scale on.
2. Wait for "0.0" LB or "0.0" KG to appear on the display.
3. Press the **lb./kg** button to toggle to the desired weight measuring unit.

## Locking the Weight Measuring Unit

The weight measuring unit can be locked to only measure in KG or LB. Follow the procedures below to lock the weight measuring unit.

### KG Lock Instructions

1. Press the **ON** button to turn the scale on. Wait for "0.0" to appear on the display.
2. Press and hold the **lb./kg** button so that KG icon is displayed and continue holding the **lb./kg** button for approximately 10 seconds.
3. When the KG icon flashes, release the **lb./kg** button. The KG weight measuring unit is now locked and will remain locked even if the scale is turned off.

### LB Lock Instructions

**Note:** To lock in LB weight mode, you must follow the instructions above to lock in KG weight mode first.

1. Once the scale is locked in KG mode, press and hold the **lb./kg** button for approximately 10 seconds until the LB icon flashes.
2. When the LB icon flashes, release the **lb./kg** button. The LB weight measuring unit is now locked and will remain locked even if the scale is turned off.

## Unlocking the Weight Measuring Unit

**Note:** If your scale is locked in KG mode, you must first follow the instructions above to lock into LB mode, and then begin the disabling instructions below.

1. Press the **ON** button to turn the scale on. Wait for "0.0" to appear on the display.
2. Press and hold the **lb./kg** button until LB and KG icons *both* flash. When LB and KG are flashing, release the **lb./kg** button. The weight measuring unit is now unlocked; pressing the **lb./kg** button will allow user to toggle between LB and KG.

## OPERATING INSTRUCTIONS (CONT)

### **Tare Function**

When using this scale, the weight of an object can be subtracted from the total weight in order to determine the weight of the patient alone. The Tare Function (**ZERO** button) automatically performs this subtraction.

1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Place the object to be tared onto the weighing platform. The display will show a value for the objects weight.
4. With the object remaining on the scale, press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the patient onto the scale. The scale will automatically deduct the weight of the object and only display the patient's weight.
6. With all the weight off the scale, "under" will be temporally displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

### **Child/Parent Function**

The Parent/Child function on Health o meter® Professional scales provides the convenience of weighing a small child or infant while being held by a parent. Follow these simple steps to subtract the parent's weight from the total so only the child's weight is displayed.

1. With all weight off the scale, press the **ON** button to turn the scale on.
2. Wait until "0.0" LB or "0.0" KG appears on the display.
3. Ask the parent to step onto the scale without the child. The display will show a value for the parent's weight.
4. Have the parent remain standing on the scale and press the **ZERO** button. The word "TARE" or "UNDER" will appear on the display, followed by "0.0" LB or "0.0" KG.
5. Place the child into the parent's arms. The scale will automatically deduct the weight of the parent and only display the child's weight.
6. When the parent steps off of the scale, "under" will be temporally displayed and then the display will return to "0.0" LB or "0.0" KG.
7. Press **ZERO** after weighing process is complete to return to normal weighing mode.

# MAINTENANCE

## **Maintenance**

The following pages provide instructions for maintenance, cleaning, calibration, and troubleshooting for the 349KLX scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

**Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.**

1. Check overall appearance of the scale for obvious damage, wear, and tear.
2. Inspect the AC adapter for cord cracking or fraying, or for broken/bent prongs.

**Note:** This scale is an extremely sensitive weighing device. If the auto-off function is disabled, battery power will be consumed at a rapid rate.

## **Cleaning and Disinfecting**

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

**Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.**

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
  - mild soap and water solution
  - 70% isopropyl alcohol
  - solution with 1-5% hydrogen peroxide concentration

After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.

2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

## **Replacement Parts**

Replacement parts may be available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

<b>Part #</b>	<b>Description</b>
ADPT40	Power adapter (USA and Canada)
ADPT30	Power adapter (International)

# CALIBRATION

## Calibration

**This scale has been factory calibrated and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of the 349KLX scale is performed using certified weights. 50 / 100 / 200 pounds or 40 / 80 / 120 kg are employed as the calibration standards. The display must be in the desired unit of measure (LB or KG) before entering the calibration procedure.

**Caution:** To avoid injury, use extreme care when stacking weights on platform.

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

## Scale Calibration

ACTION	DISPLAY SHOWS
Press and hold the <b>lb./kg</b> and <b>ZERO</b> buttons at the same time, then press the <b>ON</b> button to power on the scale. "349" will briefly appear on the display, followed by "CAL" and "UXX". (Two alphanumeric characters will appear next to "U".) Release all three buttons. Several alphanumeric characters will appear on the display (1-4 characters can be displayed during calibration).	"349" followed by "UXX" then "XXXX"
Press the <b>ZERO</b> button. "C-50" (in LB mode) or "C-40" (in KG mode) will flash on the display, then the display will clear with only the unit of measure shown.	"C-50" (in LB mode) or "C-40" (in KG mode)
Place 50 lbs or 40 kg on the scale. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-100" (in LB mode) or "C-80" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-100" (in LB mode) or "C-80" (in KG mode)
Place an additional 50 lb on the scale, for a total of 100 lb or an additional 40 kg on the scale for a total of 80 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "C-200" (in LB mode) or "C-120" (in KG mode) will appear on the display, then the display will clear with only the unit of measure shown.	"C-200" (in LB mode) or "C-120" (in KG mode)
Place an additional 100 lb on the scale for a total of 200 lbs or an additional 40 kg on the scale for a total of 120 kg. Several alphanumeric characters will appear. When calibration characters stabilize, press the <b>ZERO</b> button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off. Calibration is complete and the scale is ready to use.	"XXXX" followed by "END" then "OFF"

# TROUBLESHOOTING

## Troubleshooting

Before contacting service personnel, refer to the following instructions to check and correct any failures.

<b>Symptom</b>	<b>Possible Cause</b>	<b>Corrective Action</b>
Scale does not power on	<ol style="list-style-type: none"> <li>1. Dead batteries</li> <li>2. Faulty electrical outlet</li> <li>3. Bad power supply</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace batteries</li> <li>2. Use a different outlet</li> <li>3. Replace AC adapter</li> </ol>
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> <li>1. External object interfering with the scale</li> <li>2. The display did not show "0.0" before weighing</li> <li>3. Scale is not placed on a level floor</li> <li>4. Scale is out of calibration</li> </ol>	<ol style="list-style-type: none"> <li>1. Remove interfering object from the scale</li> <li>2. Ask the patient to get off the scale, zero the scale and begin weighing process again</li> <li>3. Place the scale on a level floor and begin weighing process again</li> <li>4. Check weight with known weight value</li> </ol>
Weighing is performed but the weighing process takes too long and the weight does not lock on the display	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (400 lb / 180 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The batteries are depleted	Replace batteries according to instructions

# WARRANTY

## **Limited Warranty**

### **What does the Warranty Cover?**

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar®, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

### **Who is Covered?**

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar® dealers or retail stores selling Pelstar® products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

### **What is Excluded?**

**Your warranty does not cover normal wear of parts or damage resulting from any of the following:** negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

**To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase.** Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC  
Attention R/A# \_\_\_\_\_  
Return Department  
9500 West 55<sup>th</sup> St.  
McCook, IL 60525

### **Extended Warranty Available\***

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit [www.homscales.com/scalesurance/](http://www.homscales.com/scalesurance/) or contact your medical supply distributor.

\*Not available in all countries

**PELSTAR®, LLC**

**9500 West 55th St. McCook, IL 60525-7110 USA**

**1-800-638-3722 or 1-708-377-0600**

**PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:**

[www.homscales.com](http://www.homscales.com)

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar®, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Health o meter® est une marque de commerce enregistré de Sunbeam Products, Inc. Utilisée sous licence.
Les produits Health o meter® Professional sont manufacturés, conçus, et possédés par Pelstar, LLC.