

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. Calibration of your scale is performed using kilograms (KG) or pounds (LB), according to the units of measure used upon entering into the calibration path. When calibrating in pounds, a total of 300 lbs is required, applied in increments of 100 lb, 200 lb and 300 lb. If calibrating in kilograms, a total of 120 kg is required, applied in increments of 40 kg, 80 kg and 120 kg.

THE 499KG CAN ONLY BE CALIBRATED IN KILOGRAMS.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the LB/KG button (499KL) or the blank blue button (499KG) and ZERO buttons at the same time, then press the ON/OFF button to turn on the scale. Release all three buttons when “CAL” appears on the display. Numbers will appear indicating that you can begin calibration.	“CAL” followed by numbers
When calibration numbers stabilize, press the ZERO button. “C-100” (for LB mode) or “C-40” (for KG mode) will flash on the display, followed by “0” or “1”.	Numbers followed by “C-100” (LB mode) or “C-40” (KG mode) then “0” or “1”.
Place 100 lbs or 40 kg on the scale. Numbers will appear on the display. When calibration numbers stabilize, press the ZERO button. “C-200” (for LB mode) or “C-80” (for KG mode) will appear on the display, followed by “0” or “1”.	Numbers followed by “C-200” (LB mode) or “C-80” (KG mode) then “0” or “1”.
Place an additional 100 lbs or 40 kg on the scale, for a total of 200 lbs or 80 kg. Numbers will appear on the display. When calibration numbers stabilize, press the ZERO button. “C-300” (for LB mode) or “C-120” (for KG mode) will appear on the display, followed by “0” or “1”.	Numbers followed by “C-300” (LB mode) or “C-120” (KG mode) then “0” or “1”.
Place an additional 100 lbs or 40 kg on the scale, for a total of 300 lbs or 120 kg. Numbers will appear on the display. When calibration numbers stabilize, press the ZERO button. “END” will appear on the display, followed by “OFF”. The scale will automatically shut off.	Numbers followed by “END” then “OFF”