

# Health o meter®

P r o f e s s i o n a l



PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

www.homscales.com

## USER INSTRUCTIONS

Health o meter® Professional 800KL

### Cautions and Warnings

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale with anything on the platform.
- For accurate weighing, the scale must be placed on a flat, stable surface.
- The scale contains sensitive electronic parts. Avoid rough treatment. Do not drop, kick, or jump on it. Treat it with care to ensure the best performance.
- Do not store scale where you store cleaning chemicals. The vapors from some household products may affect the electronic components of your scale. Do not store scale on its side.
- This scale is an extremely sensitive weighing device. In order to prevent run down of the battery, do not store anything on the scale.

In no event whatsoever shall Pelstar®, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

### Specifications


<b>Capacity and Resolution</b>	Capacity: 400 lb / 180 kg / 28 st 8 lb Resolution: 0.1 lb / 0.05 kg
<b>Environmental</b>	Operating temperatures: 5°C to 40°C (41°F to 104°F) Storage temperatures: 0°C to 50°C (32°F to 125°F) Atmospheric Pressure: 500hPa to 1060hPa; Maximum Humidity: 85% RH in use or in storage
<b>Platform Size</b>	12 1/4" (w) x 12 1/4" (d) x 1 3/4" (h) / 311mm (w) x 311mm (d) x 44mm (h)

### Before First Use

1. The scale is shipped with the batteries installed. Pull and remove the battery insulator tab from within the battery compartment before first use.
2. Place scale on flat, stable surface.
3. To turn on the scale, tap firmly with your foot on the center of the platform. Wait for 0.0 to appear, then step onto the scale (disregarding the reading) and step off. This process insures the scale is set and is now ready for use. Do this whenever the scale is moved.


### To Set Weight Mode (KG, LB, or ST)

Slide the switch to the desired weight mode.

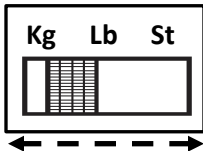

### To Weigh

To turn on the scale, tap firmly with your foot on the center of the platform. Wait for 0.0 to appear, then step onto the scale.

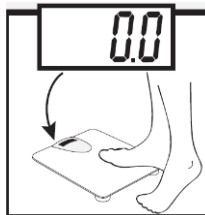


The measurement will flash on the display twice then lock onto the weight. If you continue to stand on the scale, your weight will be displayed for 10 seconds and then the scale will turn off automatically. Weight will remain displayed for 10 seconds after stepping off the scale.




## Troubleshooting

If the display shows “Lo” message, replace batteries per the included instructions.

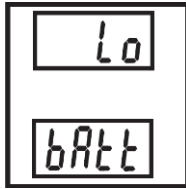


If the display shows “Err E” the load on the scale exceeds the capacity (400 lb / 180 kg /28 st 8 lb).

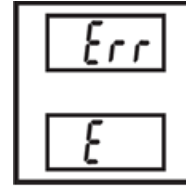


## Battery Instructions

The scale is shipped without batteries already installed. The batteries are included in a small plastic bag. Follow these instructions to insert the batteries on a new scale or to replace depleted batteries on your current scale. The battery compartment is located on the underside of the scale.

1. To open the battery cover, push the battery compartment tab to the right and lift upwards. **Note:** You do NOT need to remove any screws to open the battery compartment.

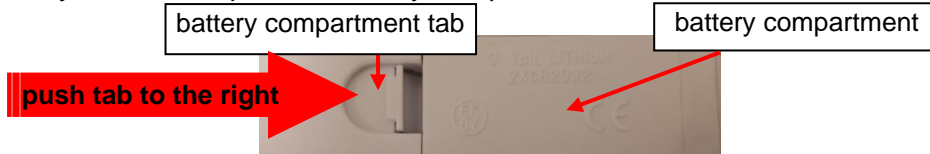

2. Install two new AAA batteries. Follow the battery illustration to ensure the batteries are inserted in the proper direction.

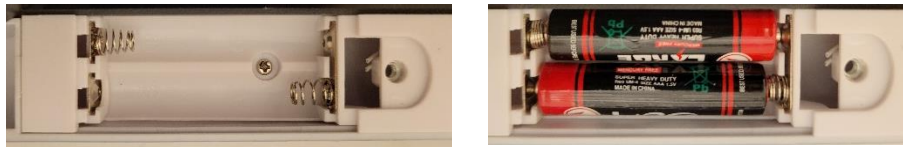

3. Replace the battery compartment cover by sliding the end of the cover into the battery compartment. Then push down on the tab and compartment to ensure the battery cover is firmly attached. You do NOT need the screw to close the battery compartment. You may use the screw to further secure the battery compartment if desired. Do not use the scale without the battery compartment cover attached.



## Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

1. Health o meter<sup>®</sup> Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
  - mild soap and water solution
  - 70% isopropyl alcohol
  - solution with 1-5% hydrogen peroxide concentration
2. After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.
3. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
4. Do not submerge the scale in water or any other liquid.
5. Do not pour or spray fluids directly on the scale.

## 1-Year Limited Warranty

This Health o meter<sup>®</sup> Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

For information or support, please contact Customer Service at 1 (800) 815-6615 or [HomProCS@homscales.com](mailto:HomProCS@homscales.com)

For complete warranty terms visit [www.homscales.com](http://www.homscales.com)

Health o meter<sup>®</sup> is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter<sup>®</sup> Professional products are manufactured, designed, and owned by Pelstar<sup>®</sup>, LLC.

We reserve the right to improve, enhance, or modify Health o meter<sup>®</sup> Professional product features or specifications without notice.

© Pelstar, LLC 2023