

Health o meter[®]
P r o f e s s i o n a l

Models 844KL / 844KL-USB Digital Floor Scale



ISO 9001:2015
CERTIFIED

ISO13485:2016
CERTIFIED

User Instructions

Para ver las instrucciones de uso en español visita www.hom scales.com



PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA
www.hom scales.com

weigh**easier**[®]

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Cautions and Warnings.....	3
Specifications	3
Certifications	3
Service	3
Set-up Instructions	4
Operating Instructions	5
Troubleshooting	6
Maintenance & Cleaning	6
Warranty.....	7

Note: This scale has been factory calibrated and cannot be calibrated by the user.

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTIONS AND WARNINGS / SPECIFICATIONS / CERTIFICATIONS / SERVICE

Cautions and Warnings

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale with anything on the platform.
- For accurate weighing, the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.



In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

Specifications

This digital floor scale displays weight in pounds, kilograms or stones. It also features a weight hold that allows the weight measurement to stay displayed for 5 seconds after stepping off scale. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements and provides features that make the weighing process simple, fast and convenient. This scale is set up to determine the weight of a stationary patient.

Capacity and Resolution	Capacity: 440 lb / 200 kg / 31 st Resolution: 0.1 lb / 0.05 kg / 0.125 lb	
Environmental	Operating temperatures: 10°C to 35°C (50°F to 95°F) Storage temperatures: 0°C to 50°C (32°F to 125°F) Atmospheric Pressure: 500hPa to 1060hPa Maximum Humidity: 85% RH in use or in storage	
Physical Dimensions	Platform Size: Width: 12 5/8" (321 mm) Depth: 12 5/8" (321 mm) Height: 1 7/8" (48 mm)	Product Footprint: Width: 12 5/8" (321 mm) Depth: 12 5/8" (321 mm) Height: 1 7/8" (48 mm) Weight: 4 lb (1.8 kg)

Certifications

 	<p>Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
--	---

Customer Service Information

This scale is shipped assembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

For further information or telephone support, please contact Customer Service at:
 Telephone: 1 (800) 815-6615 Email: HomProCS@hom scales.com

SET-UP INSTRUCTIONS

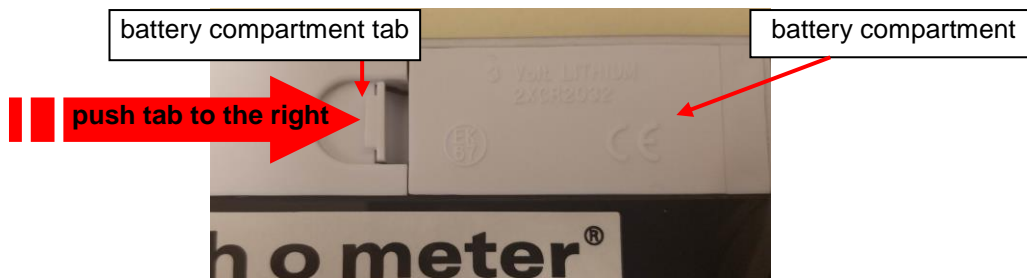
To avoid damaging the scale when unpacking it, **do not use a box cutter, knife, scissors or any sharp object to open the protective inner packaging.** Remove scale from the carton and unwrap packing material carefully, to prevent scratching the unit. Set the carton aside for storage.

Battery Instructions

The scale is shipped with batteries already installed. Before first use, you must remove the pull tab located within the battery compartment tab. The battery compartment is located on the underside of the scale. To replace the batteries, follow these steps.

1. To open the battery cover, push the battery compartment tab to the right and lift upwards.

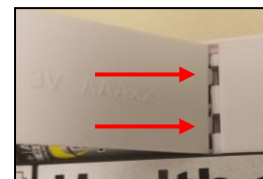
Note: You do NOT need to remove any screws to open the battery compartment.



2. To remove the batteries, press upward on the white tab that is holding the battery. Moving the tab will loosen the battery so it can be removed. Remove both batteries and replace with two new 3V Lithium (CR2032) batteries. Follow the battery illustration to ensure the batteries are inserted with the + side facing up.

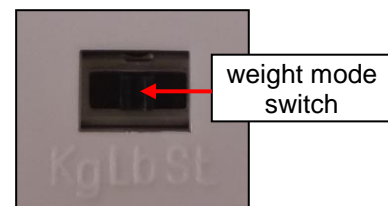


3. Replace the battery compartment cover by sliding the end of the cover into the battery compartment. Then push down on the tab and compartment to ensure the battery cover is firmly attached.



Selecting the Weight Mode (Kilograms/KG, Pounds/LB, or Stones/ST)

1. To select the weight mode, turn the scale over to access the weight mode switch.
2. Slide the switch to the desired weight mode.
3. The far left position is for Kilograms. Center position is for Pounds. The far right position is for Stones.



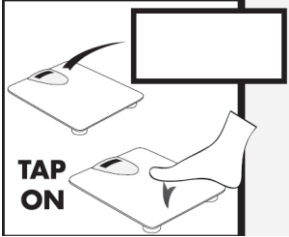
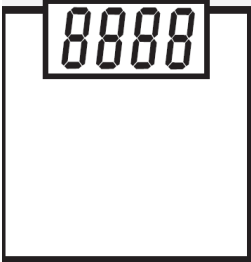
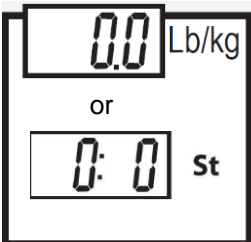
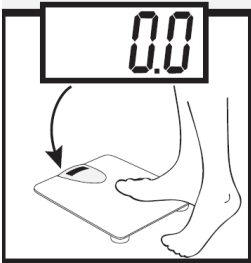
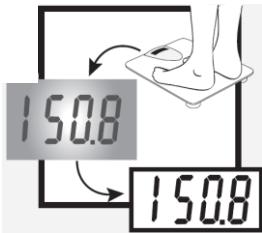
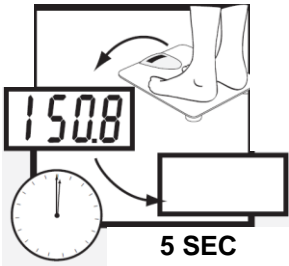
Connectivity Option

The 844KL-USB includes a USB port for connectivity to a PC or other device. To learn more about the connectivity options for this scale, please contact Health o meter® Professional Customer Service at 1-800-815-6615 or homprocs@homscales.com.



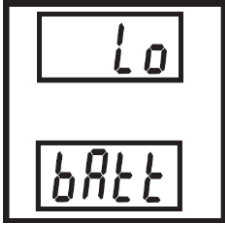
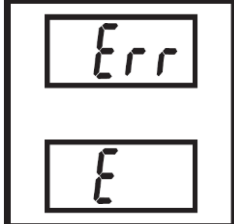
OPERATING INSTRUCTIONS

Measuring Weight

1. To turn on the scale, tap firmly with your foot on the center of the platform.	
2. Four digits will appear on the display.	
3. Wait for 0.0 to appear on the display.	
4. Step onto the scale. Stand still until weight appears.	
5. After the weight has locked, the measurement will flash on the display.	
6. Weight will remain displayed for 5 seconds after stepping off the scale.	

TROUBLESHOOTING / MAINTENANCE & CLEANING

Refer to the following instructions to check and correct any failure before contacting Customer Service.

<p>If the display shows “Lo bAtt” message, replace batteries per the instructions on page 4.</p>	 The image shows a digital scale display with two rectangular boxes. The top box contains the text "Lo" and the bottom box contains the text "bAtt".
<p>If the display shows “Err E” the load on the scale exceeds the capacity. 844KL Capacity: 440 lb / 200 kg / 31 st</p>	 The image shows a digital scale display with two rectangular boxes. The top box contains the text "Err" and the bottom box contains the text "E".

For further assistance please contact technical support toll free at 1 (800) 638-3722.

Maintenance

Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the entire scale for any obvious damage, wear and tear.
2. Do not store near chemicals, heaters or radiators.
3. Do not drop, kick, or jump on the scale.
4. This scale is a sensitive weighing device. Do not store anything on top of the scale platform.
5. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty.

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

1. Health o meter® Professional recommends using a mild soap and water solution on a soft cloth or disposable wipe:
2. After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure all scale parts are completely dry after cleaning.
3. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
4. Do not submerge the scale in water or any other liquid.
5. Do not pour or spray fluids directly on the scale.

WARRANTY

1-Year Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 West 55th Street

McCook, IL 60525



9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

© Pelstar, LLC 2021