

# Medical Scale Assessment Checklist

Use this checklist to ensure your medical scales are safe, efficient, and meet the needs of your facility:



## Platform Scale Assessment

- ☐ Check to see if height rod is loose or does not hold its position
- ☐ Check to see if the handlebars are loose
- ☐ Check to see if the main column is loose
- ☐ Look for signs of rust
- ☐ See if the pan that encompasses the platform is filled with dirt and debris
- ☐ Check to see if bottom tray is bent and scale wobbles
- ☐ Inspect the platform mat for tears, peeling, or damage

## Wheelchair Scale Assessment

- ☐ Look for signs of rust
- ☐ See if the scale looks acceptable/ dirty
- ☐ Ask if Clinicians are having difficulty with raising and lowering the platform
- ☐ Check to see if the platform is large enough to handle all wheelchair sizes
- ☐ Check that ramps have manageable slope inclines for easy use

## Pediatric Scale Assessment

- ☐ Check to see if the tray or seat is dirty or chipped
- ☐ Check to see if pegs are broken
- ☐ Check to see if the handles are loose
- ☐ Look for signs of rust
- ☐ Ask if Clinicians have trouble locking on an infant's weight

**Health o meter Professional Scales** offers field representatives to help assess your equipment and determine if repairs or replacements are needed. Replacement parts may also be available to restore your scale's performance. For assistance, contact Customer Service at 1-800-815-6615 or [homprocs@homscales.com](mailto:homprocs@homscales.com).

