

# Health o meter®

## P r o f e s s i o n a l

### Model Number: 280HR Infant Measuring Mat



#### **Intended Use**

This Health o meter® Professional infant measuring mat is intended to be used by trained medical staff. This product was designed to measure pediatric patients who are safely positioned on the mat. Do not modify the product or use it for anything other than its intended purpose.

#### **Cautions and Warnings**

Before using the measuring mat please make time to read the following safety instructions:

- To prevent patient injury, the patient must be attended throughout the entire measuring event.
- Ensure to measure the infant on a flat, secure surface.

**In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.**

#### **Specifications**

Model	280HR
Range of measurement	10 ~ 110 cm (3.9" ~ 43.3")
Division of measurement	5mm / 1/4"
Product dimensions	1400mm (L) x 300mm (W) / 55.1" (L) x 11.8" (W)
Product weight	.4kg / .88lb
Operating Temperature	5°C ~ 35°C / 41°F ~ 95°F
Storage Temperature	0°C ~ 50°C / 32°F ~ 122°F



**PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA** [www.homscales.com](http://www.homscales.com)


Health o meter® Professional 280HR

© Pelstar, LLC 2023

weigh**easier**®

## **How to Use the Measuring Mat**

1. Gently lay the infant onto the measuring mat. Ensure the infant's head is aligned at the top of the mat along the headpiece positioner.
2. While keeping the infant positioned on the mat, lift the footpiece positioner towards the infant's feet.
3. Straighten the infant's legs and gently press them downwards onto the mat.
4. Align the footpiece positioner onto the soles of the baby's feet.
5. Read the length measurement using the CM or IN markings on the measuring mat.

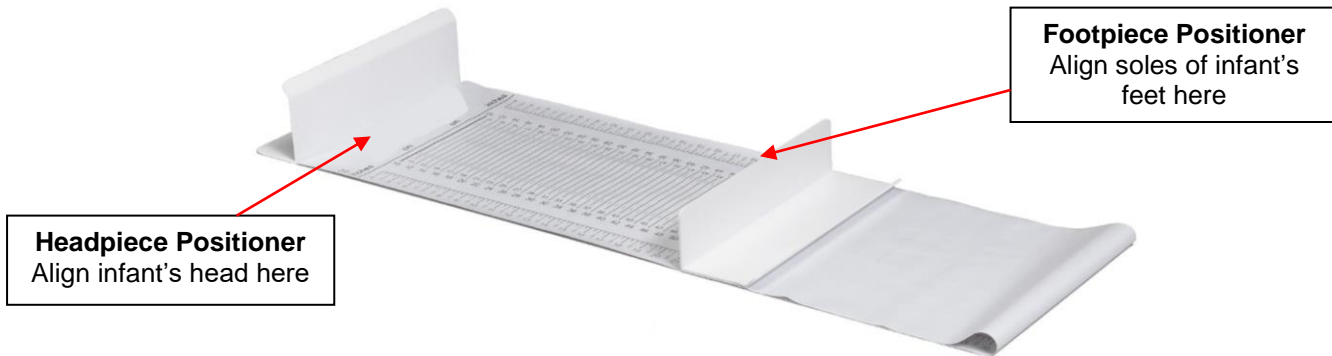

## **Cleaning and Disinfecting**

1. Health o meter<sup>®</sup> Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
  - mild soap and water solution
  - 70% isopropyl alcohol
  - solution with 1-5% hydrogen peroxide concentrationAfter cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure all parts are completely dry after cleaning.
2. Never use rough or abrasive materials to clean the mat.
3. Do not submerge the mat in water or any other liquid.
4. Do not pour or spray fluids directly on the mat.

## **Warranty**

This Health o meter<sup>®</sup> Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. For the complete warranty coverage information visit [www.homscales.com](http://www.homscales.com).



**PELSTAR<sup>®</sup>, LLC**

**9500 West 55th St. McCook, IL 60525-7110 USA**

**1-800-638-3722 or 1-708-377-0600**

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

[www.homscales.com](http://www.homscales.com)

Health o meter<sup>®</sup> is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter<sup>®</sup> Professional products are manufactured, designed, and owned by Pelstar<sup>®</sup>, LLC. We reserve the right to improve, enhance, or modify Health o meter<sup>®</sup> Professional product features or specifications without notice.

© Pelstar<sup>®</sup>, LLC 2023