209HR WALL-MOUNTED DIGITAL HEIGHT ROD

**ISO13485:2003
CERTIFIED**

USER INSTRUCTIONS

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

www.homscases.com

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

CAUTIONS AND WARNINGS.....	3
SPECIFICATIONS	3
CERTIFICATIONS / DISPOSAL.....	3
WALL MOUNT INSTRUCTIONS.....	4
POWERING THE TOUCH BAR.....	6
INITIALIZING THE TOUCH BAR.....	7
OPERATING INSTRUCTIONS.....	8
MAINTENANCE	9
WARRANTY.....	10

Please register your product for warranty coverage at:
www.homscales.com/product-support/technical-support/product-warranty-registration

For User Instructions updates and revisions please go to:
www.homscales.com

CAUTIONS AND WARNINGS

INTENDED USE

This Health o meter® Professional height rod is intended to be used in a professional medical environment by trained medical staff. This product was designed to measure the height of patients who are safely positioned in front of the height rod. Do not modify the product or use it for anything other than its intended purpose.

To prevent patient/caregiver injury or damage to your product, please follow these instructions very carefully.

- For accurate measurement, the 209HR must be affixed to a stable wall surface.
- For accurate measurement, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- When wall mounting the 209HR, ensure that the mounting location is free of electrical lines, gas pipes, and other safety hazards. Ensure that the wall is of construction that can support the 209HR.
- To prevent patient injury, the patient must be attended throughout the entire measuring event.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS

GENERAL

The 209HR High-Strength Wall-Mounted Digital Height Rod has a large headpiece and features a digital indicator that allows for accurate measurement of a patient's height. Each 209HR is designed to determine the height of a stationary patient. Height is displayed in feet / inches or centimeters.

209HR SPECIFICATIONS

Measuring Distance	39.4" / 100 cm
Resolution	0.1" / 1 mm
Power Requirements	2 AAA Alkaline batteries
Physical Dimensions	Width: 2 1/8" / 54 mm; Depth: 1/2" / 13 mm; Height: 45 5/8" / 1160 mm

CERTIFICATION DESCRIPTIONS

	Pelstar, LLC has been officially certified as a manufacturer of Health o meter® Professional medical devices to be in compliance with stringent ISO requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
---	--

209HR DISPOSAL

This Health o meter® Professional product must be disposed of properly. Follow national, regional or local regulations that apply to you for disposal of domestic waste.

WALL MOUNT INSTRUCTIONS

To avoid damaging the 209HR when unpacking it, **do not use a box cutter, knife, scissors or any sharp object when opening the protective inner packaging.** If damage is found, contact your shipper or a Health o meter[®] Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the carton as you unpack the parts for assembly. Set the carton aside for storage.


Parts List

- (1) User instructions
- (2) Double-sided adhesive tape
- (1) 1m calibration rod
- (2) AAA Alkaline batteries
- (1) Small clear bag containing:
 - Nail
 - Nail holder
 - Flower cap
 - Screws

Tools Required (not included)

Hammer or Phillips head screwdriver

Note: DO NOT USE A MAGNETIC SCREWDRIVER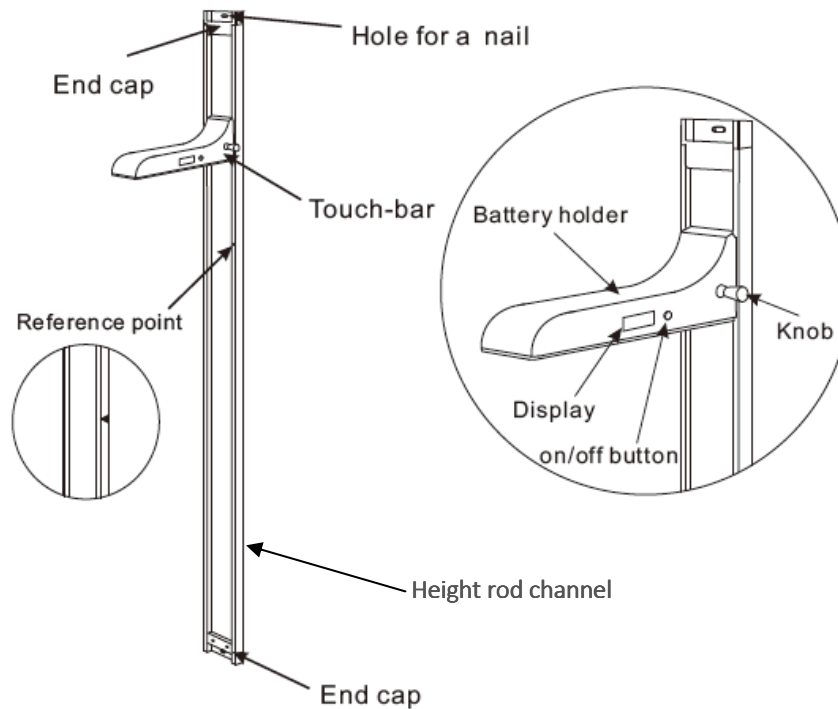

WALL MOUNT INSTRUCTIONS (CONT.)

Caution: When mounting the 209HR, ensure that the mounting location is free of electrical lines, gas pipes, and other safety hazards. Ensure that wall can support the 209HR.


Note: Do not mount the 209HR on a door.

Determining the Mounting Location:

The height rod channel can be mounted on the wall at any height, based on intended usage. Mount the height rod channel lower on the wall for pediatric use or higher on the wall for use with adults.

Guidelines for determining the base line location of the height rod channel are shown below. The height rod may be mounted with the bottom of the device anywhere from 21.7" - 37.4" (55 cm – 95 cm) from the floor.

Measurable ranges are different from mounted height.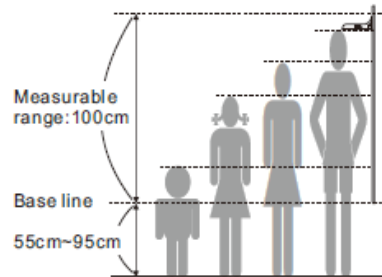

Desired Maximum Height Measurement	Base line Location (Distance from floor to bottom of height rod)
78.7" / 200 cm maximum height available	37.4" / 95 cm (distance from floor)
63" / 160 cm pediatric use only	21.7" / 55 cm (distance from floor)

Base Line	Measurable Range
95 cm / 37.4"	100 – 200 cm / 39.4" – 78.7"
85 cm / 33.5"	90 – 190 cm / 35.4" – 74.8"
75 cm / 29.5"	80 – 180 cm / 31.5" – 70.9"
65 cm / 25.6"	70 – 170 cm / 27.6" – 66.9"
55 cm / 21.7"	60 – 160 cm / 23.6" – 63"


Wall Mounting Instructions:

<p>1. Determine the location on the wall where the height rod will be mounted. While holding the height rod channel at the determined location, using a pencil or screw, mark the wall at the base line and in the nail hole at the top of the height rod channel.</p>	<p>Base line should not exceed 37.4" / 95 cm from the floor.</p>
<p>2. Remove the lining paper from the double sided adhesive tape at both ends of the height rod channel and affix the height rod channel onto the wall. To secure the height rod channel on the wall use either the included nail and flower cap or one of the screws. Insert the nail or screw through the hole at the top of the height rod channel and into the wall. Place the flower cap onto the nail head.</p> <p>NOTE: Do not use a magnetic screwdriver.</p>	

POWERING THE TOUCH BAR

Installing / Replacing Batteries:

1. Locate the battery cover on the top of the touch bar. Remove the battery cover by pressing inward on the locking tab.
2. Install 2 new AAA batteries. Health o meter® Professional recommends using an alkaline or lithium battery, rated for 130°F (54°C) operation. Do not use zinc-carbon batteries.
3. Re-attach the battery cover to the touch bar.

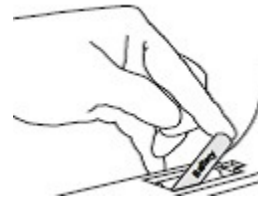Note: After the batteries are replaced, the measurement unit on the touch bar will always default to the measurement unit last used.

Low Battery Troubleshooting:

1. If the touch bar does not power on with one button press the batteries may be low. Try double-clicking the power button to power on the touch bar.
2. A. If “batt” is flashing on the display, it indicates that the battery power is low. To clear the “batt” message and continue using the touch bar, raise or lower the touch bar past the reference mark until “batt” stops flashing. Then push the touch bar down the height rod channel to begin measuring. Replace the batteries as soon as possible.

B. If the touch bar will not power on after double-clicking the power button replace the batteries.

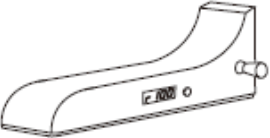
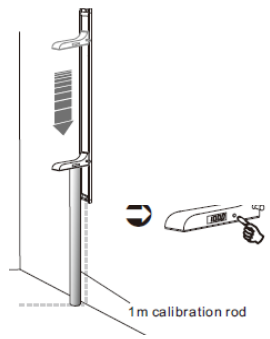
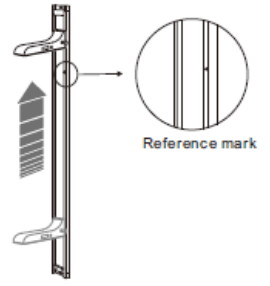
Note: As battery power is depleted the brightness of the digits on the display will gradually diminish.

Warning: If the height rod will not be used for some time, remove the batteries to avoid a safety hazard. Disposal of batteries must be performed in accordance with the prevailing national, regional or local regulations that apply to you.

INITIALIZING THE TOUCH BAR

The digital touch-bar must be initialized before first time use and after re-mounting.


<p>1. Power on the touch bar by pressing the on/off button. Start will flash on the display.</p>	
<p>2. Press and hold the on/off button for approximately 7 seconds until 100 flashes on the display.</p>	
<p>3. Obtain the 1 m calibration rod. Stand the rod vertically against the height rod channel as shown.</p>	 <p>1m calibration rod</p>
<p>4. Ensure that the touch bar is above the reference mark. Then push the touch bar down until it touches the top of the calibration rod. Press the on/off button and "100.0" will flash on the display.</p> <p>NOTE: if there is no calibration rod, any 1m measuring stick or rod can be used. NOTE: wrong initializing will cause height value error.</p>	 <p>1m calibration rod</p>
<p>5. Move the touch bar up the height rod channel until it passes the reference mark, beeps and Start flashes on the display. Move the touch bar down the height rod channel until it passes the reference mark again, beeps, and a height measurement appears on the display. The height rod is now ready for use.</p>	 <p>Reference mark</p>

OPERATING INSTRUCTIONS

Selecting Height Measuring Unit

After initializing the touch bar in centimeters, the height measuring unit can be changed to display height in inches or feet/inches if preferred. Follow this procedure to change the unit of measure.

1. Open the battery cover.
2. Pull out one end of a battery.
3. Press and hold the **on/off** button while replacing the battery.
4. Dashes will appear on the display followed by "InIt". Continue holding the **on/off** button until your desired unit of measure appears. Units of measure will appear successively and are shown as: nnnn (cm), Inch and Feet
5. Release the **on/off** button when the desired unit is displayed, the touch bar will power off.

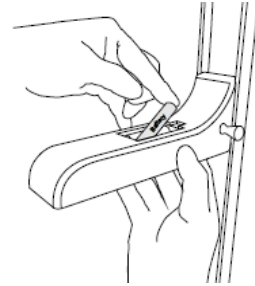<p>1. Power on the touch bar by pressing the on/off button. 52.2 will appear in the display.</p>	A line drawing of the touch bar attached to a vertical height rod. The touch bar is in the top position, and the display on its side shows the number "52.2".
<p>2. Depending on the position of the touch bar, slide it up or down the height rod channel past the reference mark. The present height value will appear.</p> <p>Note: Sliding the height rod upward will not measure height. Height is only calculated when the touch bar is moved downward.</p>	A line drawing of the touch bar on the height rod. A circular callout labeled "Reference mark" points to a specific point on the rod. Two vertical arrows, one pointing up and one pointing down, indicate the direction of movement. The display on the touch bar shows "180.0".
<p>3. Slide the touch bar to the top of the height rod. It is now ready for use. Position the patient with their back against the height rod and heels touching the wall. Ensure that the patient's posture is upright, and that the patient's head is straight and level. Slide the touch bar down until it rests gently on the patient's head. The patient's height will appear on the display.</p>	A line drawing of the touch bar at the top of the height rod. The display on the touch bar shows "180.0".

Notes:

1. The touch bar will automatically power off after 30 seconds of inactivity.
2. When not in use, leave the touch bar in the top position.
3. Keep magnets away from the device to prevent permanent damage and inaccurate measurements.

MAINTENANCE

Maintenance

The following page provides instructions for maintenance and cleaning of the 209HR. Maintenance operations, other than those described in this manual, should be performed by qualified service personnel.

Caution: Before first use, or after long periods of non-use, check the product for proper operation and function, and for any obvious damage or wear and tear. If the 209HR does not operate correctly, refer to qualified service personnel.

Cleaning

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

1. Clean all external surfaces with a clean damp cloth. Mild antimicrobial or antibacterial soap and water solutions may be used. Dry with a clean soft cloth. Do not use abrasive cleaners.
2. Do not use Isopropyl Alcohol or other solutions to clean the surface.

Replacement Parts

Replacement parts may be available if a piece of the height rod needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

Part #	Description
209HEAD	209HR headpiece
209ROD	209HR calibration rod
209HW	209HR hardware

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the product at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following:

negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 West 55th Street

McCook, IL 60525

PELSTAR, LLC

9500 West 55th Street • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2018
Health o meter® es una marca registrada de Sunbeam Products, Inc. usada bajo licencia.
Los productos Health o meter® Professional son fabricados, diseñados, y de propiedad de Pelstar, LLC.