Health o meter®

P r o f e s s i o n a l

Model Number: BABYHR Measuring Rod For Baby

USER INSTRUCTIONS

Thank you for your purchase of this Health o meter® Professional product. Please read this manual.

To avoid damaging when unpacking, do not use a box cutter, knife, scissors or any sharp object when opening the protective inner packaging. The following information details what you will find inside the carton as you unpack the parts for assembly. Set the carton aside for storage.

Parts List

(1) Measuring rod	(2) Plastic washers	(2) Wood screws M4 x 30mm
(1) User instructions	(2) Wall anchors	

Tools Required (Not Included)

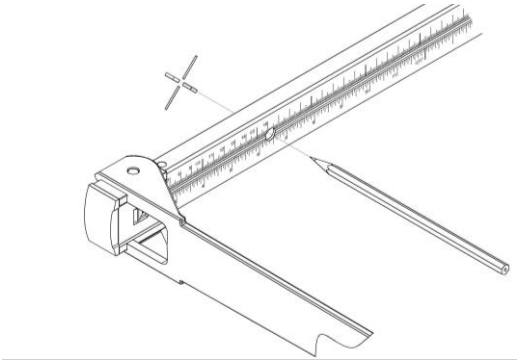
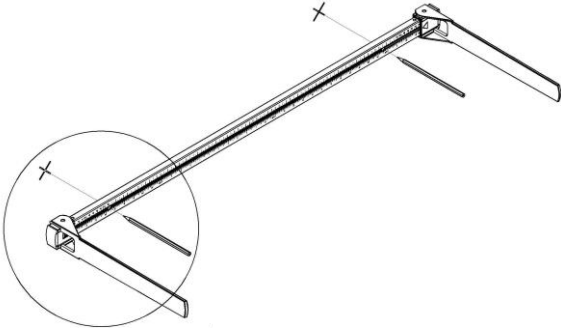
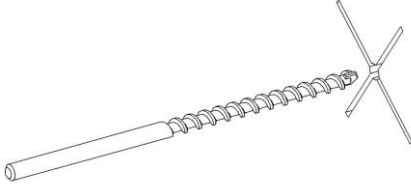
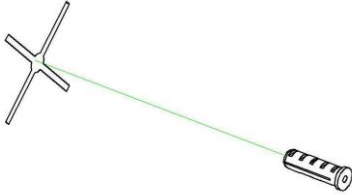
5.5 mm electric or manual hand drill, Phillips head screwdriver, marking pen or pencil.

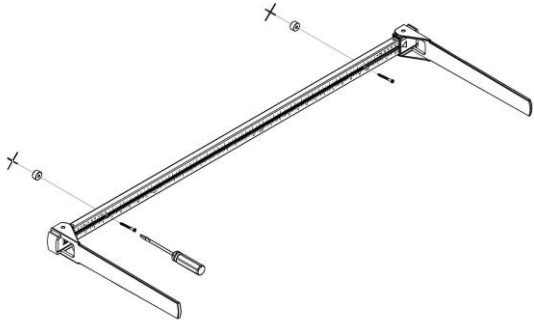
Cautions and Warnings

Before using the new measuring rod, please make time to read the following safety instructions:

- Follow the operating instructions.
- Use the assembly material supplied.
- When assembling, ensure that screws are tight and that the rod has been assembled correctly.
- Fold down the measuring slide after height has been measured to avoid risk of injury.

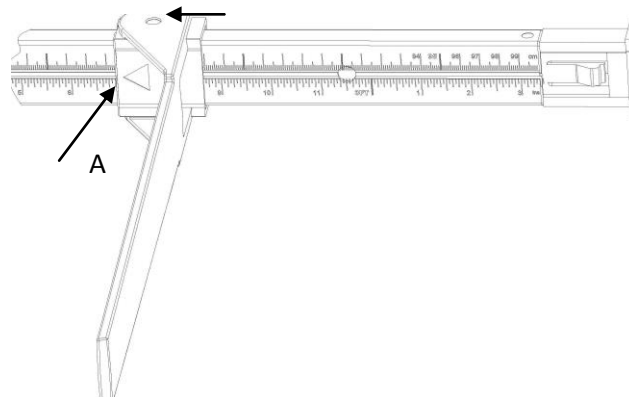
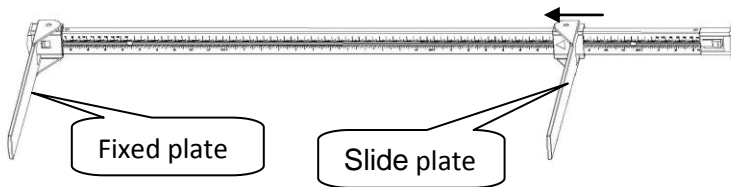
Wall Mount Instructions

Step	Instructions	Illustration
1	Position the rod along the wall, next to a patient exam table or bed. Mark a cross at an appropriate height via the hole on the rod.	
2	Align the cross with one hole then mark the position for the second hole horizontally.	
3	Using a 5.5mm drill bit, drill through both markings to make holes at least 30 mm deep for the wall anchors.	
4	Insert the wall anchors firmly into the hole on the wall.	

5	Screw height rod to the wall with the two wood screws and the two plastic washers placed on the back of the height rod.	
---	---	--

How to Use the Height Rod

1. Lay baby carefully on bed or exam table. Use the rod's fixed plate to touch the baby's feet.
2. Pull down the plate until the slide plate touches the top of the baby's head.
3. Measured result can be read at (A).

Specifications

Model	BABYHR
Range of measurement	7 –100 cm / 3 – 39 inch
Division of measurement	1mm / 1/16 inch
Net weight	600 g
Temperature range	10 – 40° C

LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, and the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters , such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600

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PELSTAR, LLC