## Care and Maintenance

1. An occasional wiping with a soft cloth and mild soap will keep your **Health o meter®** scale attractive. Abrasive cleaners may scratch the lens or dull the finish of the scale.
2. Since moisture may affect internal parts, the scale should be kept dry.
3. Your **Health o meter®** scale does not require oiling or lubrication of any kind.

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### 1 YEAR LIMITED WARRANTY

#### What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of 1 year. If product fails to function properly, return the product properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

#### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

#### What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or province to province or jurisdiction to jurisdiction.

#### To get Warranty Service

**Make sure you keep your sales receipt or document showing proof of purchase.**

Call 1 (708) 377-0600 to receive a return authorization. Attach proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar LLC  
Attention R/A# \_\_\_\_\_  
Return Department  
9500 W. 55<sup>th</sup> St.  
McCook, IL 60525

If your scale is not covered by warranty, or has been damaged, an estimate of replacement costs will be provided to you for approval prior to replacing.

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Professional

Pelstar, LLC  
9500 W. 55<sup>th</sup> St.  
McCook, IL 60525

Customer Service (708) 377-0600  
Customer Service FAX (708) 377-0601  
Technical Service: (800) 638-3722

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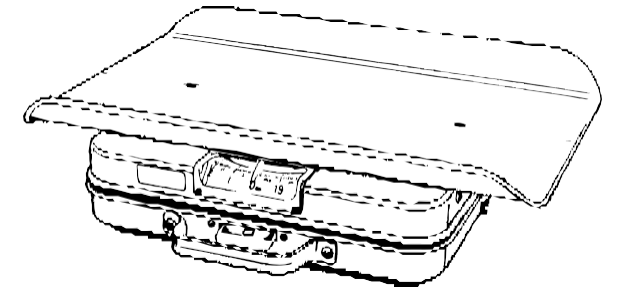
Model 386  
Modele 386  
Modelo 386

Please register your product at [www.hom scales.com](http://www.hom scales.com)

## Operating Instructions Mode d'emploi Instrucciones de Operación

Read & Save These Instructions  
Lire et conserver ces instructions  
Lea y guarde estas instrucciones

---

## Introduction

The Model 386 Baby Scale is a compact, portable unit that features the sensitive **Health o meter®** mechanism and an easy to read dial. It is simple to operate and with ordinary care will give years of accurate service.

For best performance, observe the following:

1. A pad or blanket used between the baby and the scale should be on the tray when the dial is adjusted to zero.
2. See that the scale is used on a hard level surface, preferably a large table for safety.
3. If scale is relocated, readjustment of zero may be required. Repeat procedure described below.

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---

## Assembly and Zero Adjustment

1. Remove all packing material from scale.
2. Place scale where it will be used (a hard level surface is best).
3. Make certain that the dial lock lever (below lens) is in an upright position.
4. When scale is first set up or when it has been carried and relocated, the scale should be weighed to seat the mechanism. It may also be necessary to adjust the dial to zero. Adjusting the zero should be done as follows:
  1. Press down on center of tray and remove pressure.
  2. Wait for dial to stop moving.
  3. If necessary, adjust dial to zero with adjustment knob (counterclockwise decreases reading) located under tray on rightside. Repeat weighing cycle until reading is stabilized.

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---

## Weighing

1. Make certain that the scale is adjusted to zero before weighing the baby. If a blanket or pad is used, it should be placed on the tray before adjusting the scale to zero.
2. Place baby on tray, wait for dial to stop moving, and read weight.
3. The dial lock lever can be pressed to hold the indicated weight when the baby is removed from the scale.
4. When the reading is recorded, the lever should be released and the dial will return to zero.

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©2018 Pelstar, LLC