Health o meter[®]

P r o f e s s i o n a l

Model Number: ADULTHR Height Measuring Rod

USER INSTRUCTIONS

Thank you for your purchase of this Health o meter[®] Professional product. Please read this manual

To avoid damaging when unpacking, do not use a box cutter, knife, scissors or any sharp object when opening the protective inner packaging. The following information details what you will find inside the carton as you unpack the parts for assembly. Set the carton aside for storage.

Parts List

- | | |
|---------------------------------|-----------------------------------|
| (1) Measuring rod | (2) Bolts with attached nuts |
| (1) User instructions | (6) Machine cap screws M4-0.7x6mm |
| (1) Green base- mounting flange | (2) Washers |

Tools Required (Not Included)

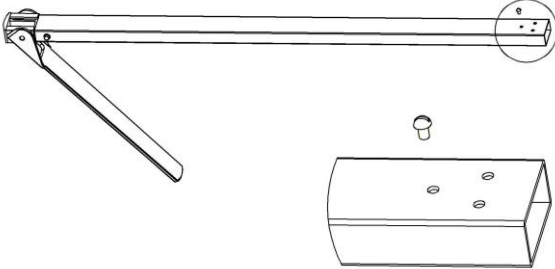
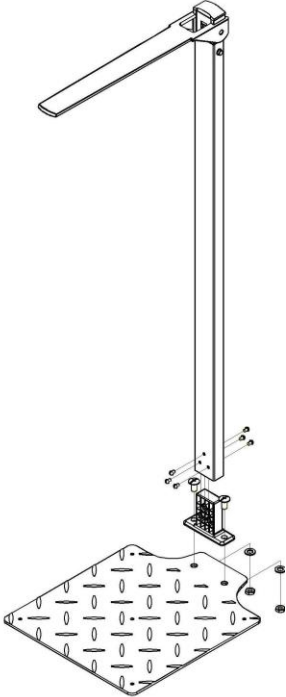
Phillips head screwdriver

Cautions and Warnings

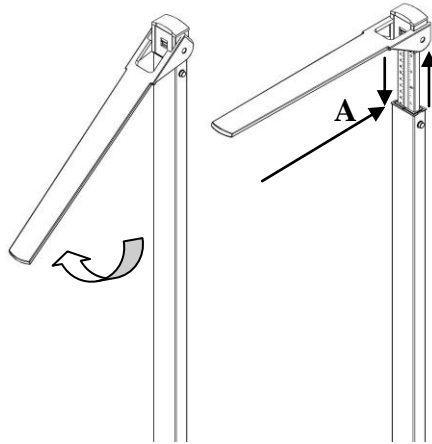
Before using the new measuring rod, please make time to read the following safety instructions:

- Follow the operating instructions.
- Use the assembly material supplied.
- When assembling, ensure that screws are tight and that the rod has been assembled correctly.
- Fold down the measuring headpiece after height has been measured to avoid risk of injury.

Assembly Instructions

Step	Instruction	Illustration
1	Remove the screw and bolt at bottom-back of the height rod. (This screw was only for preventing the middle rod from dropping down during transport.)	
2	Position the green base-mounting flange over the matching holes on footplate.	
3	Insert the bolts through the top of the flange into the footplate. Place the washer and nut onto the bolt on the underside of the footplate. Using a Phillips screwdriver, tighten the bolt securely to the footplate.	
4	Slide the base of the height rod onto the green base-mounting flange. Insert the 3 screws on each side of the height rod into the rod and securely tighten using a Phillips screwdriver.	

How to Use the Height Rod

1. Raise the headpiece to a horizontal position above the patient's head by pulling out the inner rod (the middle rod cannot be pulled out).
2. Position the patient against the height rod, ensuring the patient's posture is upright and the patient's head is straight and level.
3. Push the headpiece down until it rests on patient's head.
4. Measured result can be read at (A).

Specifications

Model	ADULTHR
Range of measurement	85 – 210 cm(33 ¹ / ₂ – 82 ⁵ / ₈ inch)
Division of measurement	1mm / 1/16 inch
Temperature range	10 – 40° C

LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, and the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge.

Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters , such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600

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PELSTAR, LLC